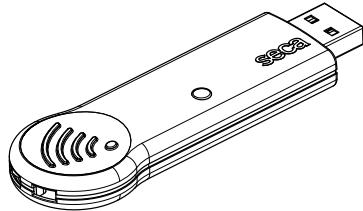
seca 456

(D)	Bedienungsanleitung und Garantieerklärung	4
(GB)	Instruction manual and guarantee	9
(F)	Mode d'emploi et garantie	13
(I)	Manuale di istruzioni e garanzia	18
(E)	Manual de instrucciones y garantía	23
(DK)	Betjeningsvejledning og garantibevis	28
(S)	Bruksanvisning och garanti	33
(N)	Bruksanvisning og garantierklæring	38
(FIN)	Käyttöohje ja takuu	42
(NL)	Bedieningshandleiding en garantieverklaring	47
(P)	Instruções de utilização e declaração de garantia	52
(GR)	Οδηγίες χειρισμού και εγγύηση	57
(PL)	Instrukcja obsługi i gwarancja	62
(RU)	Инструкция по эксплуатации и гарантийное свидетельство	67
(J)	使用説明書および保証	72
(CN)	说明书和保证	77
	Declaration of Conformity for USA and Canada	81

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1. INTENDED USE

The **seca 360° wireless USB Adapter 456** is mainly used in hospitals, medical practices and inpatient care facilities in accordance with national regulations.

The **seca 360° wireless USB Adapter 456** is an accessory for operating a PC in a **seca 360° wireless** network. The USB adapter allows wireless data transmission between the PC and other **seca 360° wireless** devices. The USB adapter only works on PCs with seca PC software installed.

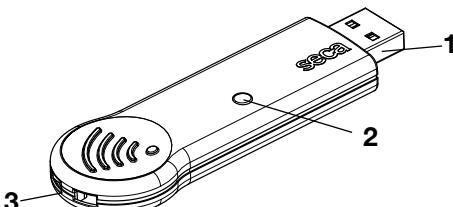
2. SAFETY INFORMATION

- ▶ Follow the instructions for use.
- ▶ Keep the user manual in a safe place.
- ▶ Follow the instructions in your PC manual and those for the **seca 360° wireless** devices used.
- ▶ To prevent malfunctions, do not operate or store the device in an environment in which it is exposed to high humidity, vibrations, dust, heat or direct sunlight.
- ▶ Never immerse the device in disinfectant or other liquids.
- ▶ Do not start up the device immediately after transporting it. Severe temperature fluctuations and changes in humidity may lead to moisture forming on the device. This may damage both the device itself and the PC to which it is connected.
- ▶ Make sure the device cannot be inadvertently disconnected from the PC to prevent malfunctions and to avoid improper usage.
- ▶ Keep other medical electrical devices a minimum distance of approx. 1 meter away to prevent incorrect measurements or wireless transmission interference.
- ▶ Keep HF equipment such as cell phones and screens a minimum distance of approx. 1 meter away to prevent incorrect measurement or wireless transmission interference.
- ▶ The actual transmission output of HF equipment may require minimum distances of more than 1 meter. For details, go to www.seca.com.

- ▶ If the device has to be operated in the immediate vicinity of individual or stacked medical electrical devices, ensure that the devices are working properly.
- ▶ Do not use extension cables. These may impair the EMC immunity of the device.
- ▶ Do not open the device housing. This may lead to damage to the device. No warranty will be provided by seca in such cases.

3. OVERVIEW

3.1 Controls

No.	Element/symbol	Function/meaning
1	USB stick	Plugged into a free USB port on the PC
2	LED	• Green: USB adapter correctly detected by PC • Blue: data being transmitted
3	Security loop	Make sure the USB adapter cannot be inadvertently disconnected from the PC

3.2 Identification on the device and the type plate

Text/symbol	Meaning
REF	Model number
SN	Serial number, consecutive
	Follow instructions for use
	Device is equipped with a radio module, data are received and transmitted wirelessly
FCC	Symbol of the US Federal Communications Commission (FCC)
FCC ID	Device license number from the Federal Communications Commission (FCC)

Text/symbol	Meaning
IC	Device license number from Industry Canada
	Do not dispose of device with household waste

3.3 Identification on the packaging

	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. moisture for transport and storage

4. OPERATION

The USB adapter is supplied ready to use.

1. Plug the USB adapter into a free USB port on your PC.
2. Make sure the USB adapter cannot be inadvertently disconnected from the USB port.
3. Set up a wireless network as described in the Administrator's Manual for the seca PC software.

5. HYGIENE TREATMENT

- Use a soft cloth dampened with mild soapsuds to clean the surfaces of the device.
- Disinfect the device as required with a commercially available disinfectant, e.g. with 70 % isopropyl alcohol.
- This device may not be sterilised.

6. TECHNICAL DATA

Technical data seca 360° wireless USB Adapter 456	
Measurements • Depth • Width • Height	approx. 27 mm approx. 85 mm approx. 10 mm
Net weight	0.01 kg

Technical data seca 360° wireless USB Adapter 456	
Ambient conditions, operation <ul style="list-style-type: none">• Temperature• Air pressure• Humidity	+10 °C to +40 °C (50 °F to 104 °F) 700 hPa - 1060 hPa 30 % - 80 %, no condensation
Ambient conditions, storage <ul style="list-style-type: none">• Temperature• Air pressure• Humidity	-10 °C to +65 °C (14 °F to 149 °F) 700 hPa - 1060 hPa 0 % - 95 %, no condensation
Ambient conditions, transport <ul style="list-style-type: none">• Temperature• Air pressure• Humidity	-10 °C to +65 °C (14 °F to 149 °F) 700 hPa - 1060 hPa 0 % - 95 %, no condensation
Power supply	5 V via PC's USB slot
Wireless transmission <ul style="list-style-type: none">• Frequency band• Transmission power• Channel spacing• Channels• Data rate	2,433 Ghz -2,480 GHz < 10 mW 199,951 kHz 0-99 50 kBaud

7. DISPOSAL

Do not dispose of the device with household waste. The device must be disposed of properly as electronic waste. Comply with the national provisions applicable in your country. For further information contact our service department at:

service@seca.com

8. WARRANTY

We offer a two-year warranty from the date of delivery for defects attributable to faulty material or poor workmanship.

9. DECLARATION OF CONFORMITY

seca gmbh & co. kg hereby declares that the product meets the terms of the applicable European directives. The unabridged declaration of conformity can be found at: www.seca.com.

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DECLARATION OF CONFORMITY FOR USA AND CANADA

seca
seca 456

FCC ID: X6T182A01

IC: 8898A-182A01

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions. (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE:

This device complies with Part 15 of the FCC Rules and with RSS-210 of Industry Canada. Operation is subject to the following two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.

NOTE:

Changes or modifications made to this equipment not expressly approved by seca may void the FCC authorization to operate this equipment.

NOTE:

Radiofrequency radiation exposure Information:
This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with minimum distance of 1 m between the radiator and your body. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Medical Measuring Systems and Scales since 1840

17-10-05-328c_04-2019B

seca gmbh & co. kg

Hammer Steindamm 3–25

22089 Hamburg · Germany

Telephone +49 40 20 00 00 0

Fax +49 40 20 00 00 50

info@seca.com

seca operates worldwide with headquarters
in Germany and branches in:

seca france

seca united kingdom

seca north america

seca schweiz

seca zhong guo

seca nihon

seca mexico

seca austria

seca polska

seca middle east

seca brasil

seca suomi

seca américa latina

seca asia pacific

and with exclusive partners in
more than 110 countries.

All contact data under www.seca.com

seca[®]
Precision for health