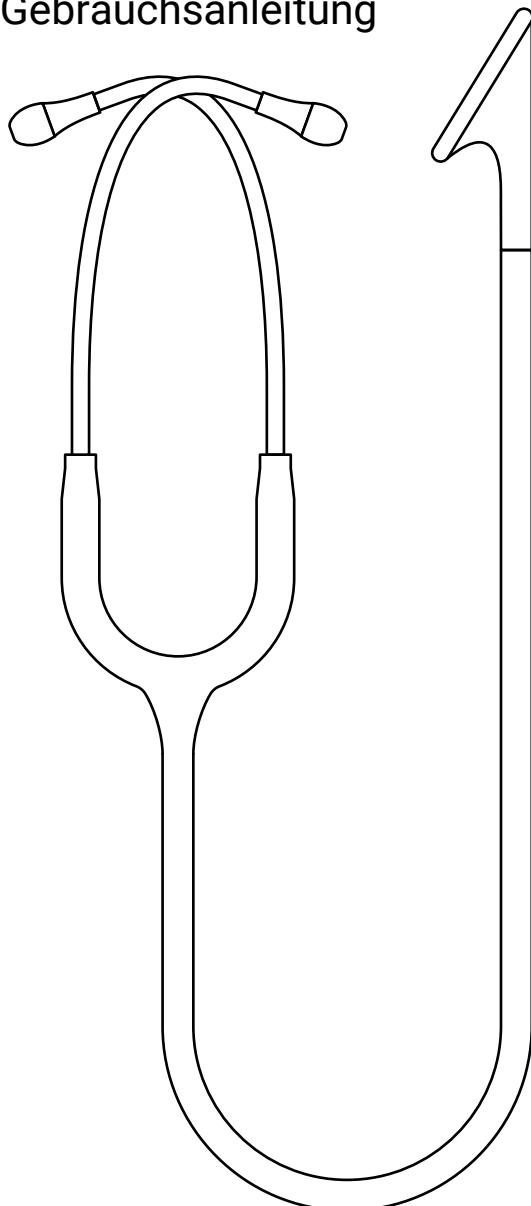
# Tűlip

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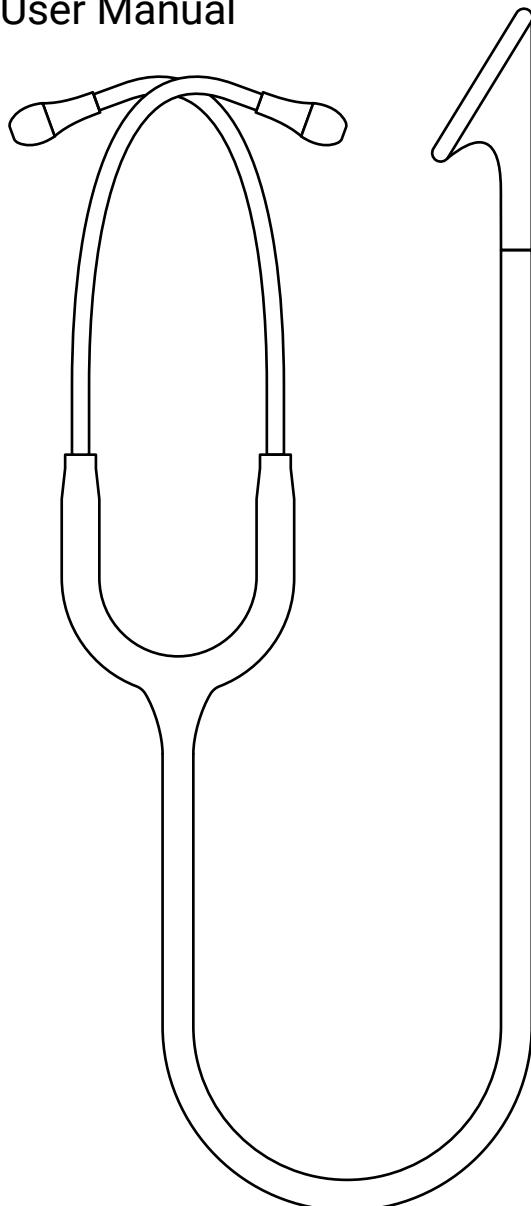
## User Manual

### My intended use

I, Tūlip, may only be used by authorised and qualified professionals.

I am an acoustic diagnostic tool and help you to hear body sounds better, especially those of the heart and lungs.

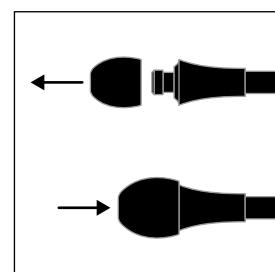
Read the following instructions closely and we will be friends for a long time.

### Inappropriate use

Inappropriate use means you use me differently to what this user manual states. My manufacturer is not responsible for damage caused by inappropriate use. In this case only you, the user, will be responsible. So please be careful!

### Operation

In order to use me correctly, put my ear tips into your ears first. You will know that the ear tips are properly placed when they point towards the outer ear canal. If this isn't the case, test which size suits you best to ensure optimal noise cancellation.

The ear tips come in three different sizes (S, M & L). To switch these, simply remove the previous ear tips and slide the new ones onto the adapter's end.

You can adjust my ear tubes' tension by pulling them apart or squeezing them together. Pulling apart the ear tubes is particularly helpful if you feel that the ear tips are sitting too tight in your ear canal.


Check my membrane by gently rubbing your finger over

it before applying it to the patient. Please wear gloves to avoid transferring bacteria to the membrane.


If you can't hear any noise, make sure my components are assembled correctly. I'm ready to go.

For auscultation, place my chest piece on the body area you want to examine. Apply gentle pressure to the chest piece to hear low frequencies. Increase the pressure to listen to all frequencies at a higher volume. My chest piece should be used directly on the skin. This is the only way to guarantee that rustling does not distort the quality of auscultation.


### Changing the membrane

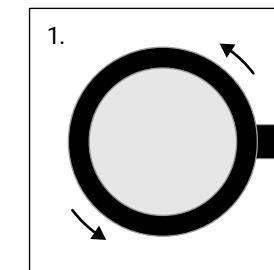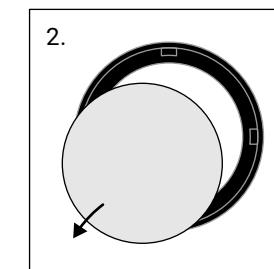
Release the ring by turning it counterclockwise.

Remove the old membrane from the ring.

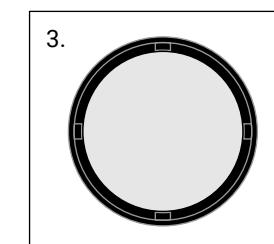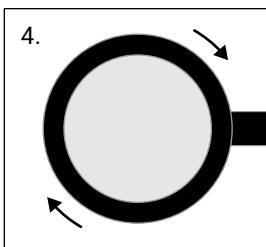Place a new membrane into the ring so that it lays flat.

Tighten the ring and membrane onto the chest piece so that they fit tightly.

Please do not be afraid that the ring may break when tightening. If it's not tight enough, there's a possibility that it will come off the chest piece.

## Safety notice

Please do not use me without membrane or ear tips. Never put my ear tips too deep into your ear canal or use me near loud sound sources. This can lead to hearing damage!

I am not a children's toy. I contain small, swallowable parts (ear tips), so please make sure to keep me out of children's reach.

## Materials

I look so elegant because my chest piece is made of aluminium. My fastening ring looks the part, too, as it is made of nylon. The ear tips are made of silicone.

My tube is made of PVC. This material becomes hard when it comes into contact with skin for a long time. Therefore, please avoid contact with the skin as much as possible.

## Cleaning, disinfection and storage

Clean and disinfect me regularly. This is of great importance – it serves both to protect the patient, the user and third parties, but also to maintain my quality.

### Manual cleaning

Please cleanse me, especially my chest piece and

membrane, after each patient. It's really easy: Use a cleaning cloth with 70% isopropyl alcohol or soapy water. Keep me away from other liquids or solvents.

I also don't like being dipped in liquids, so please refrain from this.

When cleaning, don't forget to pay particular attention to my tube. Care for it with special plastic care products so that it does not harden over time.

For a thorough cleaning, please remove my individual parts. Before you reassemble me, all parts and surfaces should be dry.

In addition, you should disinfect me regularly. If I am particularly dirty, you can use a 2% bleach solution. Please note that excessive use can lead to discolouration of the tube.

### Sterilisation

If you want to enjoy me for a long time, please don't sterilise me.

### Storage

After use, put me back in the packaging or in a safe place so that I don't get any defects on my chest piece!

Also, do not expose me to strong UV radiation or heat, since they can lead to my material fading or hardening.

## Maintenance and warranty

You can replace my ear tips or membrane at any time.

The warranty for the entire product expires when using spare parts that were not purchased on the DocCheck shop.

If you use me as intended and follow these instructions for use, the warranty for manufacturing and material defects is two years, starting from the date of sale. Obvious misuse and accidental damage are excluded. Cosmetic damage, such as scratches on the surface, are also excluded from the warranty.

## Check and functional test

Before use, please check that all my parts such as membranes or ear tips are assembled and securely in place.

Visual inspection for damage and wear is also useful.

## Disposal

Please only dispose of me in accordance with local / regional / international guidelines.

## Manufacturer

DocCheck Shop GmbH

Vogelsanger Str. 66, 50823 Cologne, Germany

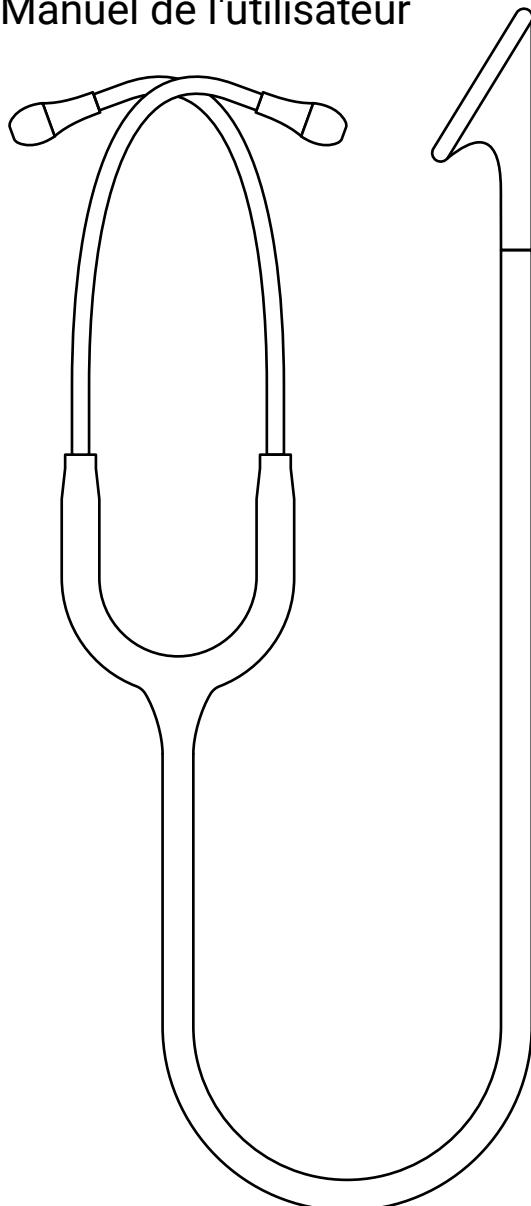
[doccheckshop.com](http://doccheckshop.com)

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