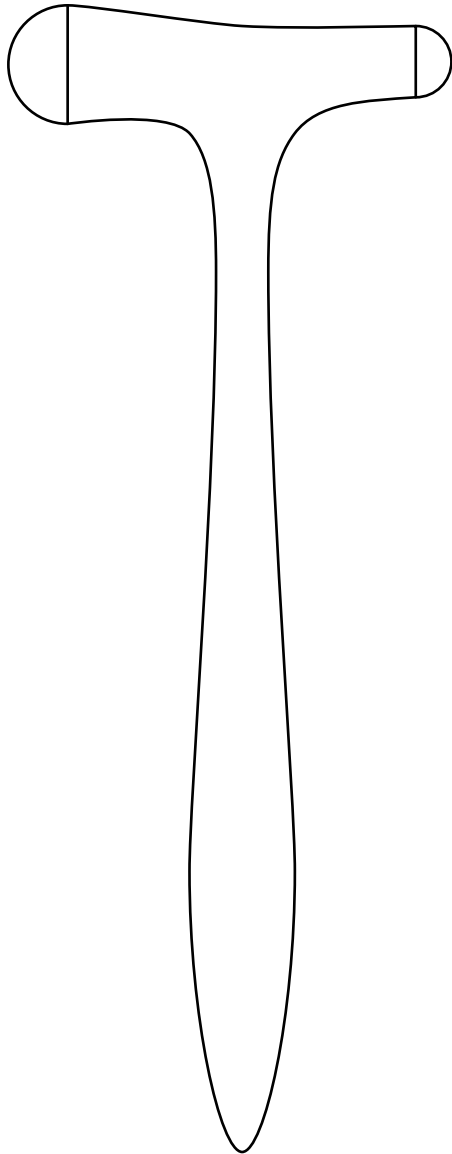
# Clôppe

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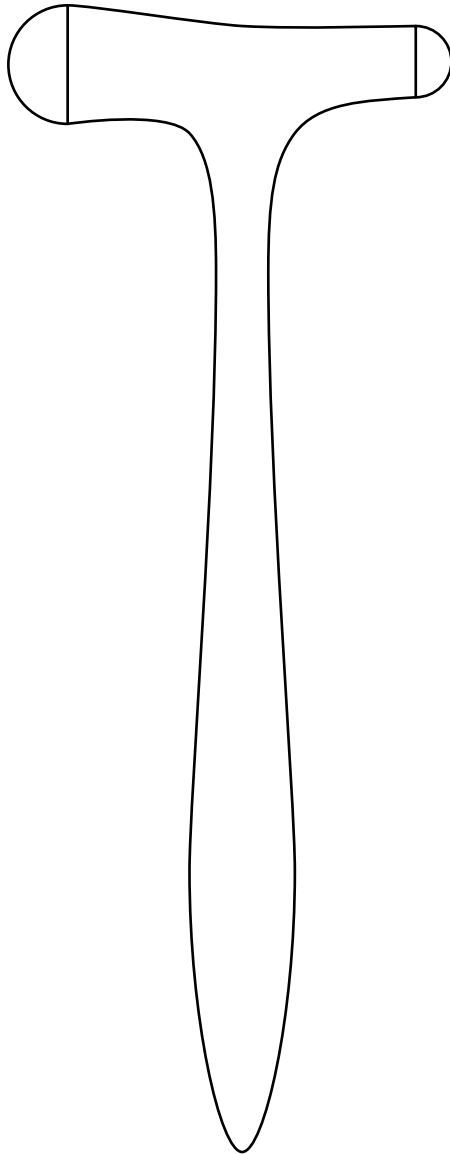
 NFC-Chip

 Mein Held  
[doccheck.com](https://www.doccheck.com)

# Clôppe

## User Manual

### My purpose

I, Clôppe, am only intended to be used by healthcare professionals for medical diagnosis. I am a diagnostic instrument for percussion (medical tapping) and for triggering reflexes. And yes, I am reusable.

### Unsuitable usage

If you use me differently than described in these instructions, in a way that does not correspond to my purpose, my manufacturer is not liable for damage(s) resulting from this improper use. In this case, you, the user, bear the risk. So be careful!

### Usage

You strike me from the wrist, swinging loosely, with one of the two hammer heads on the corresponding tendon or the periosteum into which the tendon radiates.

Please strike carefully and not too hard. This causes a reflex contraction, i.e. a contraction of the relevant muscle. But, I have a second role too! As you can see, I taper at the bottom of the handle.

With this tip you can check the Babinski reflex on the sole of the foot.

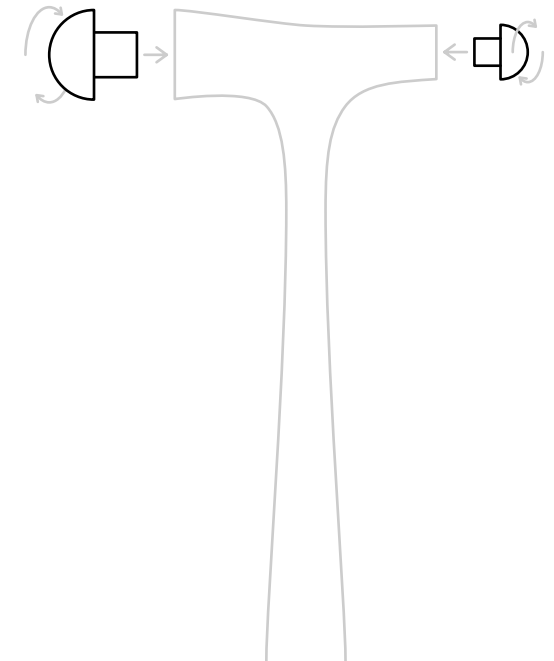
### Important safety advice

Because of my small parts that can be swallowed (rubber heads), you should not put me in the hands of children under 3 years of age! Believe me, toddlers love to nibble off my attractive rubber parts.

### Materials

I have such a beautiful shape because I'm completely made of aluminium. I have a small NFC chip hidden in my body, which is covered by a plastic capsule. My rubber heads are made of silicone.

### Removing or replacing the hammer heads

My rubber heads are held in place by a screw connection. If you want to remove them, turn them counter-clockwise and pull them out gently. If you want to unscrew the heads or tighten them, turn them clockwise with pressure. Please do this with a lot of feeling so that the heads do not break off.

## Cleaning, disinfection and storage

### Manual cleaning

Please ensure to clean me regularly. It really is quite easy:

Remove my rubber heads as described above and thoroughly and wipe all my individual parts with a damp cloth. My manufacturer recommends cleaning wipes with 70% isopropyl alcohol or soapy water. Please do not put me back together until I am fully dry.

If you use other cleaning agents or disinfectants, please test if they are compatible first otherwise you could damage my pleasant appearance.

Frequent cleaning has little effect on me, and I love to be clean!

### Sterilisation

You CANNOT sterilise me. I will break, which would be sad for us both.

### Storage

I am an unpretentious guy. Just store me dry and at room temperature. But I also have nothing against cool rooms. I don't like excessive UV exposure and heat, which can lead to my material

becoming bleached, brittle or hardened.

## Maintenance and warranty

If parts of mine are damaged, you can sort them out or replace them (rubber heads).

If you use me as intended and follow these instructions for use, the warranty on manufacturing and material defects is two years, starting with the date of sale. Obvious misuse and accidental damage are excluded. So please don't use me to drive nails into the wall.

## Control and functional tests

Please check before use that my removable parts (rubber heads) are firmly seated. A visual inspection for damage and wear is also useful.

## Disposal

Please only dispose of me in accordance with local/regional/international guidelines.

## Manufacturer

DocCheck Shop GmbH  
Vogelsanger Str. 66, 50823 Cologne, Germany  
[doccheckshop.com](https://www.doccheckshop.com)

## Symbol clarification

Medical device

CE mark

Manufacturer data (address)

Production date

Article number

Batch

No latex / natural rubber content

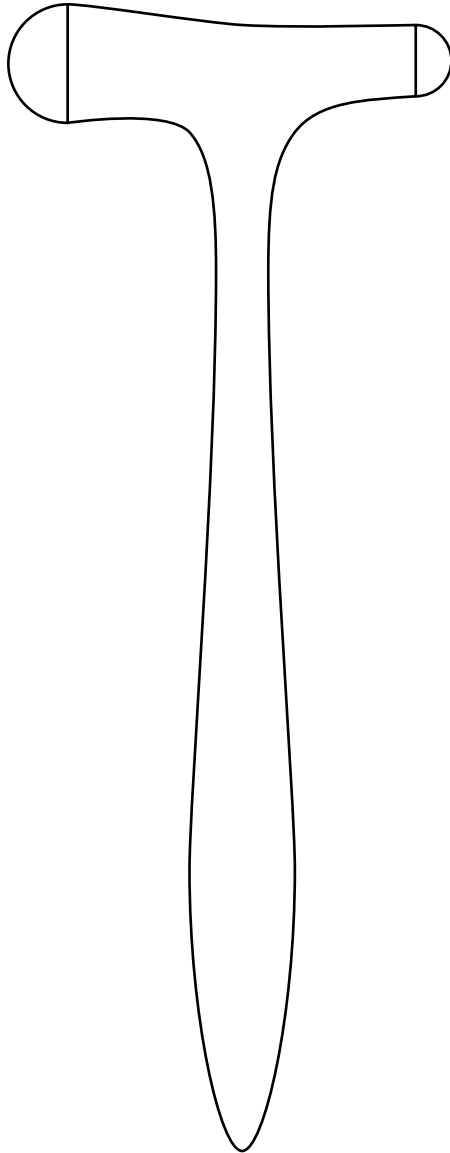
NFC-chip

My hero  
[doccheck.com](https://www.doccheck.com)

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