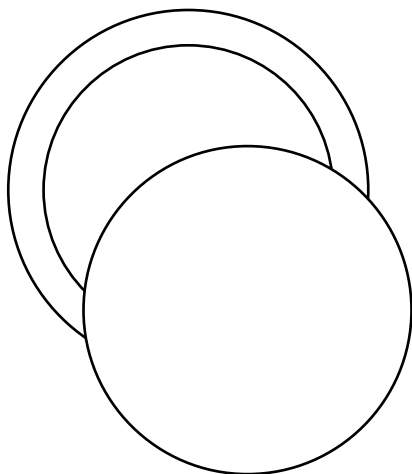
# Drums

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für

**DocCheck "Tülip"**

**Littmann® Master Classic II™**

**Littmann® Classic II Kinderstethoskop**

**Littmann® Classic III™**

**Littmann® Cardiology IV™**

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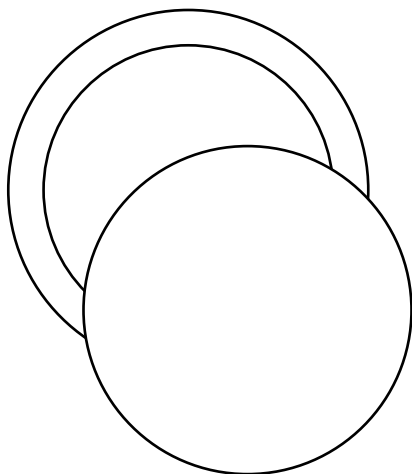
Mein Held  
[doccheck.com](https://www.doccheck.com)

# Drums

## User Manual

for

**DocCheck "Tülip"**

**Littmann® Master Classic II™**

**Littmann® Classic II Kinderstethoskop**

**Littmann® Classic III™**

**Littmann® Cardiology IV™**

Please check the packaging to see which model I am suitable for.

### My intended use

Do you want to give your stethoscope a makeover?  
Then use me to replace your basic membrane!

Please note that I do not work with each and every stethoscope. Only use me with the models mentioned on my packaging.

Only authorized and qualified health professionals are allowed to use me.

### Inappropriate use

If you stray from this manual's directions, then you will not use me correctly. My manufacturer is not liable for any damages that result from inappropriate use. In this case, you alone take responsibility. So please be careful!

### Notes

#### General notes

For using your stethoscope, please refer to its user manual.

Note to the user and/or patient: All serious incidents related to me must be reported to DocCheck Shop GmbH and the competent authority of the Member State.

#### Notes on printed fiberglass membranes

Even though my manufacturer has taken various measures to keep my colors as vibrant as possible, environmental factors may impact their longevity.

Please replace me after 30 days of use or once I show signs of discoloration or damage.

### Notes on antimicrobial copper membranes

Studies show that I can reduce bacteria and microbial activity. This is thanks to my copper surface, which reacts to microbes as soon as they touch me.

I reduce the following bacterial strains by 99.9 % within 24 hours:

- MRSA
- E. coli
- Staphylococcus aureus

However, I am of course not a substitute for responsible, hygienic treatment of your patients!

I'm 99.9 % copper. A small proportion of the population may be allergic to this material. Do not use me if you suspect that you or your patient has such an allergy.

Be careful not to damage my surface with sharp objects. That can shorten my lifespan.

I am electrically conductive.

My surface colour could change after some time because it reacts to oxygen. My beautiful copper sheen then changes to a darker brown. This is a clear signal to replace me.

### Notes on aluminum membranes

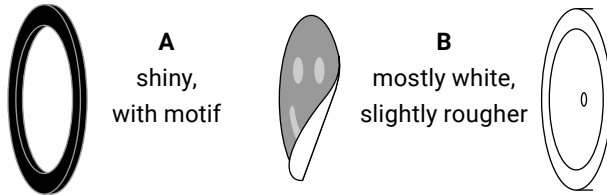
A small percentage of people might be allergic to my material. Do not use me if you suspect that you or your patient might be allergic.

I am electrically conductive.

### Montage


I have two sides – make sure, depending on the variant, which side faces outward after assembly:

#### Printed fiberglass membranes:


#### Antimicrobial copper membranes:

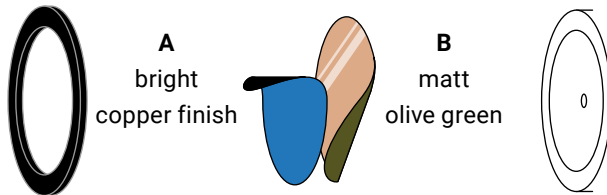
Remove the blue protection film first!


#### Engraved aluminium membranes:

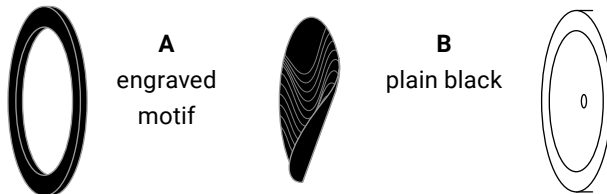
#### Sound-optimised aluminium membranes:

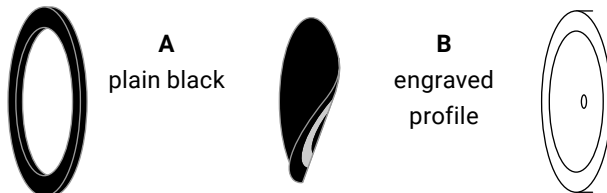
### Assembly for "Tulip"

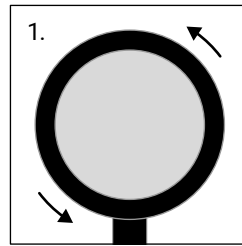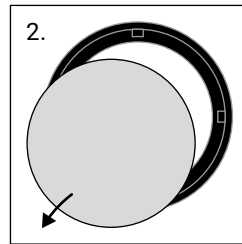
Detach the locking ring by turning it counterclockwise.

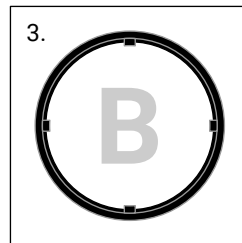Take out the old membrane from the ring.

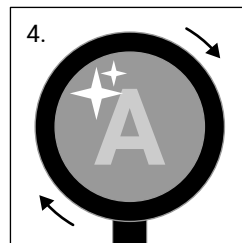
Place me in the ring so that I lie flat and my back side **B** is facing you.


Screw me and the ring on tight, and I will not budge anymore.  
Done!


### Assembly for Littmann

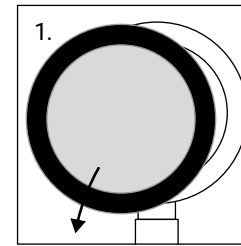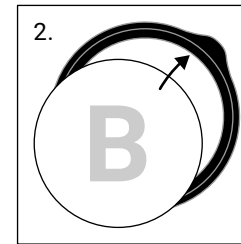
Remove the old membrane from your stethoscope.

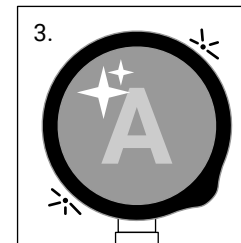Take the appropriate supplied retaining ring. Place me inside so that I lie flat and my back side **B** is facing you.

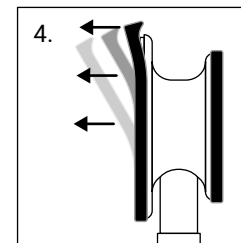Click me with the ring on your stethoscope's chestpiece.  
You're done!

If you want to remove me, lift the ring by the tab and pull it from the stethoscope piece by piece.

Please verify that your stethoscope works correctly every time you change your membrane! Due to the material I am made from, my acoustic performance may vary from your original membrane. Please familiarize yourself with my qualities before using me for any diagnostic purposes.

## Cleaning & Disinfection

Please clean and disinfect me regularly. This is not only important for ensuring I stay intact, but also for protecting your patients, yourself, and others.

I can tolerate isopropyl alcohol just like any basic stethoscope membrane. To remove any dust or dirt, please clean my surface with a smooth cloth and some isopropyl alcohol.

## Warranty

If you use me as intended and follow these instructions, the warranty for manufacturing and material defects is two years, starting from the date of purchase. Obvious misuse and accidental damage are excluded. Cosmetic damages, such as scratches on the surface, are also excluded from this warranty.

## Disposal

Please refer to local / regional / international guidelines before throwing me away.

## Manufacturer

DocCheck Shop GmbH  
Vogelsanger Str. 66, 50823 Cologne, Germany  
[doccheckshop.com](https://www.doccheckshop.com)

## Explanation of symbols

Medical device

CE marking

Manufacturer data (address)

Production date

Item number

Batch

Does not contain latex/natural rubber

Contains aluminium

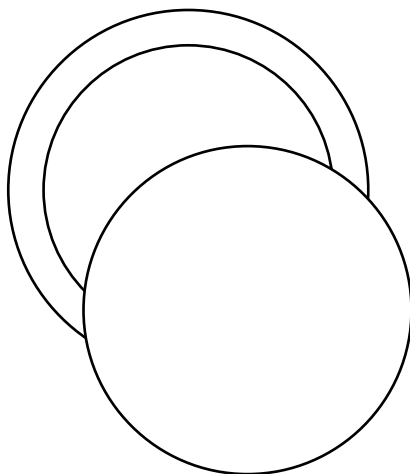
Contains copper

My hero  
[doccheck.com](https://www.doccheck.com)

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