EN

DE

FR

IT

ES

HU

# bloom thyroid test

## thyroid-stimulating hormone (TSH)

instructions for use

Download the  
bloom app  
to get started

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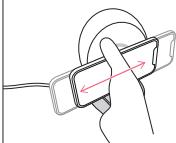
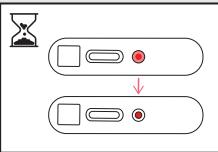
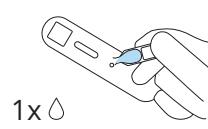
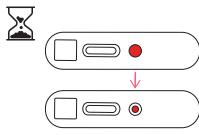
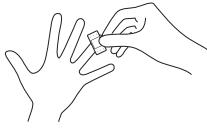
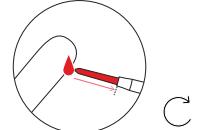
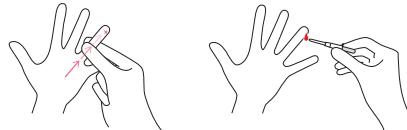
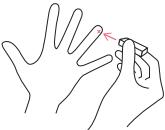
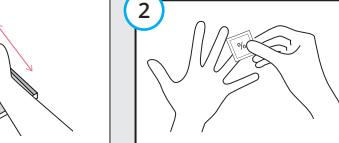
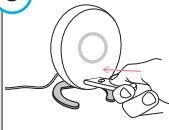
**[REF]** TSH-REF-2-1.X (5 tests)

## Table of contents

<b>1. General information .....</b>	<b>3</b>	<b>3.2. General notes on safety .....</b>	<b>5</b>
1.1. Intended use.....	3	3.3. Limitations of use.....	5
1.2. Terms and definitions .....	3	<b>4. Operating steps .....</b>	<b>5</b>
<b>2. Bloom System .....</b>	<b>3</b>	4.1. Preparation.....	5
2.1. Overview of the Bloom System .....	4	4.2. Blood sample collection .....	6
2.2. Bloom Thyroid Test.....	4	4.3. Process sampling and report collection.....	7
2.2.1. Principle of the method .....	4	4.4. Device cleaning.....	8
2.2.2. Test performance.....	4	<b>5. Storage .....</b>	<b>8</b>
2.2.3. Interferents.....	4	<b>6. Troubleshooting .....</b>	<b>8</b>
2.2.4. Components .....	4	<b>7. Symbol information .....</b>	<b>8</b>
<b>3. Safety Regulations.....</b>	<b>4</b>	<b>8. Contact information.....</b>	<b>8</b>
3.1. Disclaimer .....	4		

### 1. General information

Thyroid stimulating hormone (TSH) is produced by the pituitary gland, a small gland located in the brain. TSH stimulates the thyroid to release thyroid hormones triiodothyronine (T3) and thyroxine (T4). These thyroid hormones play an important role in the function of tissues and organs and are also involved in the regulation of body temperature and metabolism. As the release of TSH is very sensitive to small changes of thyroid hormone levels, a TSH test can be used to assess the status of the thyroid.

#### 1.1. Intended use

The Bloom Thyroid Test is a single-use self-testing kit for manual in-vitro diagnostic quantitative measurement of thyroid stimulating hormone in capillary blood, used to screen for hypothyroidism. It is intended for self and near patient testing for individuals 18 years old and above.

### 1.2. Terms and definitions

- **Warning:** Points out a situation where a potentially hazardous condition exists that could result in a critical or catastrophic situation such as death or severe long-term injury if not avoided.
- **Caution:** Points out a situation where a potentially hazardous condition exists that could result in a serious situation such as short-term injury or impairment requiring medical intervention to correct if not avoided.
- **Information:** Points out a situation where a potentially hazardous condition exists that could result in a not serious situation, such as a slight inconvenience, if not avoided.

### 2. Bloom System

The Bloom Thyroid Test only works together with the Bloom App and Bloom Lab. These form the "Bloom System". To get started,

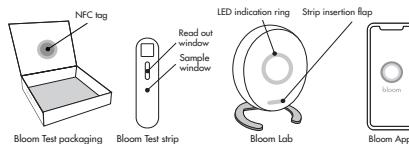
download and install the Bloom App on your smartphone. Ensure you have access to a Bloom Lab.

#### Warning

**Do not use the Bloom System as a diagnostic device.** Using the information provided by the Bloom System for diagnosis can lead to mistreatment and severe consequences. Results should be reviewed by a medical practitioner.

## 2.1. Overview of the Bloom System

### Bloom System.

The Bloom System is a medical testing system. It provides screening and monitoring of target markers and medical guidance. It combines lateral flow immunoassays (Bloom Test strips), an established diagnostics technology, with an electronic reader for test strip analysis (Bloom Lab) and an app for displaying and interpreting results (Bloom App).

## 2.2. Bloom Thyroid Test

### 2.2.1. Principle of the method

The Bloom Thyroid Test strip is a lateral flow immunoassay based on colloidal gold nanoparticles. It consists of a sample window, a conjugate pad containing anti-TSH antibodies bound to gold particles, a test line with anti-TSH antibodies and a control line with anti-IgG antibodies. When blood and buffer are applied, the mixture migrates by capillary force along the test strip. The TSH present in the blood will bind to the anti-TSH antibodies conjugated to gold particles. These immune complexes are then captured by the anti-TSH antibodies in the test line, creating a colored line detected by the Bloom Lab. The control line is detected by the Bloom Lab to verify whether the test has run correctly. Results are received in the form of a Bloom Thyroid Report via the Bloom App.

### 2.2.2. Test performance

Limit of Detection	Quantifiable Range	Precision	Sensitivity*	Specificity*
1.5 mIU/L	2.6 - 30.0 mIU/L	14.1 %	89.0 %	90.1 %

\*determined based on a cut-off of 4.5 mIU/L.

### 2.2.3. Interferents

Acetaminophen, ampicillin-Na, hGH and doxycycline hydiate will interfere with the test result. Unusually high levels of heterophile antibodies or highly lipemic samples might interfere with the test result. Cefoxitin Na, citalopram, fluoxetine HCl and metronidazole might slightly interfere with the test result. No interference was observed from rheumatoid factors up to a concentration of 510 IU/ml.

### 2.2.4. Components

The Bloom Thyroid Test includes one set of instructions for use and enough material for either one or five tests (indicated on the outer package). Material for one test comprises of the following:

1		1 x Bloom Thyroid Test strip in aluminium pouch		
2		1 x Alcohol wipe for finger cleaning		Servoprax GmbH, Wesel, Germany
3		1 x Sterile safety lancet for finger pricking		Medipurpose Pte. Ltd., Plaza, Singapore <b>STERILE</b>
4		1 x Plaster for incision patching		BSN Medical GmbH, Hamburg, Germany
5		1 x Capillary tube (lithium heparin) for blood collection		Sarstedt AG & Co., Nümbrecht, Germany
6		1 x Buffer		
7		1x Disinfecting cleaning wipe for Bloom Lab		B. Braun Medical AG, Sempach, Switzerland

Caution: Do not use equipment other than that specified by Bloom Diagnostics to guarantee the proper function of the device.

## 3. Safety Regulations

### 3.1. Disclaimer

- Do not take the Bloom Thyroid Test if you are pregnant or have given birth in the past 12 months, have or have had thyroid cancer or pituitary adenoma, or if you find yourself in an acute

situation of emergency or distress. In case of doubt, always consult with your medical practitioner.

- The Bloom Report displays the result of the Bloom Thyroid Test and additional information. It is not intended to diagnose or inform about the treatment of medical conditions. Never avoid, put off, change or start treatment based on the content of the Bloom Report. In order to receive a diagnosis and treatment, consult a medical practitioner. In case of health concerns seek medical attention immediately.
- With the exception of the measured test value, all information presented in the Bloom Report is the result of Bloom Diagnostics' research conducted with literature available at the time of development. All content provided is subject to change depending on scientific discoveries and advances, and might not apply globally. While the information in the test report is provided with great care, Bloom Diagnostics cannot guarantee its accuracy or completeness.

### 3.2. General notes on safety

- The validity of the Bloom Thyroid Report relies on the correctness of the answers. Once submitted, the answers cannot be modified.
- Do not use the Bloom Thyroid Test if you are under the effects of alcohol or drugs.
- Do not perform multiple Bloom Tests at the same time. A new test should only be started once the first has been fully completed.
- Do not use the Bloom Thyroid Test in an environment with high humidity, such as the bathroom.
- Unpack the individually sealed components of the Bloom Thyroid Test directly before use.
- Do not use the Bloom Thyroid Test if its packaging or any of its sub-packaging is damaged or show evidence of tampering.
- The Bloom System should not be used with any hardware or software that is not manufactured or developed by Bloom Diagnostics for the explicit purpose of being used with the Bloom System. This includes using generic power cables or uncertified or generic test strips.
- Never attempt to modify, deconstruct, damage, or tamper with any elements of the Bloom System as this can result in malfunction and will make the support service and product's warranty null and void. The Bloom Thyroid Test should not be used when it is known to be malfunctioning, broken, defective, or contaminated.
- The Bloom Lab and the entire Bloom System should only be purchased from authorized retailers and never second-hand.
- This device may be used by people with reduced mental capabilities provided that they are supervised and are fully aware of the subsequent risks of use.

### 3.3. Limitations of use

- This test can not be used for screening of hyperthyroidism or secondary/tertiary hypothyroidism.
- Your TSH levels might be below the device's detection limit and indicated with a "<" (normal reference range: 0.4-4.5 mIU/L). See "2.2.2. Test performance" for more information.

### 4. Operating steps

Always follow the Bloom System's operating steps in a sequential order.

**Caution:** When using the Bloom Thyroid Test containing five tests:

- Keep the test packaging until you have performed all five tests, in order to initiate a test.
- Ensure the test kit is only handled by one individual.

#### 4.1. Preparation

Before you start, ensure that you have:

- Checked the use-by-date has not been exceeded (located on the package in the format YYYY-MM-DD);
- A smartphone with a working internet connection and the Bloom App installed;
- A working Bloom Lab, set up according to its instructions for use;
- Checked the Bloom Thyroid Test package components for completeness and damage;
- A clean, stable workspace and are seated while carrying out the test.

#### Step 1: Connect Bloom App and Bloom Lab

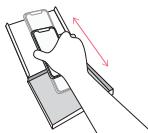
- Log in to the Bloom App and tap on the "+" button in the menu bar to initiate a new test.
- Swipe the phone across the light ring of the Bloom Lab to connect.

**Information:** Successful pairing is indicated by a white heart beat pattern on the Bloom Lab.

#### Step 2: Register the Bloom Test

- Open the Bloom Test packaging.
- Swipe the phone across the lid of the Bloom Test packaging to register. Or register manually, by entering its serial number, which is indicated on the bottom of the packaging with "SN".

## 4.2. Blood sample collection

**Caution:** All blood samples should be treated as potentially infectious. Prevent the contamination of surfaces by performing the blood sample collection over a removable surface, such as a paper sheet or the opened Bloom Test packaging.

**Warning**  
The accuracy of the test result heavily relies on the correct collection of the blood sample. After registering the Bloom Thyroid Test, follow all instructions with utmost care.

### Step 1: Select and disinfect the finger

1. Warm up hands by washing them with warm water or rubbing them together. This will improve the blood flow.
2. Select and disinfect the middle or ring finger using the

*Disinfect the finger.*

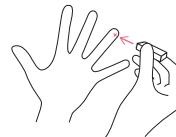
**Caution:** Do not perform the blood sample collection in one of the following situations:

- On an arm where an infusion has been performed within the last 24 hours.
- On a finger of the side of the body where a mastectomy has been performed.
- On a swollen, infected, wounded or previously punctured finger.
- If you have a history of fainting at the sight of blood.
- If you suffer from hemophilia.
- If you have severely impaired dexterity, or lack of fine motoric skills.
- Do not use any object other than the sterile lancets provided in the Bloom Thyroid Test to prick the skin.
- The lancet should only be used on clean and dry middle or ring fingers. Skin should be healthy, without any dermatological issues or wounds.

### Step 2: Use the sterile lancet on the disinfected fingertip


*1. Remove the protective cap of the sterile lancet.*


*2. Press the red end of the lancet on the side of the cleaned fingertip to trigger it.*

### Step 3: Collect blood sample immediately after the puncture is created

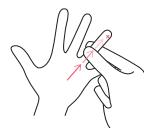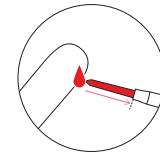*1. Massage hand from palm 2. Let the capillary tube soak up the blood drop.*


*3. Fill the capillary tube up to the white stopper.*

### Step 4: Apply the plaster to the puncture on the fingertip using light pressure to prevent continuous bleeding

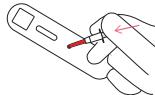**Step 5: Transfer the collected blood onto the Bloom Test strip immediately**


1. Open the Bloom Test strip pouch.
2. Place the Bloom Test strip on a flat surface.
3. Empty the filled capillary tube onto the sample window of the Bloom Test strip by pushing down the white piston. Ensure that the capillary tube is completely emptied onto the test strip.
4. Wait for absorption (see Bloom App timer).

*Transfer blood onto the sample window.*

**Step 6: Add buffer and let sample sink in**

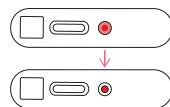1. Remove the cap of the buffer dispenser and add one drop of buffer onto the sample window
2. Wait for absorption (see Bloom App timer).


3. Confirm the blood transfer by clicking the "confirm sample transfer" button in the Bloom App.

**Caution:** Examination results can be affected if the Bloom Test strip is moved before the liquid sample has been absorbed, dropped or not handled as intended.

**Step 7: Insert Bloom Test strip into Bloom Lab**

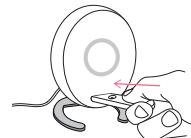1. Insert the Bloom Test strip into the Bloom Lab. You should feel the Bloom Test strip lock into place. The Bloom Lab will show a white heartbeat pattern on the LED ring when the test strip is inserted correctly.

*Insert Bloom Test strip into the Bloom Lab.*

2. Processing begins. The progress of the measurement is communicated by the LED ring of the Bloom Lab with a spinning white loader. The Bloom App shows the estimated time remaining.
3. Once processing has been completed, the Bloom App indicates the successful completion of the testing.

**Caution:**

- Place the Bloom Lab on a stable, flat surface and do not move the device during measurement.
- Do not remove the Bloom Test strip from the Bloom Lab during the testing procedure.
- Ensure that the Bloom Lab does not come into contact with blood.
- Make sure that the Bloom Test strip is clean and free from dust before inserting it into the Bloom Lab.

**Possible errors:**

- In case of a recoverable error, the LED ring of the Bloom Lab will show a red pattern, indicating that something went wrong. See section 7.1. for further details.
- In case of a fatal error, such as a general error requiring service of the Bloom Lab or power adapter, the LED ring of the Bloom Lab will show a solid red ring, indicating that a general hardware failure error occurred.

**Step 8: Dispose components**

The used Bloom Thyroid Test components are potentially biohazardous. Please dispose of them carefully according to applicable laws and regulations.

#### 4.3. Process sampling and report collection

The final step of the testing process is to complete the questions needed to compile the personalized Bloom Thyroid Report using the Bloom App. While the Bloom Lab is processing the test result, the Bloom App will prompt you to do the following:

## Check the Bloom App

1. Answer the questions. When you are presented with a multiple-choice question, choose all answers that apply and then confirm to proceed.
2. After around 20 minutes, the results from the Bloom Lab will be transmitted to the Bloom App. The app will alert you that the Bloom Thyroid Report is ready.
3. View the personalized Bloom Thyroid Report.

**Information:** If the Bloom App disconnects from the Bloom Lab during the measurement process, the Bloom Lab completes the measurement and saves the results. If the same Bloom App reconnects with the Bloom Lab before another measurement is initiated, the saved measurement value will be sent to the Bloom App and the Bloom Lab will subsequently delete the measurement value and any unique identifier of the Bloom App. After a 10-minute period other Bloom Apps will be able to connect to the Bloom Lab and perform new tests, which will automatically delete the previous results.

## 4.4. Device cleaning

**Caution:** Please use protective equipment, such as gloves, when cleaning the surfaces of the Bloom Lab to avoid contamination.

After performing the Bloom Thyroid Test, please make sure to leave a clean workspace. Thoroughly wipe all surfaces of the Bloom Lab after every use with the provided disinfection cleaning wipe. Please check gaps for stains and clean thoroughly when needed.

## 5. Storage

In order to avoid hazardous situations, the Bloom Thyroid Test should be stored with care at all times, this includes:

- The recommended storage temperature between +8°C and +30°C (46°F and 86°F)
- A dry indoor environment without direct or excessive heat and humidity below 80% (bathrooms not suitable)
- Out of reach for children to avoid injury through misuse of components like lancets.

## 6. Troubleshooting

For further information about the Bloom Lab, Bloom App, frequently asked questions and troubleshooting, visit [www.bloomdiagnostics.com/faq](http://www.bloomdiagnostics.com/faq).

## 7. Symbol information

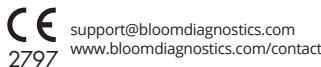
	CE Mark		Manufacturer
	Use-by date		Consult instructions for use
	Fragile, handle with care		Serial number
	Keep dry		Keep away from sunlight
	Sterilized using irradiation		Do not use if package is damaged
	Single use		Batch number
	Contains sufficient material for one test		Contains sufficient material for five tests
	Catalog number		Recycling
	Humidity exposure limit <80%		Temperature exposure limits between +8°C and +30°C (46°F and 86°F)
	In vitro diagnostic medical device		

## 8. Contact information

For additional information regarding technical support and local distributors, contact:

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Bloom Diagnostics AG