

## Easy-to-Read 3M<sup>™</sup> Attest<sup>™</sup> Steam Chemical Integrators, Type 5

## Unprocessed

STEAM Ty	pe 5	
eam nemical		← Must pass line
egrator		
RE.	JECT	ACCEPT

3M<sup>™</sup> Attest<sup>™</sup> Steam Chemical Integrators, Type 5 comply with ISO 11140-1:2014.

© 2020 3M. All rights reserved. 3M and Attest are marks and/or registered marks of 3M. Unauthorized use prohibited. Used under license in Canada.

70-2009-8636-5

		ACCEPT
3M		
Steam	STEAM Type 5	
Chemical		← Must pass line
ntegrator	COMPANY	
	REJECT	ACCEPT
LOT 201910	ICX REF 1243	
284	STEAM Type 5	
Steam Chemical		← Must pass line
ntegrator		
	REJECT	ACCEPT
LOT 201910	ICX REF 1243	
3M	STEAM Type 5	
Steam Chemical		← Must pass line
ntegrator		
	REJECT	ACCEPT
LOT 201910	ICX REF 1243	

The color bar has reached the **ACCEPT** window in all three samples shown above, indicating that the necessary conditions for sterilization have been met. If the color bar crosses into the **ACCEPT region, it is considered a pass.** 

	REJECT	
3M \Lambda	STEAM Type 5	
Steam Chemical		← Must pass line
ntegrator		
	REJECT	ACCEPT
TOT SOTATOCX	REF 1243	
3M 🔥	STEAM Type 5	
Steam		← Must pass line
Chemical Integrator	a prestation that the	< wiust pass line
11 cegrator		
	REJECT	ACCEPT
X3016102 TOT	REF 1243	
3M 🔥	STEAM Type 5	1
Steam Chemical	Mer - Calendary Contra	← Must pass line
ntegrator		t must puss mis
C42	REJECT	ACCEPT
1812	REF 1243	ACCEPT

The color bar is in the **REJECT** window, indicating that the necessary conditions for sterilization have not been met. The pack should be reprocessed and the cause of sterilization failure should be investigated. If the color bar is in the **REJECT** region or on the line, it is considered a fail.