## HEINE Ophthalmic Examination Lamp

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tél. +49 (0) 81 52 / 38 - 0  
Fax +49 (0) 81 52 / 38 - 2 02  
E-Mail: info@heine.com · www.heine.com  
med 6996 2014-05-23

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### General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries). Date of consignment means that HEINE hands the goods over to the transport carrier, freight forwarder or any other person designated by the Customer for the transport of the goods without loading the collecting vehicle. The guarantee covers irreproachable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criteria: colour temperature, useful service life, safety, optical quality and performance). The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

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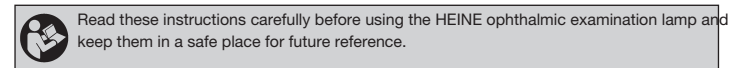
## Instructions for use

ENGLISH

### HEINE Ophthalmic Examination Lamp

In accordance with local regulations this product should be disposed of as an electronic device separately.

#### Safety information

**Range of applications:** The HEINE ophthalmic examination lamp is used for examination of the eye, particularly the anterior segment.

**Safety in use:** Use only power sources which are designed for medical applications.

#### Use and function

By rotating the ribbed control (1) you can select one of the three circular apertures, a blue filter, or a slit aperture. Use the thumb and forefinger of the hand which holds the instrument to operate the control.

The images formed by the apertures are focused at a distance of 100 mm and have the following dimensions:

Large aperture and blue filter:	10,5 mm dia.
Medium aperture:	6,5 mm dia.
Small aperture:	4 mm dia.
Slit aperture:	10,5 x 0,7 mm

#### Changing the bulb

**Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.**

- Remove the lamp head from the handle.
- If the lamp is fitted with an automatic connector (2), unscrew this from the lamp head.
- Pull the used bulb out of the lamp head.
- Clean the glass of the new bulb before replacement with a soft cloth.
- If appropriate, screw the automatic connector (2) back onto the lamp head. Re-connect the lamp head to the handle.

#### Care and maintenance

Clean the housing with a mild cleaning agent and soft cloth. To clean the lens, use a cotton wool bud soaked in alcohol and wipe with a spiral movement starting from the centre of the lens.

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