AIDIAN

QuikRead go® Instrument


135936-6

- English
- Deutsch
- Français
- Italiano

	Explanation of symbols	Zeichenerklärung	Explication des symboles	Spiegazione dei simboli
	<i>In vitro</i> diagnostic medical device	<i>In-vitro-Diagnostikum</i>	Dispositif médical de diagnostic <i>in vitro</i>	Dispositivo medico-diagnostico <i>in vitro</i>
	Serial number	Seriennummer	Numéro de série	Numero di serie
	Catalogue number	Bestellnummer	Référence du catalogue	Riferimento di Catalogo
	Manufacturer	Hersteller	Fabricant	Fabbricante
	Date of manufacture	Herstellungsdatum	Date de fabrication	Data di fabbricazione
	Contents	Inhalt	Contenu	Contenuto
	Instrument	Gerät	Appareil	Strumento
	Power supply	Netzteil	Alimentation	Alimentatore
	Mains cable	Netzkabel	Câble d'alimentation	Cavo di collegamento alla rete
	Consult instructions for use	Gebrauchsanweisung beachten	Consulter la notice d'utilisation	Consultare le istruzioni per l'uso
	Caution	Achtung	Attention	Attenzione
	Temperature limitation	Temperaturbegrenzung	Limites de température	Limiti di temperatura
	Fragile, handle with care	Zerbrechlich, mit Sorgfalt behandeln	Fragile, manipuler avec précautions	Fragile, maneggiare con cura
	Keep dry	Vor Nässe schützen	Conserver au sec	Conservare a secco
	China RoHS GB/T 26572	China RoHS GB/T 26572	China RoHS GB/T 26572	China RoHS GB/T 26572

QuikRead go®

Cat. No. 133893
Cat. No. 149915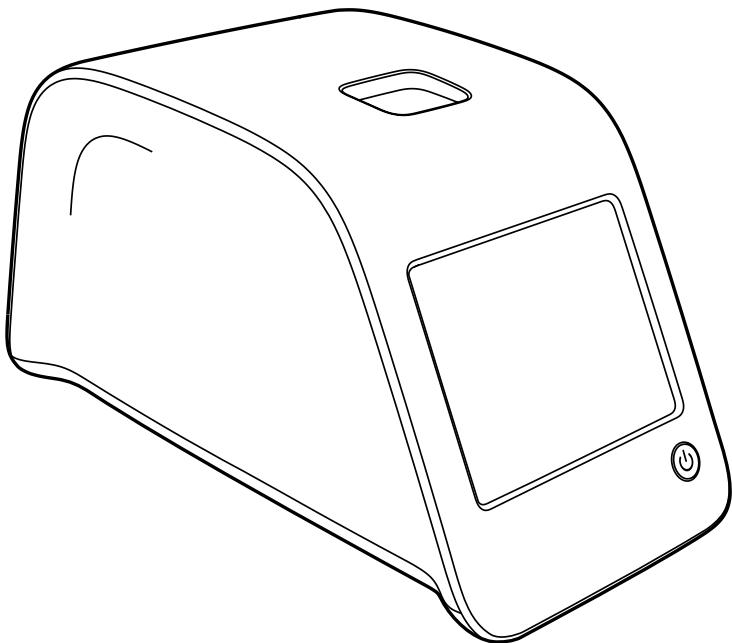

Instructions for use • English	4
Bedienungsanleitung • Deutsch	36
Mode d'emploi • Français	70
Istruzioni per l'uso • Italiano	104

CONTENTS

1 INTRODUCTION	5	User interface in general.....	16
Intended use	5	Main menu.....	16
QuikRead go instrument.....	5	Status area symbols.....	16
Safety information.....	5	Layout.....	17
Precautions and limitations.....	5	Structure of the user interface	18
2 GETTING STARTED	6	3 USAGE	19
Unpacking.....	6	Performing an assay.....	19
Accessories	6	Basic measuring mode	20
QuikRead go parts.....	6	Quality control.....	21
Lifting/transferring the instrument	8	Other measuring modes	21
Location and environment	8	Results	21
During use	8	Viewing results	21
During transport and storage	8	Deleting result history	21
Power supply cable & accumulator unit.....	9	Printing results.....	21
Connectors and cables.....	9	Transferring results to USB-storage ...	21
Plugging the power supply cable.....	9	Sending offline results to LIS/HIS.....	21
Inserting the accumulator unit	10	Settings	22
Power (on, off, sleep mode).....	11	Personal settings.....	22
Turning power on.....	11	Measurement flow	23
Turning power off.....	11	Maintenance settings.....	25
Sleep mode	11	Admin settings	25
Use of the touch screen.....	11	Manufacturer settings	27
Set-up wizard.....	12	Profiles	27
Language.....	13	Creating a profile	27
Date and time	14	Applying a profile.....	27
Screen brightness.....	14	Basic settings	27
Audio volume.....	15		
Power save.....	15		
Finishing the Set-up wizard	15		
		4 MAINTENANCE	28
		Instrument calibration	28
		Cleaning the instrument.....	28
		Software update.....	28
		Changing the clock battery	28
		5 TROUBLESHOOTING	29
		6 INSTRUMENT SPECIFICATION..	32
		Declaration of conformity	32
		Technical specification	32
		Photometer	32
		Touch screen display	32
		Dimensions & power requirements....	32
		Software update	32
		Instrument identifier	32
		Memory.....	32
		Power supply	32
		LIS connection	32
		USB connection	32
		Service	33
		Warranty	33
		Disposal	33
		QuikRead go user levels and rights	
		when Security login is enabled	34

1 INTRODUCTION

Intended use

QuikRead go® is an easy to use *in vitro* diagnostic test system. It has been designed to measure various analytes from patient samples needed as an aid in diagnosis and treatment monitoring. The system consists of the QuikRead go Instrument and the QuikRead go reagent kits.

QuikRead go instrument

QuikRead go is a photometer capable of producing quantitative and qualitative results. The instrument has been designed and calibrated for both photometric and turbidimetric measurements. The instrument will guide you through the assay procedure by means of a series of messages and animations shown on the display.

The QuikRead go instrument measures the absorbance of the cuvette contents and converts the absorbance value into a concentration value or positive/negative result on the basis of preset test calibration data. The calibration data defining the overall assay curve or cut-off value for each test is encoded on each cuvette label. This information is transferred to the QuikRead go instrument automatically during the measurement.

The assays are performed according to the user instructions accompanying each QuikRead go reagent kit. The results are available in minutes. The instrument can be operated on the mains power or with an accumulator unit, it has USB connections for an external printer or keyboard, or barcode reader.

The QuikRead go instrument can be connected to a remote Laboratory and Hospital Information System (LIS/HIS). The instrument uses a standardised data transfer protocol. Contact your local supplier for more details.

Safety information

For your own safety, comply with all warning and caution statements. To alert you to potential electrical or operational hazards, warning and caution statements are provided where applicable.

Before taking QuikRead go into use, please read the following precautions and limitations carefully.

Precautions and limitations

- Do not spill any liquids or drop any objects onto or into the instrument.
- Spillage of potentially infectious material should be wiped off immediately with absorbent paper tissue and the contaminated areas swabbed with a standard disinfectant or 70% ethyl alcohol. Material used to clean spills, including gloves, should be disposed of as bio-hazardous waste.
- Read beforehand and carefully follow the QuikRead go reagent instructions for use supplied with each reagent kit.
- Only QuikRead go reagents can be used.
- Do not mix components with different lot numbers or different assays.
- Never place a cuvette without a tightly placed cap into QuikRead go.
- Ensure that the cuvette sealing foil is totally removed.

- Use only the power supply supplied with the instrument and make sure that the plug is placed so that it is removable.
- Use only the official QuikRead go accumulator supplied by Aidian.
- Do not put your fingers or any external devices into the QuikRead go during measurement.
- Do not remove or shut down a USB-device during data transfer.
- Do not open the instrument's covers by opening any screws. If the warranty seal is broken, the instrument warranty is not valid.
- Use a secure internal network or Virtual Private Network (VPN) when connecting the QuikRead go to a LIS/HIS using LAN.
- Do not use or connect the instrument to LAN if the warranty seal is broken.

2 GETTING STARTED

Unpacking

Open the packaging box and check that it contains all the necessary items:

- Instrument
- Instructions for use
- Power supply
- Mains cable
- Certificate of analysis

Carefully examine the instrument to ascertain that it has not been damaged during shipment.

If damage has occurred or any parts are missing, immediately notify your local supplier.

Accessories

Printer

The instrument can be connected to an external printer. A list of compatible printers and configuration parameters can be found at www.quikread.com.

Connect the compatible printer to a USB port and follow the instructions on the display.

Barcode reader

An external barcode reader can be connected to the QuikRead go instrument. A list of compatible barcode readers can be found at www.quikread.com.


Connect the compatible barcode reader to a USB-port and follow the instructions on the display.

WLAN adapter

For a wireless network connection. Use only an adapter supplied by Aidian. Insert the adapter to any free USB port.


QuikRead go parts

The components of QuikRead go are shown in **Picture 1** (instrument from above), **Picture 2** (instrument from the rear) and **Picture 3** (instrument from below).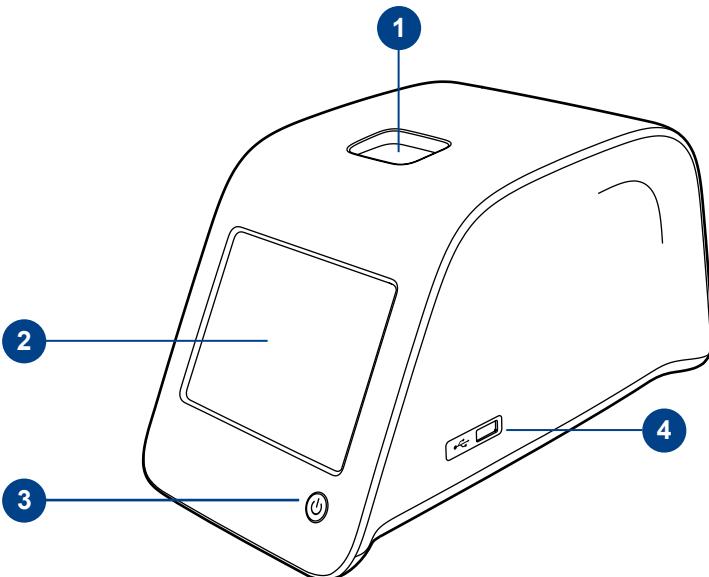


Picture 1

1. Measurement well for cuvette
2. Touch display
3. Power button
4. USB-port 1 (type A)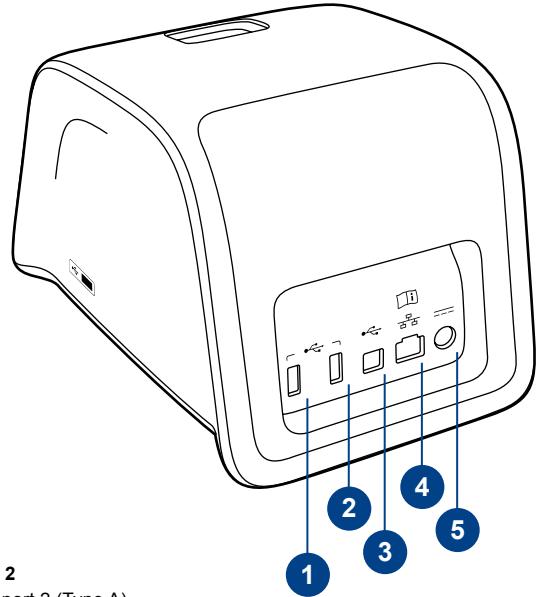

Picture 2

1. USB-port 2 (Type A)
2. USB-port 3 (type A)
3. USB-port 4 (Type B)
4. RJ-45 port
5. Connector for power supply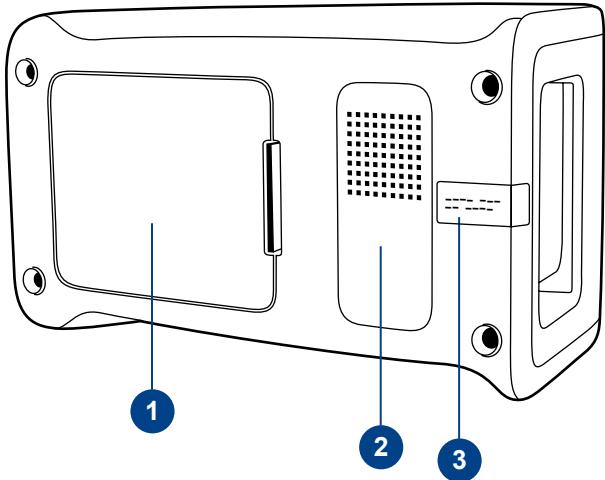


Picture 3

1. Accumulator unit cover
2. Instrument label with serial number
3. Warranty seal

Lifting/transporting the instrument

When lifting or transporting the QuikRead go instrument, always handle it carefully. In the back of the instrument there is a handle to enable lifting it by one hand (**Picture 4**).

A recess is added to the instrument side surfaces to help the grasp (**Picture 5**).

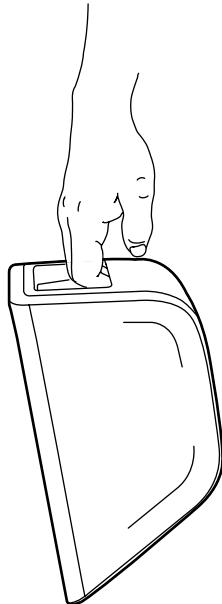Picture 4

Lifting the instrument by one hand.

Picture 5

Lifting the instrument by two hands.

Location and environment

During use

The instrument should be placed on a flat clean, horizontal surface, and the following points should be observed:

- Indoor use
- Altitude up to 2000 m
- The ambient temperature must be between 15°C and 35°C.
- Maximum relative humidity 80 % for temperatures up to 31°C decreasing linearly to 67 % relative humidity at 35°C (non-condensing).
- Mains supply voltage fluctuations up to ±10 % of the nominal voltage.
- Installation category II (2500 V transient).
- Do not place the instrument in direct sunlight.
- Do not place the instrument into a strong magnetic or electric field.
- Do not move the instrument while a test is being measured.
- Do not measure in a moving vehicle.
- Pollution degree 2.

During transport and storage

- The ambient temperature must be between 2°C and 35°C.
- Protect from rain and humidity.
- Handle the instrument with care.

Power supply cable & accumulator unit

The QuikRead go instrument can be used either by the power supply cable or the accumulator unit. The accumulator unit is charged automatically when the power cable is plugged in.


Connectors and cables

In the back of the instrument there are five connectors with symbols describing their use. One USB connector is placed on the right side of the instrument. All symbols are described in **Picture 6**.


The RJ-45 connector can be used for serial and LAN connections. Wiring diagram of the cable is described at www.quikread.com.

Plugging the power supply cable

Plug the power supply cable into the back of the instrument. Plug the power supply into a power outlet.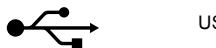

USB

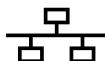RJ-45


Please consult
instructions for use

Power

On-Off switch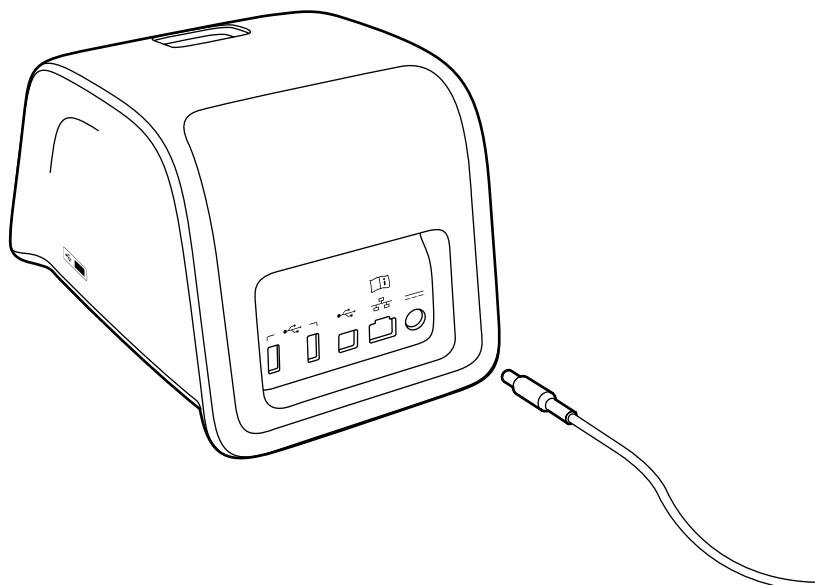

Picture 6

Symbols on QuikRead go


Picture 7

Connecting Power Supply Cable

Inserting the accumulator unit

Please follow carefully the steps below for inserting the accumulator unit to the QuikRead go instrument.

1. Turn off the instrument (if on).
2. Unplug the power supply cable.
3. Turn the instrument onto its side on a table.
4. Open the accumulator unit cover.
5. Attach the accumulator connector to the accumulator unit.
6. Press the accumulator unit into its place and ensure that it is placed properly.
7. Close the accumulator unit cover.
8. Turn the instrument back to a standing position.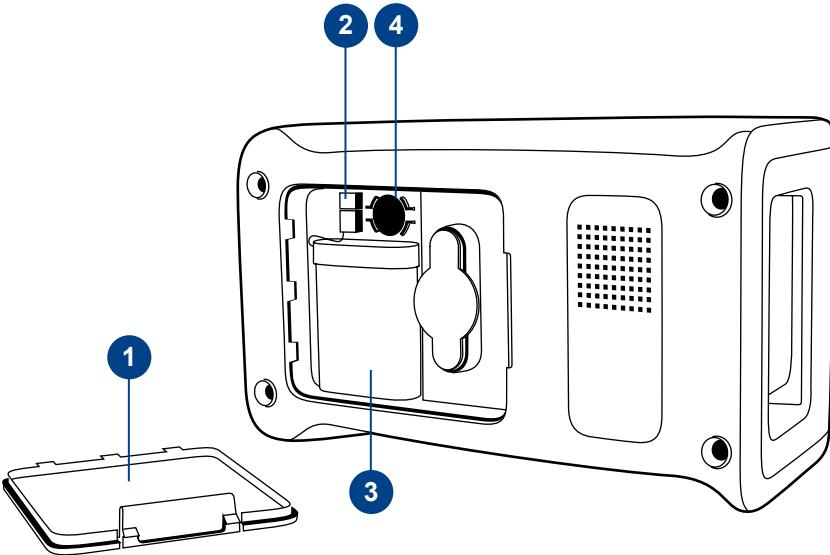

Picture 8

1. Accumulator unit cover
2. Accumulator connector
3. Accumulator unit
4. Clock battery

Power (on, off, sleep mode)

The QuikRead go instrument can be in three modes: on, off or sleep mode.

Turning power on

To turn the instrument on, press the Power button on the front panel. The light on the Power button will indicate that the instrument is powered on.

If nothing happens, ensure that the power outlet is connected, or if the instrument is on accumulator usage, that the accumulator is charged.

After pressing the Power button, the backlight of the display will light up, the instrument will start and the main menu will appear.

Starting the QuikRead go instrument for the first time will open a Set-up wizard (see chapter Set-up wizard).

Turning power off

To turn the instrument off, press the Power button for approx. two seconds. The instrument will ask you to confirm the shut down by asking "Do you want to shut down?". If Yes is chosen on the touch display, the instrument will turn off. In case a cuvette was inside the instrument while shutdown, the cuvette will be lifted up and the instrument will ask you to remove it.

Sleep mode

The purpose of the sleep mode is to save accumulator charge in accumulator usage. The sleep mode will activate automatically when the instrument has been inactive for longer than the time chosen in personal settings (see chapter *Settings* → *Power save*). The sleep mode functionality can be "Full standby" or "Close lid only".

The instrument indicates the sleep mode by blinking the *Power* button light.


To wake the instrument up, press the *Power* button.

Use of the touch screen

The QuikRead go instrument has a colour touch screen. It is operated by touching the virtual buttons by fingers. The screen can be used both with bare fingers and with gloves on. The touch screen does not require much pressure, and pressing too hard or using sharp items may damage the screen.

There is always a multisensorial feedback to touching a button: The button will indicate the touching both visually by changing appearance and with an audible sound.

A command is registered when the finger is released from the button touched. If the release happens outside the initial button area, no command is given.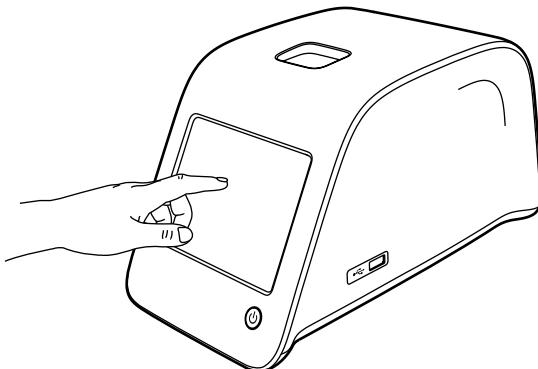

Picture 9

Using the touch screen by pressing the screen gently using a finger.

Set-up wizard


When starting the QuikRead go instrument for the first time, you will be asked to complete a Set-up wizard. During the Set-up wizard you will be asked to select the language, and to set the date and time.

The default language is English. The language can be changed on the first step on the Set-up wizard.

Start the Set-up wizard by choosing *Start* (see **Picture 10**).

Note

The Set-up wizard can be started also manually from *Settings* → *Measurement flow* → *Maintenance* → *Basic Settings*.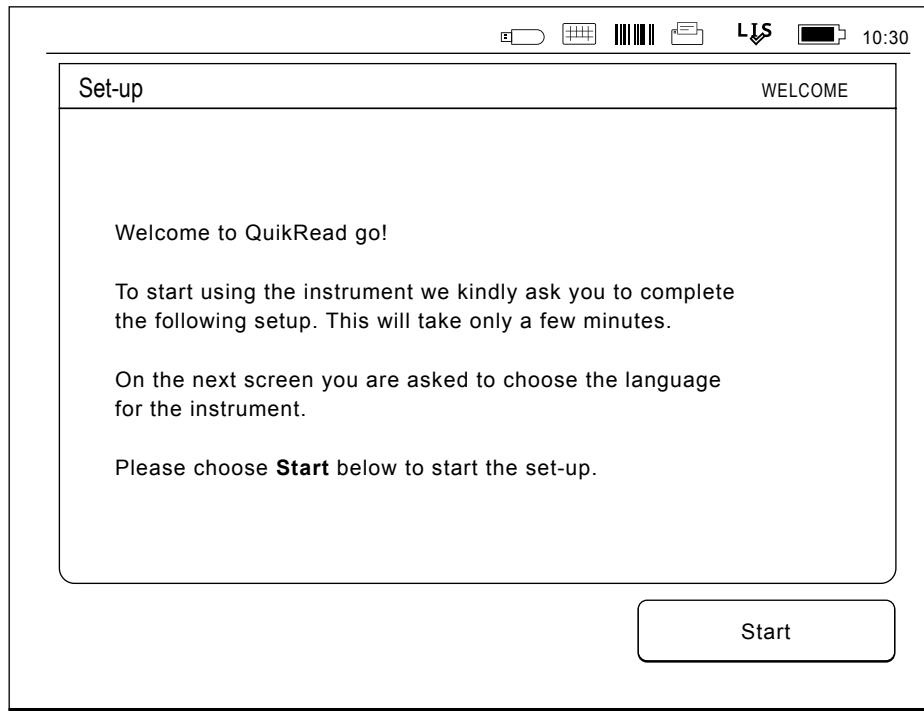

Picture 10


To start the Set-up wizard, choose *Start* in the Set-up screen.

Language

Choose the language you wish to use on the instrument. If you cannot see the desired language on the list, choose *More languages* for more options. Select the language by touching the corresponding button.

You will then be asked to confirm your language choice. You will see the confirmation request both in English and in the chosen language. If the chosen language is correct choose *Yes*, if not, choose *No*.

The chosen language can be changed anytime later.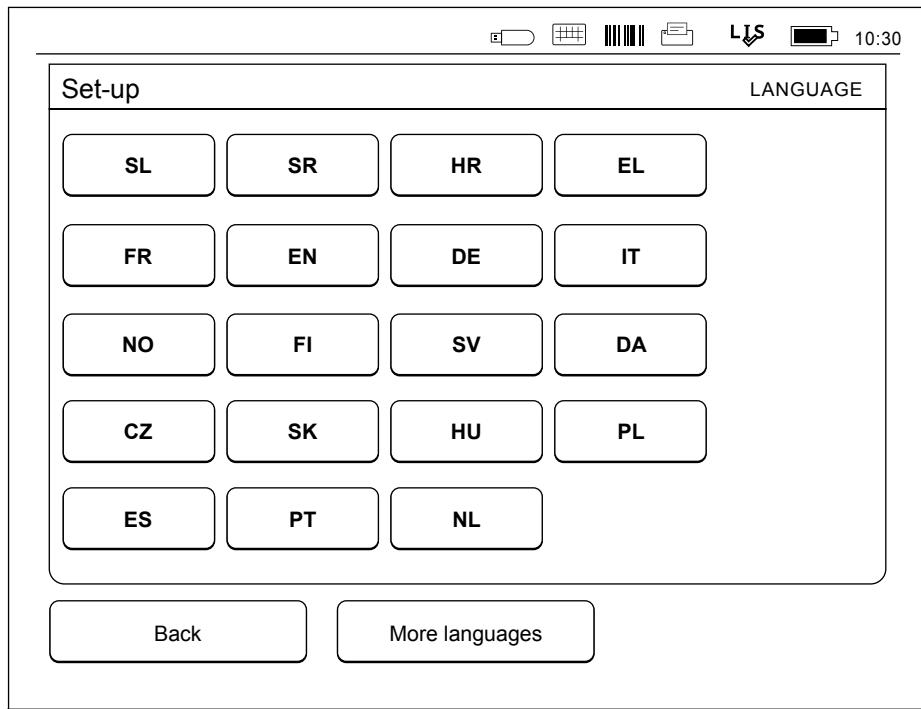

Picture 11

The first step in the Set-up wizard is to choose the operating language for the QuikRead go instrument.

Date and time


The second step in the Set-up wizard is to adjust the date and time. To do this follow the instructions below:

1. Choose *Edit* on the *Time* row.
2. Adjust time with the arrow buttons.
3. Choose between a 12-hour and 24-hour clock.
4. Accept with *OK*.
5. Press *Edit* on the *Date* row.
6. Adjust date with the arrow buttons.
7. Select the format of date.
8. Accept by *OK*.
9. Choose *Next* to continue.
10. Choose *Next*.

Screen brightness

The third step in the Set-up wizard is to adjust screen brightness. To do this follow the instructions below:

1. Adjust screen brightness with the arrow buttons.
2. Accept by choosing *Next*.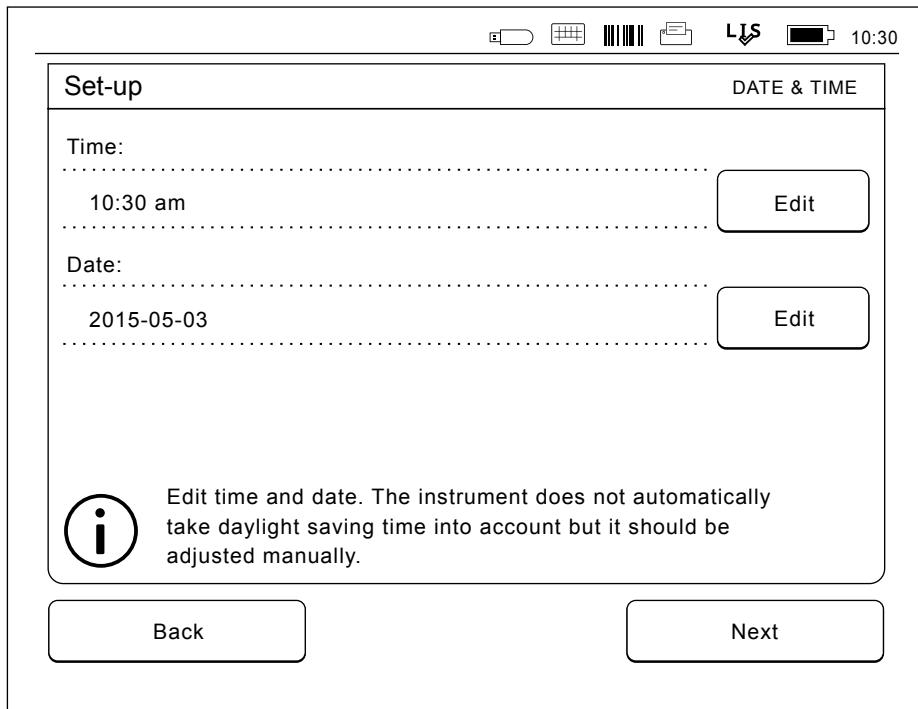

Picture 12

The second step in the Set-up wizard is to adjust the time and date for the instrument.

Audio volume

The fourth step in the Set-up wizard is to adjust audio volumes:

1. Adjust the keypad tone volume with the arrow buttons.
2. The sound volume can be tested using the *Test* button.
3. Adjust the Alert tone volume with the arrow buttons.
4. The sound volume can be tested using the *Test* button.
5. Accept by choosing *Next*.


Power save

Adjust the power save time to reduce power consumption when the instrument is being powered from the battery pack. A lower time increases the operation time.

1. Choose the time after which the QuikRead go instrument will enter sleep mode (hibernate).
2. Choose the functionality of the sleep mode.
In Full standby QuikRead go will enter the sleep mode and turn off the backlight of the display after the time specified. In the Close lid only mode only the lid will be closed.

Finishing the Set-up wizard

You have now completed the Set-up wizard. You may start using the instrument or continue making additional settings in *Advanced Set-up*, which takes you to *Measurement flow* where you can adjust laboratory- or work-routine oriented settings.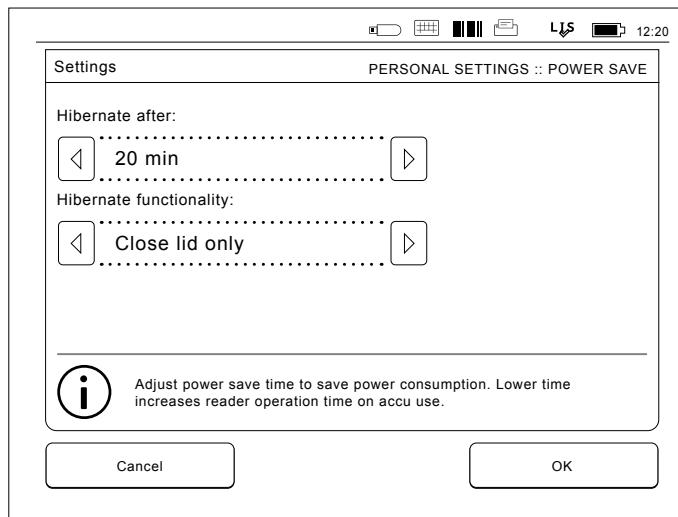

Picture 13 Power save

User interface in general


The QuikRead go instrument is used via a graphical user interface. In this chapter the main principles of the user interface are explained.

Main menu

All the features of the user interface can be accessed via the main menu (see **Picture 14**).

Status area symbols

The status area may contain the following symbols:

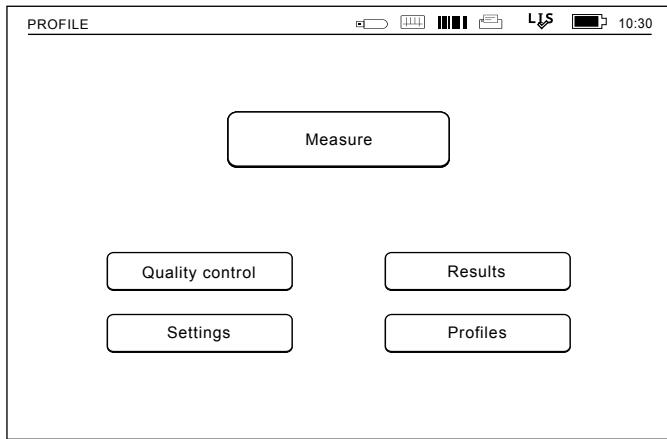Picture 14

LIS Connection state:

POCT state:

Power state:

Others:

Barcode

Printer

USB storage

Keyboard

Connected to QR go Feeder

Picture 15

Status area symbols

Instrument locked

Layout

The screen area of the user interface is divided into five functional areas (see **Picture 16a** and **16b**):

1. Status area

Indicates the status of the QuikRead go instrument via symbols.

2. Notification area

Indicates the current stage of the process via colour. The default colour is grey, while green means that something is currently being performed, yellow means that a user action is needed and red indicates an error.

3. Content area

The actual data is at the centre of the screen

4. Information area

On most screens there is additional information for guidance.

5. Navigation area

Standard buttons for navigation can be found at the bottom of the screen.

This screenshot shows the main result screen of the QuikRead go. At the top left is the word 'PROFILE'. The top right displays icons for battery level, signal strength, and connectivity, along with the time '12:20'. The main content area shows a measurement for 'CRP' with a value of '20 mg/l'. Below this, there are fields for 'Patient ID' (xxxxxx) and 'Measurement time' ('2017-05-03 12:19'). A section labeled 'Test:' shows 'CRP' next to a button labeled 'Result info'. At the bottom of the screen are three buttons: 'Exit', 'Print', and 'New measurement'. Callouts numbered 1 through 5 point to specific elements: 1 points to the top right status bar; 2 points to the 'Result info' button; 3 points to the 'Patient ID' field; 4 points to the 'Result info' button; and 5 points to the 'New measurement' button.

Picture 16a Result screen


This screenshot shows the 'Result info' screen, which is a detailed view of the previous result. It includes fields for 'Operator ID' (12345) and 'Additional ID' (DOC1). It also shows 'Measurement ID' (30/A17044I01234) with buttons for 'LOT', 'REAG', and 'BUF'. The 'QuikRead go:' section shows the serial number 'A17044I01234' and the date '2019-03-05'. Below this is another 'Test:' section for 'CRP' with a 'Result' button. At the bottom are the same three navigation buttons: 'Exit', 'Print', and 'New measurement'. Callouts numbered 4 and 5 point to the 'Result' button in the 'Test:' section.

Picture 16b Result info screen

Structure of the user interface

The structure is different depending on whether the Security login is enabled in *Settings* → *Measurement flow* → *Maintenance* → *Admin settings* → *Security settings*. (see **Picture 17a and 17b**):

Their functionality is described in the next chapter.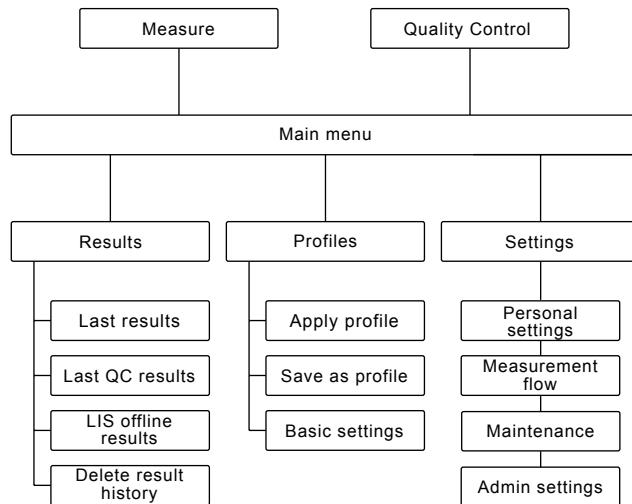

Picture 17a


The user interface structure without the Security login feature.

User interface structure without the Security login feature:

1. Measurement
2. Quality control
3. Results
4. Profiles
5. Settings

User interface structure with Security login enabled:

1. Measurement
2. Quality control
3. Results
4. Settings
5. Logout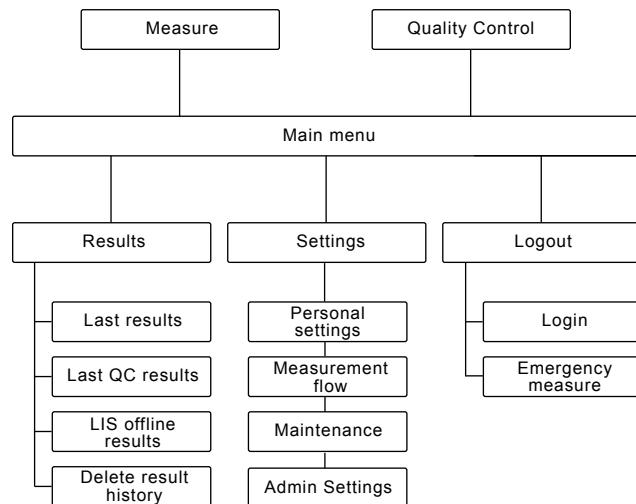

Picture 17b

The user interface structure with Security login enabled.

3 USAGE


The usage of the QuikRead go instrument can be divided into 3 main operations:

- Performing an assay
- Viewing results
- Changing instrument settings

Performing an assay

The instrument can be set to different measuring modes as needed. The basic measuring mode uses the simplest measuring protocol and is set as default in a new instrument if settings have not been changed when taking the instrument into use.

Only QuikRead go reagent kits can be used to perform an assay. Read the instructions for use of the corresponding QuikRead go reagent kit before use. The instructions provide more detailed information on carrying out tests and handling samples.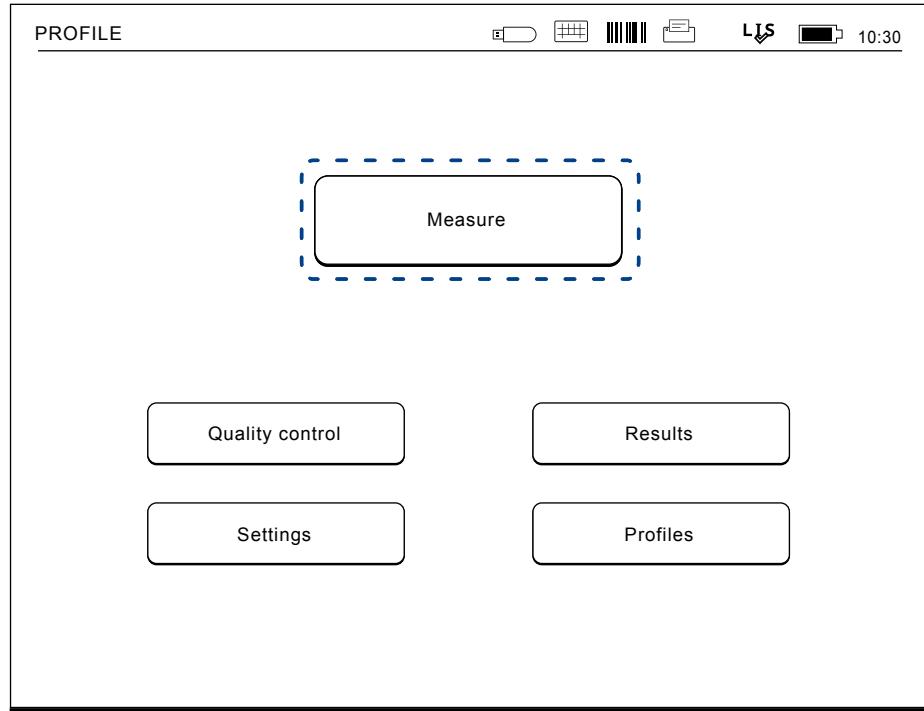

Picture 18


Start a basic measurement by choosing *Measure* on the main menu

Performing an assay in basic measuring mode

In the basic measuring mode the QuikRead go instrument performs an assay measurement and shows the result on the display with reagent lot data.

To perform a measurement do the following:

1. Choose *Measure* on the main menu and follow the instructions on the display (see **Picture 18**).
2. Drop a cuvette into the measurement well in the right position. The barcode of the cuvette must be facing towards you (see **Picture 19**).
Note: do not place a finger or any other object into the measurement well.
3. The lid closes and the instrument starts the measurement.
4. After the test is completed, the result is shown on the display and the cuvette lifted up for removal. Choose *Result info* to see additional information of the test (see **Picture 16b**).
5. Remove the cuvette. The result disappears from the display. It can be displayed again by choosing *See previous result*.
6. If you wish to make another measurement, insert a new cuvette into the measurement well. Choosing *Cancel* takes you to the main menu.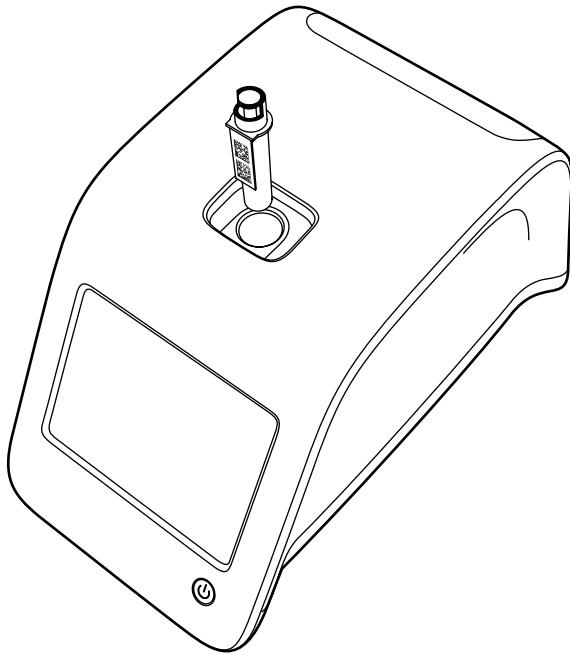

Picture 19

Drop a cuvette into the measurement well so that the barcode is facing towards you.

Quality control

QuikRead go has a separate result history file for quality control samples. Quality control samples are measured as normal samples but the results are stored in a separate result file. To start a quality control measurement choose *Quality control* on the main menu and follow the instructions on the screen.

Other measuring modes

The instrument can be used in different measuring modes in addition to the basic measuring mode. The optional protocols include the use of a patient ID, an operator ID, printing the results or sending the results to an LIS (Laboratory Information System). The measurement protocol is defined in the settings menu where these features can be enabled or disabled. If Security login is enabled, it replaces Operator ID.

The operator and patient IDs can be given via a barcode reader or written with the instrument's virtual keyboard or an external keyboard. The user feeds the operator and patient IDs before the measurement. Optionally, either or both IDs can be disabled in configurations. The operator ID can also be configured to propose the last input value. The user may change the operator ID before a measurement by overwriting the last ID with a new one.

The measurement results can be sent to a printer and/or to an LIS by enabling printer and/or LIS transfer.

Results

The results are stored in the *Results* file, where they can be viewed, printed or transferred to USB storage. The *Results* file consists of the following submenu items: *Last results*, *Last Quality Control results*, *LIS offline results* and *Delete result history*. LIS offline results are results stored in the memory of a QuikRead go instrument which is normally connected to an LIS but has temporarily been in LIS offline mode, e.g. during a home or a ward visit.

Viewing results

To view the results choose *Results* in the main menu. You can select *Last results* or *Last QC results* or *LIS offline results*. The results can be scrolled with the *up* and *down* buttons on the right. The results can be sorted by *Time*, *Analyte* or *Patient ID* by choosing the relevant buttons. Touching a result line gives detailed information of a single measurement.

Deleting result history

Choosing *Delete result history* permanently deletes all results from the history file. The instrument will ask for confirmation before deletion.

Printing results

Choosing *Print* gives an option to print the results sorted by *Date*, *Analyte* or *Patient ID*. Touch the  button and select the results to be printed. Choose *OK* to start printing.

Transferring results to USB storage

The results can be transferred to a USB storage. Connect a USB-storage to a USB port. Choose *Transfer to USB*, select the results to be transferred and choose *OK*. Don't remove the USB storage until the transfer is completed. After completion of the transfer a message "Transferring completed. You can now safely remove the USB storage." will be shown.

Sending offline results to LIS/HIS

All the results not sent to LIS can be viewed by choosing *LIS offline results*. Choosing *Send to LIS* sends the result to the LIS system, and after a successful transfer the results are deleted from the LIS offline memory. Choosing *Delete offline results* deletes the results without sending them to the LIS.

The QuikRead go instrument checks the LIS connection automatically during start-up, when entering the main menu and after every measurement. If a connection is available and there is data in the LIS offline result history, the QuikRead go instrument automatically proposes sending the offline results to the LIS.

Settings

QuikRead go settings can be configured through the touch display. The settings are divided into 4 main categories.

- Personal settings
- Measurement flow
- Maintenance
- Admin settings

Saving changes into Personal settings and Measurement flow settings is done by saving them as profiles and can be taken into use later on by applying a profile after start-up.

Changing the factory settings is done with the Set-up wizard. When started for the first time the instrument uses the factory settings. If Security login is enabled, there will be different user roles with different user rights in use, see the table User roles and rights on page 34.

Personal settings

In personal settings the operator can adjust or choose user oriented settings. These settings can be selected for temporary use until shut down of the instrument. For further use these settings need to be saved to a Profile. For continuous use the settings should be configured with the Set-up wizard: *Settings* → *Measurement flow* → *Maintenance* → *Basic settings*.

Screen

Screen brightness can be adjusted by choosing *Screen*. To increase or decrease the brightness of the screen use the arrow-buttons. Accept by *OK*, or reject by *Cancel*.

Volume

Sound volume can be adjusted by choosing *Volume*. Adjust the Keypad tone volume and the Alert tone volume by touching the arrow-buttons. Accept by *OK* or reject by *Cancel*.

Language


The language has been selected with the Set-up wizard. Here it is possible to change the language by choosing *Language* and selecting desired language. Accept by *YES* or reject by *NO*.

Power save

The time after which the QuikRead go instrument closes the lid – without entering sleep mode – can be adjusted from *Power save* → *Close lid only*. Adjust the delay value using the arrow buttons.

The instrument will close the lid without entering the sleep mode (activating the hibernation function) if it has not been used for the amount of time specified. This mode does not disrupt any LIS/HIS connection.

The time after which the QuikRead go instrument enters sleep mode can be adjusted from *Power save* → *Full standby*. Adjust the delay value using the arrow buttons. The instrument goes from idle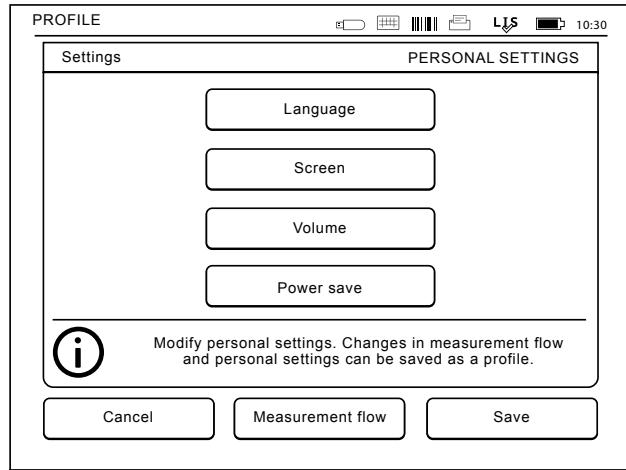

Picture 20
Personal settings menu

mode into sleep mode if it has not been used for the amount of time indicated. Entering the sleep mode closes any active LIS/HIS connection. Accept the settings by *OK*, or reject by *Cancel*.

Saving changes in personal settings

After all adjustments to personal settings have been made choose *Save*.

Saving setting to profiles for further use

On the main menu screen choose *Profiles*. Choose *Save as profile*, select an empty profile and name it or select a profile you'd like to modify, give the profile a new name if needed and accept by *OK*.

Measurement flow

In the Measurement flow settings the operator can adjust or choose laboratory/work routine oriented settings like operator and patient IDs, printing, LIS transfer and some test specific parameters.

These settings can be selected for temporary use by choosing *Save* after making the changes.

For further use the settings need to be saved to a Profile. For continuous use the settings should be configured with the Set-up wizard.

Operator ID

is an identification of the user.

- *Operator ID OFF*: The instrument does not request an Operator ID.
- *Operator ID ON*: An Operator ID has to be given before every sample measurement, and the ID is coupled with the test result.
- *Operator ID ON + Propose previous*: The instrument suggests a previous ID to be used, but it can also be changed.

Patient ID

is an identification of the patient sample.

- *Patient ID OFF*: The instrument does not request a Patient ID.
- *Patient ID ON*: A patient ID has to be given before every measurement, and the ID is coupled with the test result.


Test parameters

Some test specific parameters can be changed. Changes require giving the ADMIN password, which is QRGOSET. Choose  / Test parameters and the corresponding Test. A list of parameters to be configured will be shown.

Quality control checking

These settings are for quality controls. Quality control lots can be used to automate QC checking. Enter and edit information of quality control lots here. If POCT1-A2 is in use, information of a new lot to be sent to LIS/HIS can be entered here, but cannot be edited manually.

QC checking can be configured to give a warning only or it can be configured to disable patient measurements if the quality control measurement is outside the critical limits. A successful quality control measurement will re-enable patient measurements. Setting QC checking OFF and ON again will also reset the QC lock and re-enable patient measurements.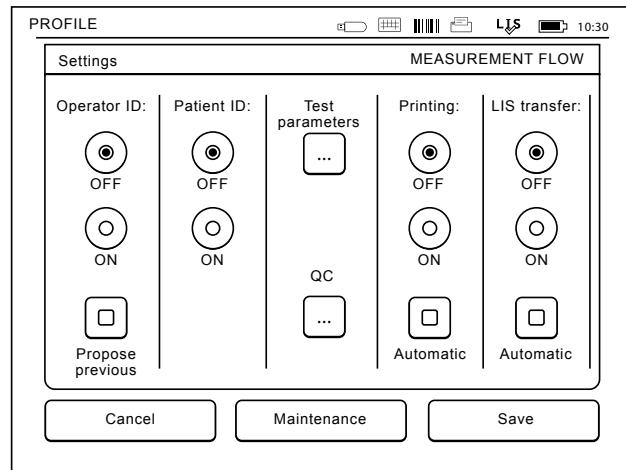

Picture 21
Measurement flow menu

To turn QC checking on, choose / QC and enter the ADMIN password.

- **QC checking OFF:** Quality control checking is not in use.
- **QC checking ON:** Quality control checking is in use.
- **QC lock:** If this is chosen, the instrument will be locked if a quality control result is outside the critical limits.


To set a new quality control sample, follow the instructions below:

1. Choose *Settings* → *Measurement flow* → QC. Give the administrator password.
2. Choose *New lot*.
3. Select the test from the *Test* list.
4. Enter the *QC lot ID*.
5. Select the unit from the *Units* list.
6. Set the *Warning range* limits.

NOTE: Decimals must be marked with a decimal point rather than a comma.

7. Set the *Critical range* limits.
8. Set the *Target value*.
9. Choose *Next*.
10. Enter the expiration date.
11. Accept twice by pressing *OK*.
12. Choose *Back*.
13. Choose *Save*.

The quality control lot has now been defined. The parameters of a lot can be edited or the lot deleted by choosing the line and selecting *Edit* or *Delete*. When starting a Quality control measurement, the quality control lot can be chosen from the list by choosing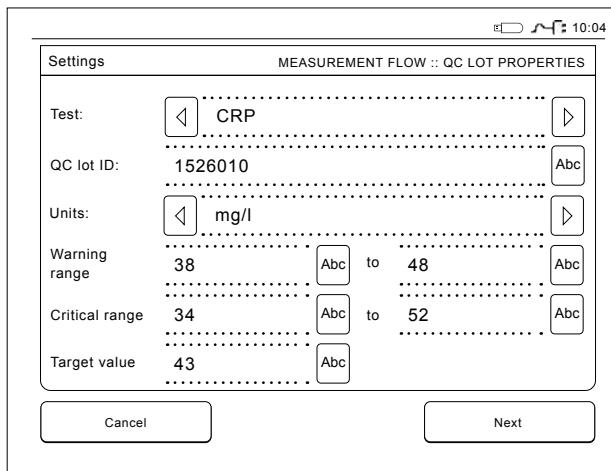

Picture 22
QC lot properties

Printing

- **Printing OFF:** The instrument does not suggest printing. It is nevertheless possible to print the result by choosing *Print* on the Measure/Result screen.
- **Printing ON:** After lifting the cuvette the instrument asks "Print current result?". Accept printing by choosing *Yes*. Cancel printing by choosing *No*.
- **Printing ON + Automatic:** The instrument prints every measurement result automatically.

LIS transfer

- **LIS transfer OFF:** The instrument does not send the results to the Laboratory Information System.
- **LIS transfer ON:** After printing, if activated, the instrument asks: "Send result to LIS?" Accept sending by choosing *Accept*. Reject by choosing *Reject*.
- **LIS transfer ON + Automatic:** The instrument sends the measurement result automatically to the LIS.

Saving Measurement flow setting to Profiles for further use

On the main menu screen choose *Profiles*. Choose *Save as profile*, select an empty profile and name it or select a profile you'd like to modify, give the profile a new name if needed and choose *OK*.

Maintenance settings

The instrument specific settings can be configured in the Maintenance menu.

Date & time

Date & time can be adjusted by choosing *Date & Time*. To do this follow the instructions below:

1. Choose *Edit* on the *Time* row.
2. Adjust time with the *arrow* buttons.
3. Choose between a 12-hour and 24-hour clock.
4. Accept by *OK*.
5. Choose *Edit* on the *Date* row.
6. Adjust date with the *arrow* buttons.
7. Select the format of date.
8. Accept by *OK* and confirm by *OK* on the next screen.
9. Choose *OK* to continue with the settings.

Error log

The instrument error codes are stored in the memory. Error codes can be scrolled with the up and down arrows on the right or they can be sorted by choosing *Time* or *Error code*.

Error codes can be transferred to a USB storage.

1. Choose *Transfer to USB*.
 2. Connect a USB storage to a USB port. Wait until the screen shows: "Transferring completed. You can now safely remove USB storage."
 3. Choose *OK* and remove the USB stick.
 4. Choose *Back* to return to Maintenance menu.
- Delete Error log* button deletes all error codes from the memory. Before deleting a confirmation text is shown.

1. Accept by Yes or cancel by No.
2. Choose *OK* on the Error log deleted screen.
3. Choose *Back* and *Cancel* and return to the main menu.

Self diagnostics

The instrument performs operational checks to ensure proper functioning. Perform a self diagnostics by selecting *OK*. Return to Maintenance by choosing *OK*.

Software update

QuikRead go software defines the instrument operations. Software can be updated to the newest available version if desired. New software will be available via the website www.softwareupdate.quikread.com or it can be ordered to be delivered on a USB storage device. Choose *Settings* -> *Measurement flow* -> *Maintenance* -> *Software update*.

If the instrument has an accu-unit, remove it before starting updating. Connect the storage to a USB port. The version numbers of current and new software will be shown on the display. Confirm the updating by choosing *Yes*. After completion of the software update, the following message is shown: "You can now safely remove the USB storage. After restart, update will continue and the screen will be blank for about 30 seconds. Do not switch the power off until update is complete. Press OK to restart." Press *OK*. Next, the following message will be shown: "Instrument needs to be restarted to complete software update." Press *Restart*. QuikRead go will restart and returns to the Main menu.

You can now safely remove the USB storage.

Touch screen calibration

The touch screen can be calibrated to optimise the usability of buttons. Start the calibration by choosing *Touch screen calibration*. Calibrate the touch screen by touching each of the black circles in turn. After "Touch display calibration succeeded" choose *OK*.

Product information

The *About* screen shows instrument-specific details:

- Instrument serial number
- Software version number
- LIS connection details

Maintenance logs

The instrument logs and security logs can be transferred to USB storage. Security logs can also be deleted. Transferring to USB storage does not empty the log files.

Admin settings

The administrative settings (*Settings* -> *Measurement flow* -> *Maintenance* -> *Admin settings*) allow administrative personnel to adjust instrument-specific settings regarding the LIS connection, the GMT value, Security settings and the QuikRead go Feeder settings. A *Factory reset* can also be started from here.

The following password is needed to change the Admin settings: QRGOSET. This password is used as a confirmatory step to ensure that the user does not access this page inadvertently.

GMT

GMT is a universal time against which the instrument's time is set. GMT is not visible to the user but is an internal clock for the instrument.

To set GMT adjust the date and time with the arrow buttons.

Internal clock

- Date and time have been adjusted to GMT at the factory.
- Date and time are stored in the memory once a day at start up.
- If the clock battery runs out, the clock stops. When the battery has been replaced, the clock continues from the last saved time. The time is adjusted in the Admin settings and requires a password. A new adjusted time cannot be before the last saved time. Giving too old a time will create an error message.

Local time

- Adjust the time according to the local time (*Settings* → *Measurement flow* → *Maintenance: Date & time*). The local time is saved to the memory as an offset to GMT.
- When changing the clock battery, adjust the time and date. The time cannot be adjusted to more than 24 hours before the last saved GMT. Giving too old a time prompts an error message "Setting of date failed. Date is in the past."

Security settings

When Security settings is set ON, user needs to log in to QuikRead go always before starting to use the instrument. Only emergency measure-

ments can be measured just with an operator ID, without a password. All users have a role and need a user account with operator ID, full name and password. User roles are

Admin:

- only one per instrument, operator ID is ADMIN. The ID is not modifiable.

Supervisor:

- has access to all except for performing factory reset and deleting security logs.

Normal User:


- can set user specific settings.
- can see QC and LIS offline results, and optionally old patient results.
- can add new QC lots locally when doing a Quality control measurement. QC checking needs to be ON.
- can see error log and transfer the log to a USB

storage device.

- features not accessible are greyed out. See table on p. 34 of user roles and rights.

To take Security settings into use, go to *Settings* → *Measurement flow* → *Maintenance* → *Admin settings* → *Security settings*

- Select *Security login ON*.
- Define access to old results.
- Admin password can be changed in *Manage admin account*. Default admin password is QRGOSET.
- Set up user and supervisor accounts in *User accounts*.
- Adjust time after a user will be logged out. This is different to the hibernate settings in *Power save*. If the Full standby option in *Power save* is in use, the user will be logged out when

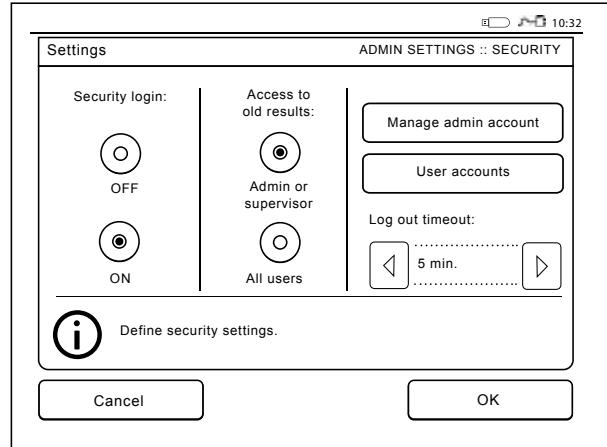Picture 23
Security settings

QuikRead go enters sleep mode. The user can also log out manually.

In case of a lost or forgotten Admin password, request a temporary password at www.softwareupdate.quikread.com. You will need an instrument serial number to proceed. The temporary password is valid for one week and can only be used once to reset the instrument to factory settings. Contact your supplier for more details about LIS connection.

LIS settings

From the LIS settings option, the LIS settings for data transfer can be adjusted. The data are transferred via a serial or a LAN connection. TCP/IP settings must be supplied before a LAN connection can function. Contact your supplier for more details about LIS connection.

LIS features

Select the following features OFF or ON.

- *LIS logging*. When set ON, LIS traffic will be logged.
- *Identify QC result*. When set ON, QC results will have a special identification in LIS communication.
- *LIS delay*. When set ON, there will be a short delay between consecutive results sent to LIS. A delay can be useful in solving congestion issues with older serial-type connections.
- *Additional ID*. When set ON, on top of Patient ID and Operator ID a third sample ID can be added, e.g. patient birthdate or Doctor ID.

The additional ID can be taken into use also with QuikRead go instruments not connected to LIS.

POCT1-A2 features

- *Operator login*. When set ON, all users need to log on always before starting to use QuikRead go. User information has to be provided by the POCT1-A2 server.
- *Patient ID validation*. When set ON, patient ID's will be checked against a downloaded patient list before sending results to the LIS.
- *Patient data shown*. When set ON, patient data will be shown before a measurement starts.
- *Emergency button shown*. When set OFF, emergency measurement is disabled in login screen.

TCP/IP settings

Select the type of LIS connection used.

WLAN settings

Select the type of WLAN connection used.

Character encoding

Select the type of character encoding used for LIS01-A2 protocol.

Contact your supplier for more details about LIS connection.

Factory reset

The user interface can be restored to factory settings. Factory reset deletes all profiles and results, and empties the error log. In instruments where Security login is enabled, the factory reset deletes security logs as well. After a factory reset all user accounts need to be recreated.

Manufacturer settings

This section is only for the manufacturer's use.

Profiles

User adjusted settings can be saved as profiles for later use. Four different user profiles can be saved to the instrument's memory. Profiles are not in use, if Security login is enabled.

Creating a profile

When the instrument has been set to work as desired the settings can be saved as a profile:

1. Choose *Save as profile*.
2. Select an (empty) profile.
3. Give the profile a name.
4. Choose *OK*.

Applying a profile

Choose *Apply a profile*. Select the desired profile.

Basic settings

Choosing *Basic settings* sets the instrument to work according to basic settings set by the Set-up wizard.

4 MAINTENANCE

The QuikRead go instrument has been designed to be as user-friendly as possible with no need for regular maintenance. For any repair requirement, contact your local supplier.

Instrument calibration

The instrument is factory calibrated. Proper functioning of the instrument is checked by the self-check procedure during every measurement. In case of malfunction, an error message is displayed.

The calibration data defining the overall assay curve or cut-off value for each test is encoded on the cuvette label. This information is transferred automatically to the instrument during every measurement.

Cleaning the instrument

Periodically clean the exterior of the instrument using a lint-free cloth dampened with water. Pay special attention to cleaning the display. Take care that no liquid runs onto the edges of the display, the measurement well or the connectors.

If necessary, a mild detergent may be used. Do not use organic solvents or corrosive substances. Spillage of potentially infectious material should be wiped off immediately with absorbent paper tissue and the contaminated areas swabbed with a standard disinfectant or 70% ethyl alcohol. Materials used to clean spills, including gloves, should be disposed of as biohazardous waste.

Software update

New software can be uploaded to the instrument with a USB storage, see page 25. Ask your local supplier for more information.

Changing the clock battery

The instrument has a battery which gives power to the internal clock. If the clock battery charge is empty a warning is displayed. The clock battery can be replaced with a same type of battery (type CR 2032 3V).

1. Turn off the instrument (if on).
2. Unplug the power supply cable.
3. Place the instrument onto its side on a table.
4. Open the accumulator unit cover.
5. If an accumulator unit is in place disconnect the accumulator connector from the accumulator unit and remove the accumulator unit.
6. Take the clock battery out of the battery holder.
7. Place new clock battery (type CR 2032 3V) into the battery holder text side down.
8. If an accumulator unit is used, attach the accumulator connector to the accumulator unit and press the accumulator unit into place and ensure that it is placed properly. Close the accumulator unit cover.
9. Turn the instrument back to a standing position and plug the power supply cable.
10. Start the instrument by pressing the *Power* button.
11. Adjust the date and time (*Settings* → *Personal settings* → *Measurement flow* → *Maintenance* → *Date and Time*).

5 TROUBLESHOOTING

The QuikRead go instrument displays error messages and guides the user in case it detects errors. Follow the instruction displayed and see the troubleshooting table in this instruction for use and

in the QuikRead go kit instruction for use. Contact your local supplier in case of needed support or a repair requirement.

Error message / Troubleshooting	Possible cause	Corrective action
QuikRead go does not start.	Power unit is not connected.	Connect power unit and retry.
	The instrument has an electronic malfunction.	Contact customer service.
Touch panel on display is not functioning properly.	Touch calibration is not correct i.e. the active area is not under the button.	Calibrate the touch screen according to the procedure described in the maintenance chapter.
	Touch panel is not responding at all.	Contact customer service.
Instrument alarm sounds cannot be heard.	Volume is set at a low level.	Set volume according to the procedure described in the chapter on personal settings.
	Instrument sound system has a malfunction.	Restart QuikRead go. If the problem persists, please contact customer service.

Error message / Troubleshooting	Possible cause	Corrective action
Printer is not printing.	The printer is off or the printer cable is not connected or the printer has malfunction or the settings are not correct.	Make sure that the printer is connected and its power is on. Check the settings. If the problem continues start-up the instrument and printer and retry printing from the Results menu. If problem continues, contact customer service.
Barcode reader is not functioning.	The barcode reader is not connected or the barcode reader has a malfunction or the settings are not correct.	Make sure that the barcode reader is connected. Check the settings. If the problem continues start-up the instrument and retry the barcode reading. If problem continues, contact customer service.
“Error code XXX. Please restart QuikRead go” message is displayed.	Humidity has accumulated on optical surfaces.	Transfer the instrument to a dry environment and restart.
	Temporary malfunction of the instrument.	Restart the instrument. If this error message is displayed frequently, contact customer service.
“Error code XXX. Please contact customer service” is displayed.	Permanent malfunction of the instrument.	Contact customer service.
Accumulator unit must be re-charged frequently.	Accumulator unit storage capacity decreases during its life time.	Replace the old accumulator unit with a new one according to the procedure described in the chapter Inserting accumulator unit.
“Accumulator level is low. Please connect to the mains cable in order to continue operation” error message is displayed.	Charge in accumulator unit is low.	Connect the power unit into the QuikRead go power connector.
Clock battery warning is displayed.	Internal clock battery is empty.	Replace the clock battery according to the procedure described in the chapter Changing the clock battery.

Error message / Troubleshooting	Possible cause	Corrective action
"Cuvette position not correct. Remove cuvette." error message is displayed.	Remnants of cuvette sealing foil is left on the cuvette collar.	Remove cuvette when the instrument has lifted it up. Make sure that all remnants are removed when doing the next measurement.
	The instrument has a mechanical malfunction.	Check the item above. If it does not apply, restart the instrument. If the problem persists, contact customer service.
Measurement prohibited.	Reagent cap is missing or cuvette is used.	Check that the cuvette has a reagent cap and the inner coloured part of the cap is not pressed down.
	Reading lot data from the barcode failed.	Try again. If the problem persists, cancel the test.
	Kit lot expired.	Discard expired kit lot. Use a new one.
	Cuvette temperature too low.	Let the cuvette warm up to room temperature. Test the same cuvette again.
	Cuvette temperature too high.	Let the cuvette cool down to room temperature. Test the same cuvette again.
	Blank too high.	Test the same cuvette again. The blanking process has not been completed or the sample may contain interfering substances. In the latter case, the test cannot be completed.
Test cancelled.	Unstable blank.	Perform a new test. There has been some problem during reagent addition. Make sure that the cap is properly closed.
	Error in reagent addition.	Perform a new test. If this message appears often, contact customer service.
	Instrument failure.	Perform a new test. If this message appears often, contact customer service.

6 INSTRUMENT SPECIFICATION

Declaration of conformity

The QuikRead go instrument complies with the electromagnetic emission and immunity requirements described in the standard IEC 61326-2-6. The instrument fulfills the FCC class A requirements.

Technical specification

The instrument has a pre-programmed microprocessor that controls the assay steps and data processing. Test identification, timing and calibration curve or cut-off value data are contained on a barcode on each cuvette. Once activated by the cuvette label, the microprocessor controls and guides all assay steps and converts the absorbance values of the samples into concentration units or cut-off values.

Photometer

The QuikRead go photometer consists of a measurement well, three LEDs and light detectors. The photometer has been designed and calibrated both for photometric and turbidimetric measurements.

Touch screen display

The user interface is based on an easy-to-use touch screen display. It is used by touch buttons that appear on the screen. It also provides the user with messages and prompts for performing each assay step, and gives test results and error messages.

- 4-wire resistive
- Display size: 116.16 x 87.12 mm
- Pixels: 640 x 480

Dimensions & power requirements

- Weight: 1.7 kg without power supply
- Size: 27 x 15.5 x 14.5 cm
- Power requirements

Voltage: 100–240 V AC

Frequency: 50–60 Hz

Power consumption: 8W

Software update

New software can be uploaded with a USB-memory storage. Ask your local supplier for more details.

Instrument identifier

Each QuikRead go instrument has a unique serial number which can be found on the label on the bottom of the instrument.

Memory

The QuikRead go instrument has an internal memory for results history. See the section Results.

Power supply

The instrument is powered by a power supply supplied with the instrument. In addition to the power supply, the instrument can use an accumulator-unit as a power source. An internal switch inside the cable connector will switch automatically from accumulator-powered use to mains-powered use. For instructions on installing an accumulator-unit, see the section Inserting accumulator unit.

LIS connection

The connection can be made using:

- An RJ-45 connector as a serial port with a special cable. Specification for wiring the cable can be found at www.quikread.com.
- An RJ-45 connector and a supported 10BASE-T/100BASE-TX Ethernet connection. Cat 5 / Cat 5e UTP (unshielded twisted pair) cable should be used.
- A WLAN connection. An external WLAN USB-stick is needed.
- Power over Ethernet (PoE) is not supported.
- Contact your supplier for more details.

USB connection

The instrument has three A-type USB connectors. These connectors can be used for the printer, barcode reader and memory stick. The instrument can be connected as a virtual com-port to a PC or computer via a B-type USB connector.

Service

The QuikRead go instrument is designed to be free of regular maintenance with build-in self check operations. In case of instrument malfunction or a repair requirement, please contact your local supplier. Before shipping the instrument to service, delete all patient results from Result history and clean the exterior of the instrument. See chapter Cleaning the instrument for detailed instructions.

Warranty

The manufacturer's warranty for the QuikRead go instrument covers defects in materials or manufacturing for a period of two years from the date of purchase. For the warranty to be valid, the warranty seal (see **Picture 3**) needs to be untouched. The manufacturer agrees to repair or replace the instrument if it becomes inoperative due to the failure of any internal part of the instrument. The warranty does not cover damage caused by use not in accordance with instructions. This warranty is valid for two years. The manufacturer is under no obligation to modify or update the instrument once it has been manufactured, unless a manufacturing defect is identified.

In the event of an instrument malfunction, please contact your local supplier.

Disposal

The QuikRead go instrument is a low-voltage electronic device. It is designed in compliance with the RoHS directive (Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment: Directive 2011/65/EU). A used QuikRead go instrument must be treated as potentially biohazardous waste. The instrument should be disposed of as Used Medical Equipment according to national and local law. Be sure to delete all patient data before disposing of the instrument.

The packaging materials are recyclable materials. The accumulator should be disposed of according to national and local accumulator collection rules based on Directive 2006/66/EC.

QuikRead go user levels and rights when Security login is enabled

Action	Normal user	Supervisor	Administrator
Patient measurement	x	x	x
QC measurement	x	x	x
See all QC results	x	x	x
Add new QC lot	x	x	x
See old patient results	- / x*	x	x
See LIS offline results	x	x	x
Transfer old results to USB	-	x	x
Personal settings (Language, Power save)	-	x	x
Personal settings (Screen brightness, Audio volume)	x	x	x
Measurement flow	-	x	x
Test parameters	-	x	x
QC parameters	-	x	x
Maintenance (Error log, Self-diagnostics, About)	x	x	x
Maintenance (Basic settings, Date & Time, Software update, Touch calibration, Maintenance logs: transfer)	-	x	x
Maintenance logs: Delete security logs	-	-	x
Admin settings (all except Factory reset)	-	x	x
Admin settings (Factory reset)	-	-	x
Change own password	x	x	x
Create / Edit / Remove a User	-	x	x
Create a new Supervisor	-	x	x

*Depending on settings in *Settings* → *Measurement flow* → *Maintenance* → *Admin settings* → *Security settings*.

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The screenshot shows the measurement results for CRP. At the top, there is a header with icons for battery level, signal strength, and connectivity, followed by the time '12:20'. Below the header, the measurement name 'CRP' is displayed, followed by the value '20 mg/l'. The main content area contains patient information ('ID Paziente: xxxxxxxxx') and measurement details ('Tempo della misura: 2017-05-03 12:19'). Below this, there is a section for the test ('Test: CRP') with a button labeled 'Info risultato'. A note below the test section says: 'Scegliere Info risultato per visualizzare il risultato. Rimuovi la cuvetta per eseguire una nuova misurazione.' At the bottom, there are three buttons: 'Esci', 'Stampa', and 'Nuova misura'.

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QuikRead go[®] is a registered trademark of Aidian Oy.
Aidian Oy