GEBRAUCHSINFORMATION

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INSTRUCTIONS FOR USE

Aktimed TAPE PLUS is an elastic self-adhesive tape bandage (kinesiology tape) with Arnica montana 10[®], Rhus toxicodendron 10[®] and Ruta graveolens 10[®] integrated into the adhesive mass. The focus is on the physical and mechanical properties of adhesive tape.

APPLICATION AREAS

- Kinesiological Taping

APPLICATION INSTRUCTIONS

- The product can only be used once and needs to be applied by a trained person only.
- Prior to application the skin should be cleaned and fat-free.
- Remove the protection film.
- Apply tape with and without strain.
- The glue unfolds the adhesive effect through body temperature. For this purpose it is advised to smooth out the tape in longitudinal direction from base to end (approx. 60 sec.).
- Apply the tape 30 min prior to sports activity.
- After showering, taking a bath or swimming the tape should be dried with a hair dryer (lowest setting) or by dabbing.
- The tape can occasionally cause skin irritation. In that case please remove the tape.
- The tape can best be removed after taking a shower or a bath.

SAFETY INSTRUCTIONS

- Do not use in case of an allergy against one of the ingredients, or test the tape on a small area of skin first!
- Do not apply on open wounds, injured skin or unhealed scars!
- Do not use in case of acute or chronic skin diseases (e.g. neurodermatitis thrust, Psoriasis, fungal infection)!
- Do not use in case of Thrombose or Thrombophlebitis!
- Do not use in case of a peripheral arterial occlusive disease distal to the occlusion!
- Do not use in case of malignant tumors, radiotherapy or chemotherapy!
- If pregnant, please apply the tape only according to your doctor's advice!
- Keep out of reach of children!

DURATION OF USE

The tape lasts up to 6 days on the skin.

TECHNICAL INFO

Size 5 cm x 5 m:

- AT 20101 PZN 12407103 Aktimed TAPE PLUS light blue
- AT 20102 PZN 12407126 Aktimed TAPE PLUS pink
- AT 20103 PZN 12407155 Aktimed TAPE PLUS black
- AT 20104 PZN 12407161 Aktimed TAPE PLUS beige
- AT 20105 PZN 12407184 Aktimed TAPE PLUS green
- AT 20106 PZN 12407178 Aktimed TAPE PLUS dark blue

Supporting material: 95% cotton, 5% spandex

Adhesive composition: synthetic rubber based adhesive mass with Arnica montana 10[®], Rhus toxicodendron 10[®], Ruta graveolens 10[®]

Store in a cool, dry place away from direct sunlight between 8°C and 25°C.

CE: Compliant EU Guideline 93/42/EWG for medical products
Manufacturer: Aktimed GmbH, Theaterstr. 18, 69117 Heidelberg, Germany
www.aktimed-tape.de

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