HEINE QUALITY
MADE IN GERMANY

HEINE mini 3000® Combi Lamp

1
2

CE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0
Fax +49(0)81 52/38-202
E-Mail: info@heine.com · www.heine.com

med 3606 1/3.12

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Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries).

We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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Instructions for use

Read these instructions carefully before using the HEINE mini 3000® Combi Lamp and keep them in a safe place for future reference.

| | |
|---|----------|
| <p>Read these instructions carefully before using the HEINE mini 3000® Combi Lamp and keep them in a safe place for future reference.</p> | <p> </p> |
|---|----------|

- In accordance with local regulations this product should be disposed of as an electronic device separately.

- Applied Part BF

Safety information according to the international standard IEC 60601-1
“Electric Safety of Medical devices”
Do not open the instrument / battery handle in the vicinity of the patient.

HEINE mini 3000® Battery Handle

Range of applications:

The HEINE mini 3000® battery handle should only be used for operation to power HEINE mini 3000® instruments.

- Batteries:**
 - 2 alkali cells (AA/ LR 6)
- Important:** Do not use alkali manganese batteries with lithium as bulb life will be reduced.
- The HEINE mini 3000® battery handle can be charged in the HEINE mini NT® with the HEINE mini 2Z rechargeable battery 2,5 V.
- Charging time:**
 - The mini NT indicates when the rechargeable battery is fully-charged. The illuminated ring is lit continuously.
 - mini 2Z rechargeable battery (2,5 V, NiMH) = ca. 4 h (730 mAh)

Instrument connector: Internal thread in handle head.

Switching on: Push switch down.

Switching off: Push switch up. If the handle is put back firmly into a coat pocket the handle is automatically switched off.

Changing the batteries: Unscrew the end-cap and shake out the old batteries/rechargeable battery. Insert new batteries/rechargeable battery with the plus contact facing the handle head.

Cleaning: The handle can be cleaned with a damp cloth (e.g. alkaline or pH- neutrale detergent).

For cleaning disinfection of the outer surface we recommend the so called **wipe disinfection** by using detergents, which are released for Medical Devices made out of **plastic**. Disinfection by spraying or immersion as well as sterilisation is not allowed.

HEINE mini 3000® Combi Lamp

Safety information

Range of applications: The HEINE mini 3000® Combi Lamp is used only for the following examinations:

- as a diagnostic light for general examination
- with a tongue-blade fitted for examination of the mouth and tonsils

Safety in use: Use only handles which are specifically designed for use in medical applications.

Please note: The performance of this instrument can only be guaranteed if genuine HEINE rechargeable batteries or alkali manganese batteries are used.

Instructions

Attaching a tongue-blade: The HEINE disposable tongue-blade should be pushed in as fas as the stop. Wooden tongue-blades are a simple push-fit in the guide (1).

Changing the bulb

Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.

Remove the illuminator from the handle and pull out the bulb by grasping the collar (2).

Cleaning and Maintenance

Do not use alcohol at high concentration. Sterilisation, machine washing, soaking and ultrasonic cleaning are not permitted.

Manual Cleaning: Clean the Combi lamp carefully with the cloth provided or with a damp microfiber cloth. We recommend HEINE Cleaning Spray.

Disinfectant wipes: Use a HEINE microfiber cloth with Isopropyl alcohol at max 70% concentration.

Remove the bulb from the illuminator before sterilization.

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