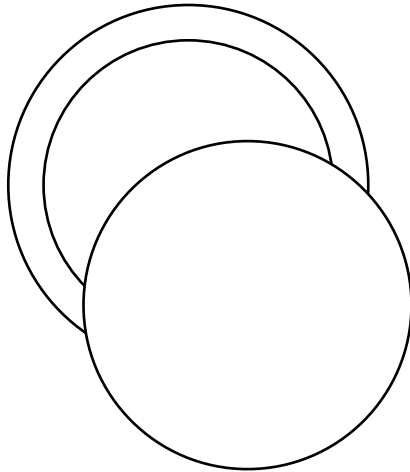
# Drums

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für

DocCheck "Tülip"

Littmann® Classic III™

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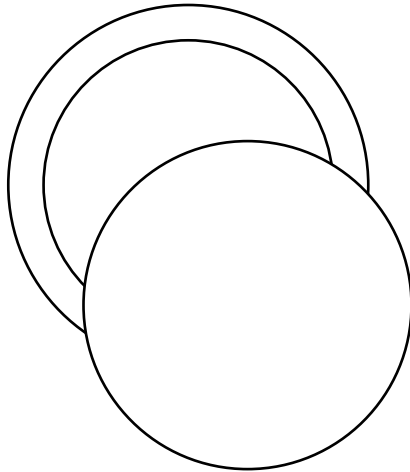
Littmann® Classic III™

### My intended use

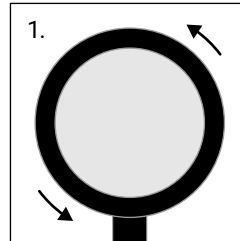
You want to transform your stethoscope into a colorful eye-catcher? Then replace your standard membrane with me, and you'll give your stethoscope a stylish update.

Please note that I am not suitable for every stethoscope. Use me only with the stethoscope models listed on the packaging.


### Unsuitable usage

If you use me differently than described in these instructions, in a way that does not correspond to my purpose, my manufacturer is not liable for damage(s) resulting from this improper use. In this case, you, the user, bear the risk. So be careful!


### Assembly for "Tülip"

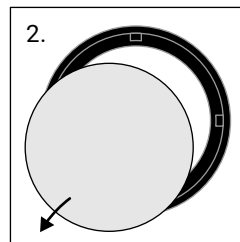
1. Release your stethoscope's locking ring by turning it counterclockwise.

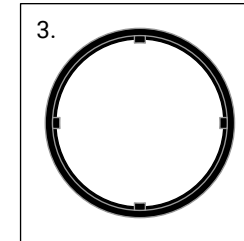2. Remove the old membrane from the ring.

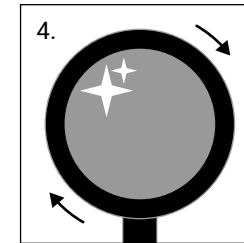
3. Place me in the ring with my white side facing up and make sure I lie flat within the ring.


4. Tighten me with the ring onto the chest piece, so I fit tightly and my beautiful motif smiles at you.


### Assembly for Littmann Classic III

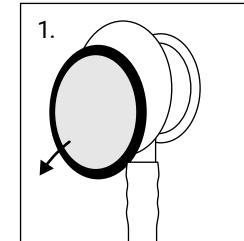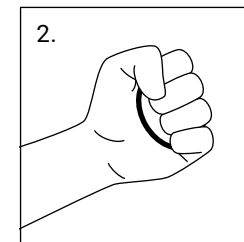
1. Remove the old membrane from your stethoscope.

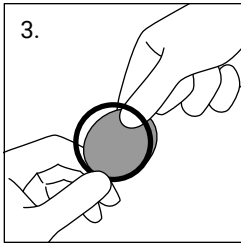
2. Take a matching ring and press it a few times in the palm of your hand. The movement and heat make the ring more flexible and assembly is easier.

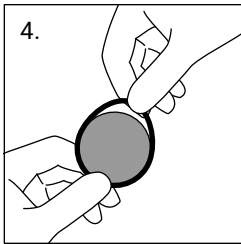Now put me about a quarter of the way into the ring. Don't try to insert me completely into the ring straight away. That could damage my sensitive body.

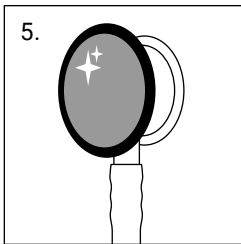Now hold me between your thumb and forefinger and gradually pull me into the ring, always pulling up a small piece of the ring and letting me snap into place. Repeat this all around until I am completely in the ring.

Now you can mount me on the chest piece of the stethoscope as usual. Make sure my beautiful motif is facing up.

If I'm already in the ring when I arrive, just follow this last step.

Please note: Some motifs are only partially printed on (for example, the series "Echo"). The printed side feels rougher on the skin: this side should not come into contact with the patient. When assembling, make sure the smooth side is the one facing upwards.

Please test the functionality of your stethoscope after each replacement!

## Cleaning & Disinfection

Clean and disinfect me regularly. This is of great importance – it serves both to protect the patient, the user and third parties, but also to maintain my quality.

Please make sure you remove all individual parts when cleaning your stethoscope. Reassemble the instrument

only when all surfaces are dry.

Clean my motif with the enclosed cleaning alcohol swabs or a cleaning cloth dampened with ISO alcohol to ensure it stays on for as long as possible. Please do not spray me with cleaning agents.

Check thoroughly that my printed back is dry before reassembling the stethoscope. This ensures you will not harm my motif while cleaning, meaning you will enjoy me for a really long time.

My manufacturer has made everything on its hands to maintain the vibrant color of my design, but environmental factors can affect the motif's longevity. Therefore, please replace me after 30 days of use or if I show discoloration or damage.

## Warranty

If you use me as intended and follow these instructions for use, the warranty for manufacturing and material defects is two years, starting from the date of sale.

Obvious misuse and accidental damage are excluded. Cosmetic damage, such as scratches on the surface, are also excluded from the warranty.

## Disposal

Please only dispose of me in accordance with local / regional / international guidelines.

## Manufacturer

DocCheck Shop GmbH  
Vogelsanger Str. 66, 50823 Cologne, Germany  
[doccheckshop.com](http://doccheckshop.com)

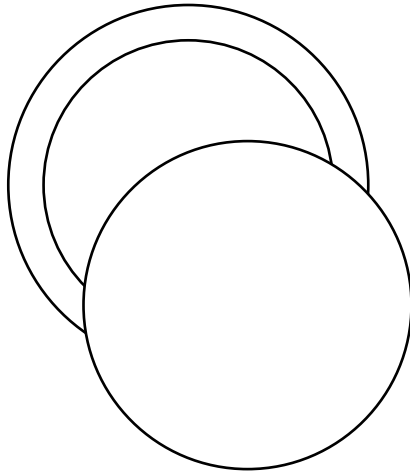
Littmann® and Classic III™ are registered trademarks of 3M™ Company, St. Paul, Minnesota, USA and are in no way affiliated with DocCheck Shop GmbH.

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