## GEBRAUCHSINFORMATION

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www.aktimed-tape.de

## INSTRUCTIONS FOR USE

Aktimed TAPE is an elastic self-adhesive tape bandage (kinesiology tape).

### APPLICATION AREAS

- Kinesiological Taping

### APPLICATION INSTRUCTIONS

- The product can only be used once and needs to be applied by a trained person only.
- Prior to application the skin should be cleaned and fat-free.
- Remove the protection film.
- Apply tape with or without strain.
- The glue unfolds the adhesive effect through body temperature. For this purpose it is advised to smooth out the tape in longitudinal direction from base to end (approx. 60 sec.).
- Apply the tape 30 min prior to sports activity.
- After showering, taking a bath or swimming the tape should be dried with a hair dryer (lowest setting) or by dabbing.
- The tape can occasionally cause skin irritation. In that case please remove the tape.
- The tape can best be removed after taking a shower or a bath.

### SAFETY INSTRUCTIONS

- Do not use in case of an allergy against one of the ingredients, or test the tape on a small area of skin first!
- Do not apply on open wounds, injured skin or unhealed scars!
- Do not use in case of acute or chronic skin diseases (e.g. neurodermatitis, psoriasis, fungal infection)!
- Do not use in case of Thrombosis or Thrombophlebitis!
- Do not use in case of a peripheral arterial occlusive disease distal to the occlusion!
- Do not use in case of malignant tumors, radiotherapy or chemotherapy!
- If pregnant, please apply the tape only according to your doctor's advice!
- Keep out of reach of children!

### DURATION OF USE

The tape lasts up to 6 days on the skin.

### TECHNICAL INFO

AT 20097 PZN 14352829 Aktimed Tape CLASSIC red  
 AT 20096 PZN 05030170 Aktimed Tape CLASSIC dark blue  
 AT 20094 PZN 05030141 Aktimed Tape CLASSIC beige  
 AT 20091 PZN 05030106 Aktimed Tape CLASSIC light blue  
 AT 20095 PZN 05030164 Aktimed Tape CLASSIC green  
 AT 20092 PZN 05030112 Aktimed Tape CLASSIC pink  
 AT 20093 PZN 05030135 Aktimed Tape CLASSIC black

Supporting material: 95% cotton, 5% spandex

Adhesive composition: acrylic adhesive

Size: 5 cm x 5 m

Store in a cool, dry place away from direct sunlight between 5°C and 25°C.

CE: Compliant EU Guideline 93/42/EWG for medical products

Manufacturer: Aktimed GmbH, Theaterstr. 18,

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