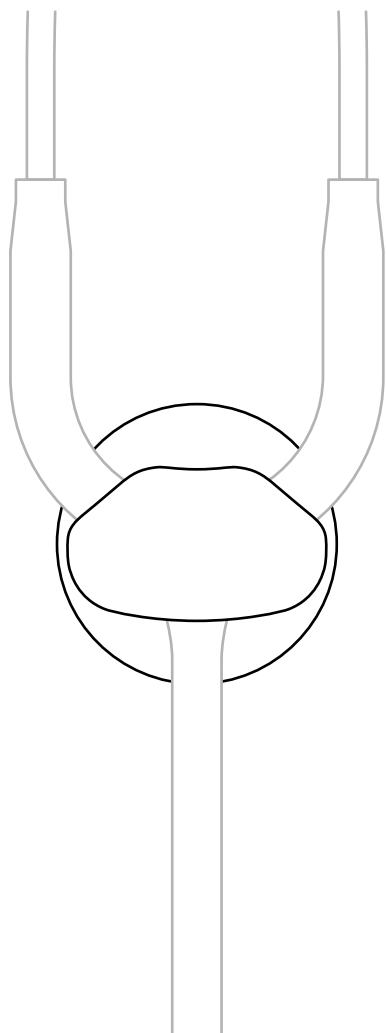
# Smack!

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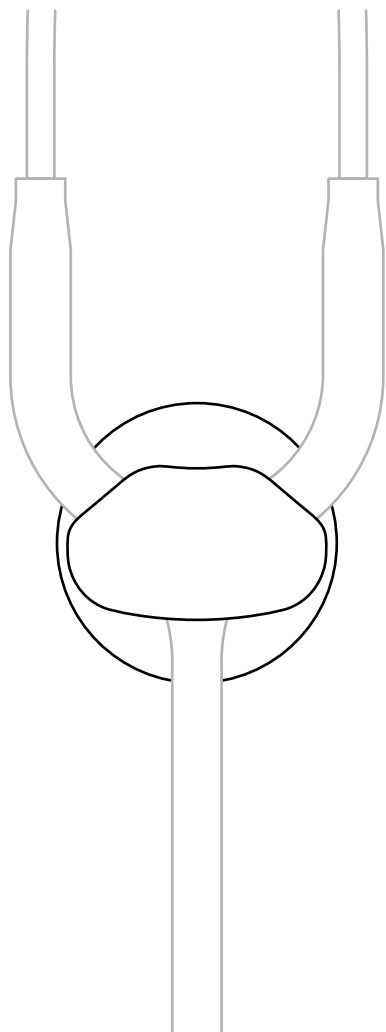
 Artikelnummer

 Charge

 Mein Held  
[doccheck.com](https://doccheck.com)

# Smack!

## User Manual

### My intended use

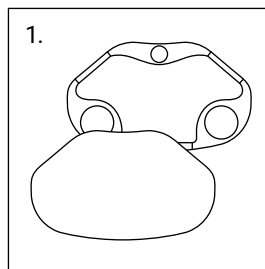
You are tired of losing your stethoscope and not remembering which one is yours? Then I am the perfect solution to your problem!

I can be personalized with an engraved text, serving as a name tag for your stethoscope. Additionally, thanks to my wall mount, I provide a safe haven for your instrument.


### Inappropriate use

If you stray from this manual's directions, then you will not use me correctly. My manufacturer is not liable for any damage that results from inappropriate use. In this case, you alone take responsibility. So please be careful!


### Mounting

Open me up.

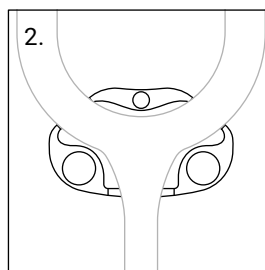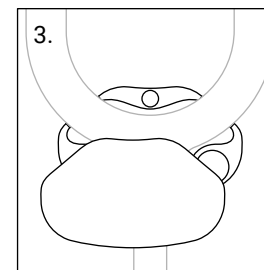
Take one half and place your stethoscope's Y-shaped part in the designated indentation.


Place the second half onto the first. And done!

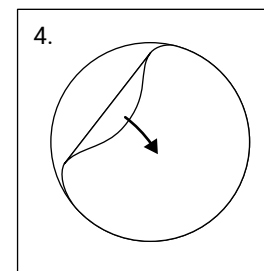My magnet is strong enough to keep me together.

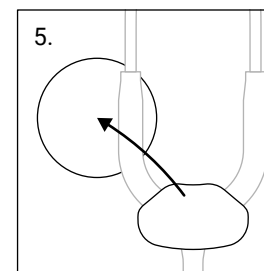Take my wall mount, remove the protective film from the back, and attach it to your chosen spot on the wall.

Now you can simply "smack" me onto the wall mount and keep me there any time you want. Just like that!

### Cleaning and disinfection

Clean and disinfect me regularly. This is of great importance as it serves both to protect the patient, the user and third parties, but also to maintain my quality.

## Safety notice

I have been equipped with magnets to fulfill my role. Please take note of the following information and only use me if it poses no risk to yourself, patients, or third parties.

Permanent magnets create strong magnetic fields that can damage TVs, laptops, computer hard drives, hearing aids, loudspeakers, and other similar devices. Please maintain a distance of at least 1 meter from said products if you are using me.

Keep me away from all (medical) devices or items that can potentially be damaged by strong magnetic fields.

Do not wear me in intensive care units or other areas where life-sustaining medical devices are used.

According to current knowledge, the magnetic fields created by permanent magnets have no measurable positive or negative effect on humans. A health hazard from the magnetic field of a permanent magnet is unlikely but cannot be completely ruled out. For your own safety, avoid permanent contact with the magnets.

Magnets can influence the function of pacemakers or implanted defibrillators. Pacemakers may enter test mode and cause discomfort. Defibrillators might stop working under certain circumstances.

Keep sufficient distance from such devices when wearing me.

Warn people using the devices mentioned above before approaching them.

Maintaining a distance of at least 80 mm is sufficient to ensure safety, given the magnetic force of the magnets used.

Magnets made from neodymium are brittle. Should two magnets collide, they might shatter. Sharp-edged splinters can be launched meters away and injure your eyes.

Avoid collisions between magnets.

Make sure that bystanders are also protected or keep their distance.

I am not a toy. Children may swallow small magnets.

Make sure that my magnets are kept away from children.

Magnets are made from metal and conduct electricity.

## Warranty

If you use me as intended and follow these instructions for use, the warranty for manufacturing and material defects is two years, starting from the date of purchase. Obvious misuse and accidental damage are excluded. Cosmetic damages, such as scratches on the surface, are also excluded from the warranty.

## Disposal

Please refer to local/regional/international guidelines before throwing me away.

## Manufacturer

DocCheck Shop GmbH  
Vogelsanger Str. 66, 50823 Cologne, Germany  
[doccheckshop.com](https://www.doccheckshop.com)

## Explanation of symbols

Manufacturer data (address)

Item number

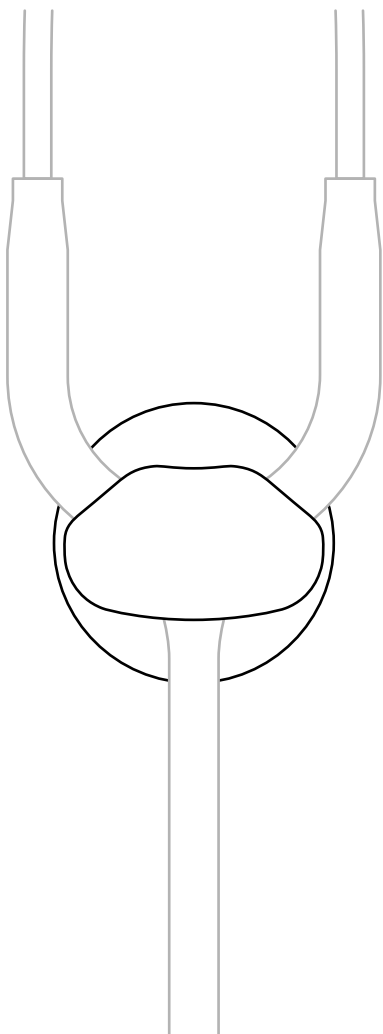
Batch

My hero  
[doccheck.com](https://www.doccheck.com)

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