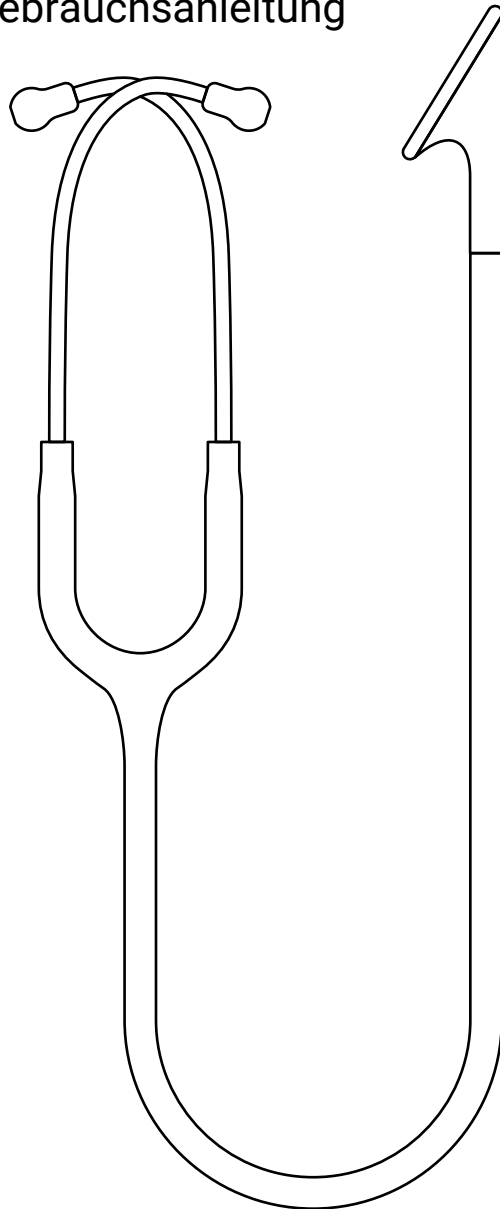
Tülip

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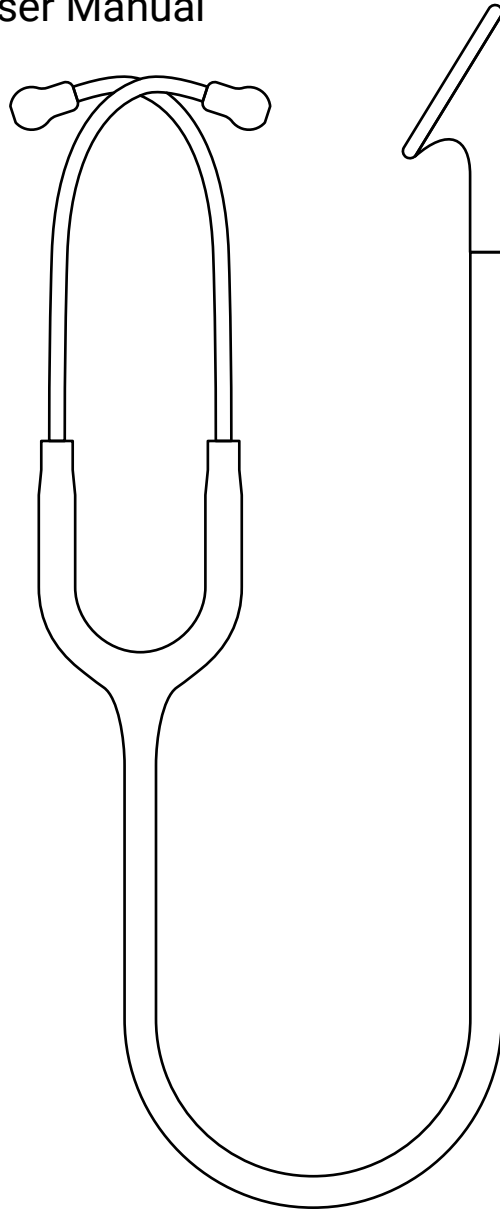
User Manual

My intended use

I, Tūlip, may only be used by authorised and qualified personnel.

I am an acoustic diagnostic tool and help you to hear body sounds better, especially those of the heart and lungs.

If you observe the following instructions, we will maintain a long friendship.

Inappropriate use

If you use me differently to what is described in these instructions for use, this constitutes improper use. My manufacturer is not liable for damage resulting from improper use. In this case only you, the user, bear the risk. So please be careful!

Operation

To be able to use me correctly, first put my ear pieces in your ears. You know that the ear pieces are properly inserted when they point towards the outer ear canal.

You can adjust the tension of my eartubes by pulling them apart or squeezing them together.

Before applying to the patient, please check my membrane by gently rubbing your finger over it. Please wear gloves to avoid transferring bacteria to the membrane.

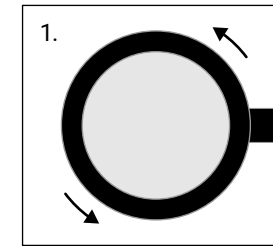
If you can't hear any noise, make sure my components are assembled correctly.

Then I'm ready to go.


For auscultation, place my chest piece on the body area to be examined. Increase the pressure to listen to all frequencies at a higher volume.

My chest piece should be used directly on the skin. This is the only way to guarantee that rustling does not distort the quality of auscultation.


Changing the membrane

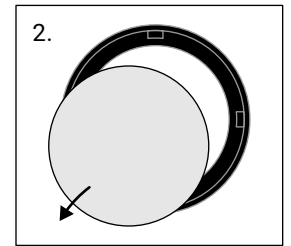
Release the ring by turning it counterclockwise.

Remove the old membrane from the ring.

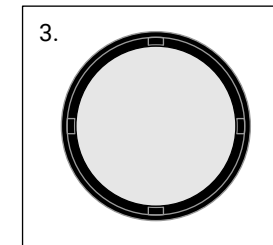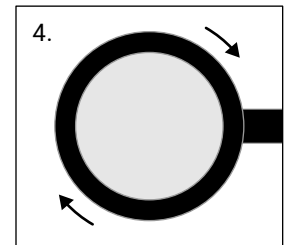Place a new membrane into the ring so that it lays flat.

Tighten the ring and membrane onto the chest piece, so that they fit tightly.

Please do not be afraid that the ring may break when tightening. If it's not tight enough, there's a possibility that it will come off the chest piece.

Safety notice

Please never use me without membrane or ear pieces assembled.

You shouldn't put my ear pieces too deep into your ear canal or use me near strong sources of sound. This can lead to hearing damage!

I am not a children's toy, so please make sure I don't get into the hands of small children.

Materials

I look so elegant because my chest piece is made of aluminium. My fastening ring looks the part too, as it is made of nylon. My tube is made of PVC. This material becomes hard when it comes into contact with the skin for a long time. Therefore, please avoid contact with the skin as much as possible.

Cleaning, disinfection and storage

Clean and disinfect me regularly. This is of great importance – it serves both to protect the patient, the user and third parties, but also to maintain my quality.

Manual cleaning

Please cleanse me, especially my chest piece and membrane, after each patient. It's really easy: Use a cleaning cloth with 70% isopropyl alcohol or soapy water. Keep me away from other liquids or solvents.

Furthermore, I don't like being dipped in liquids, so please refrain from this.

When cleaning, don't forget my tube. You should pay particular attention to this. Care for it with special plastic care products so that it does not harden over time.

For a thorough cleaning, please remove my individual parts. Before you reassemble me, all parts and surfaces should be dry.

In addition, you should disinfect me regularly. If I am particularly dirty, you can use a 2% bleach solution. Please note that excessive use can lead to discoloration of the tube.

Sterilisation

If you want to enjoy me for a long time, please don't sterilise me.

Storage

After use, put me back in the packaging or in a safe place so that I don't get any defects on my chest piece!

In addition, you should not expose me to strong UV radiation or heat, otherwise this can lead to fading or hardening of my material.

Maintenance and warranty

You can replace my ear pieces or membrane at any time.

The warranty for the entire product expires when using spare parts that were not purchased via the DocCheck shop.

If you use me as intended and follow these instructions for use, the warranty for manufacturing and material defects is two years, starting from the date of sale. Obvious misuse and accidental damage are excluded. Cosmetic damage, such as scratches on the surface, are also excluded from the warranty.

Check and functional test

Before use, please check that all my parts such as membranes or ear pieces are assembled and securely in place.

Visual inspection for damage and wear is also useful.

Disposal

Please only dispose of me in accordance with local / regional / international guidelines.

Manufacturer

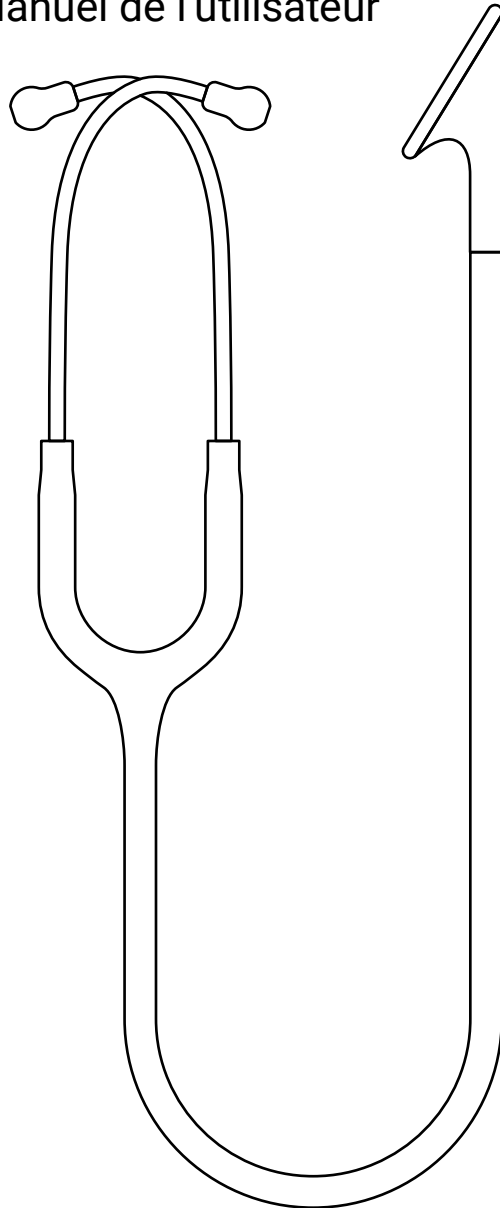
DocCheck Shop GmbH
Vogelsanger Str. 66, 50823 Cologne, Germany
doccheckshop.com

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