<b>de</b>	Gebrauchsanweisung	Ophthalmoskope
<b>en</b>	User's Manual	Ophthalmoscopes
<b>fr</b>	Mode d'emploi	Ophthalmoscopes
<b>it</b>	Istruzioni per l'uso	Oftalmoscopi
<b>es</b>	Instrucciones de empleo	Oftalmoscopios
<b>pt</b>	Manual de operação	Oftalmoscópios
<b>ru</b>	Руководство по применению	Офтальмоскопы

**KaWe EUROLIGHT**  
**KaWe PICCOLIGHT**

**CE**

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- de** Bitte lesen Sie diese Gebrauchsanweisung vor der Benutzung sorgfältig und vollständig durch und beachten Sie die Pflegehinweise.
- en** Please read this User's Manual thoroughly and carefully before attempting to use this product and heed the given care instructions.
- fr** Lisez ce mode d'emploi attentivement et entièrement avant d'utiliser l'appareil et respectez les consignes d'entretien.
- it** Si prega di leggere attentamente queste istruzioni per l'uso prima di utilizzare lo strumento e di seguire i consigli per la manutenzione.
- es** Por favor, lea con atención las presentes instrucciones de empleo en su totalidad y siga las indicaciones referentes al cuidado del aparato.
- pt** Antes de utilizar este produto pela primeira vez, favor de ler com muita atenção todo este manual de operação e observar as indicações relativas à manutenção.
- ru** Перед использованием следует полностью и внимательно прочитать настоящее руководство по применению и соблюдать указания по уходу!

<b>Deutsch</b>	5-11	<b>de</b>
<b>English</b>	12-18	<b>en</b>
<b>Français</b>	19-25	<b>fr</b>
<b>Italiano</b>	26-32	<b>it</b>
<b>Español</b>	33-39	<b>es</b>
<b>Português</b>	40-46	<b>pt</b>
<b>Русский</b>	47-53	<b>ru</b>

<b>EU</b>						
<b>de</b>	Spaltblende	großer Kreis	kleiner Kreis	Halbkreis	Rotfrei / grün großer Kreis	Fixierhilfe
<b>en</b>	Slit aperture	large spot	small spot	Semi-circle	Red-free / green large spot	Fixation target
<b>fr</b>	Diaphragme à fente	Grand cercle	Petit cercle	Demi-cercle	Exempt de rouge / vert grand cercle	Aide de fixation
<b>it</b>	Diaframma a fessura	Luce grande	Luce piccola	Mezza luce	Luce grande con filtro anerito	Ausilio per fissazione
<b>es</b>	Diafragma de hendidura	Círculo grande	Círculo pequeño	Semicírculo	Verde carente de rojo círculo grande	Ayuda de fijación
<b>pt</b>	Diafragma de fenda	Círculo grande	Círculo pequeno	Semi-círculo	Isento de vermelho / verde círculo grande	Elemento auxiliar de fixação
<b>ru</b>	Щель	Большой круг	Малый круг	Полукруг	Бескрасный / зеленый большой круг	Точка фиксация

<b>US</b>						
<b>de</b>	Spaltblende	großer Kreis	kleiner Kreis	Kobaltblau großer Kreis	Rotfrei / grün großer Kreis	Fixierhilfe
<b>en</b>	Slit aperture	large spot	small spot	Cobalt blue large spot	Red-free / green large spot	Fixation target
<b>fr</b>	Diaphragme à fente	Grand cercle	Petit cercle	Bleu de cobalt grand cercle	Exempt de rouge / vert grand cercle	Aide de fixation
<b>it</b>	Diaframma a fessura	Luce grande	Luce piccola	Blu di cobalto luce grande	Luce grande con filtro anerito	Ausilio per fissazione
<b>es</b>	Diafragma de hendidura	Círculo grande	Círculo pequeño	Azul de cobalto círculo grande	Verde carente de rojo círculo grande	Ayuda de fijación
<b>pt</b>	Diafragma de fenda	Círculo grande	Círculo pequeno	Azul de cobalto círculo grande	Isento de vermelho / verde círculo grande	Elemento auxiliar de fixação
<b>ru</b>	Щель	Большой круг	Малый круг	Кобальтовая синь большой круг	Бескрасный / зеленый большой круг	Точка фиксация

---

## Inhalt

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---

## Table of contents

<b>1. Symbols used in the application manual and on packaging</b>	13
<b>2. Purpose and intended use</b>	13
<b>3.  Safety rules/contraindications</b>	14
<b>4. Installation and maintenance</b>	14
Duration of operation	14
KaWe EUROLIGHT E10/E15/E16	14
KaWe EUROLIGHT E30/E36	15
KaWe EUROLIGHT E25/ E35	15
KaWe PICCOLIGHT E50/ E56	16
<b>5. Preparing for operation</b>	16
<b>6. Hygienic preparation</b>	16
Manual cleaning/disinfection	16
Automatic cleaning	16
Sterilization	16
<b>7. Transportation and storage</b>	16
Environment parameters	16
<b>8. Maintenance and Service</b>	16
Battery replacement	17
Lamp replacement	17
<b>9. Accessories and Spare parts</b>	17
<b>10. Disposal</b>	17
<b>11. Annex</b>	18
Warranty & Guarantee	18
Contact information	18

We would like to thank our valued customers for their decision to purchase a KaWe product. Our ophthalmoscopes are characterized by high quality and durability. KaWe ophthalmoscopes meet the requirements of Regulation (EU) 2017/745 (European Regulation on Medical Devices) and are classified as medical devices class I.

 Please read this manual carefully, adhere to it strictly, and keep it for future reference if necessary. Before using the product, carefully study operation rules, especially care requirements.

If you have any questions concerning device operation, please contact the service department (the manufacturer's address is indicated on the back).

KaWe ophthalmoscopes are available in two models:

- EUROLIGHT: Models with metal handle and plastic button
- PICCOLIGHT: Models with plastic handle and button

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Images are shown on the cover of this application manual. The difference between the two models are described in this application manual. Other options (battery operation, light sources, etc.) are not discussed in detail, as they do not have significant differences in service.

## 1. Symbols used in the application manual and on packaging

	ATTENTION!
	Please follow this User's Manual
	Item No.
	Manufacturer's number, lot number
	Manufacturer & date of manufacture
	Store in dry conditions
	Temperature range limits
	Separate collection of electrical / electronic equipment and rechargeable batteries
	CE conformity mark
	Medical product
	UDI Data Carrier

en

## 2. Purpose and intended use

KaWe ophthalmoscopes are diagnostic instruments and are primarily used for optical examination of the fundus by direct ophthalmoscopy. In this method, the central elements of the visual organ, such as optic disc, the vascular bundle, macula (Makula lutea) are examined. The ophthalmoscope allows examinations without the use of mydriatics. However, in this case, the patient's field of vision is limited.

- Target patient groups: Patients of all ages and genders who require visual examination.
- The product must only be used by authorized specialized personnel!

Intended use is the compliance with all instructions in this application manual as well as basic rules and directives. The manufacturer does not accept any liability for damage resulting from any other use.

### 3. Safety rules / contraindications

- Before each use, please check whether the ophthalmoscope is operating properly. Do not use the device in case any damage is detected.
- Ensure the ophthalmoscope is protected from moisture.
- Avoid direct contact of the ophthalmoscope with the visual organ.
- If the light is on for a long time, the areas around the lamp may become hot.
- KaWe ophthalmoscopes are not intended for use during surgery.
- Use only KaWe MedCentether 5000 wall-mounted station for power supply of KaWe EUROLIGHT E25 head.
- Do not use the device in areas subject to strong magnetic fields (MRI systems).
- When handling ophthalmoscopes – as with all energy-consuming diagnostic devices – special electromagnetic compatibility (EMC) precautions must be observed.
- The performance of ophthalmoscopes can be affected by mobile high-frequency communication devices.
- Only original spare parts and accessories from KaWe may be used.

In case of a malfunction, a large amount of heat can be generated, resulting in an increased risk of fire. In case of strong heat, turn off the handle.

### 4. Installation and maintenance

When handling the instrument, it is necessary to take into account that the examiner must bring his eye as close as possible to the instrument. Due to the lateral illumination of the patient's fundus area and the approach to a maximum distance of 2-3 cm, diagnostics can be carried out without drugs.

#### Duration of operation

The devices are not intended for continuous operation, the maximum duration of one use is 1 minute with a 10 minutes pause between uses.

### KaWe EUROLIGHT E10/ E15/ E16

- |   |  |
|---|--|
| (1) Light outlet (dustproof)                                  | (5) Correction value indication (+) black; (-) red |
| (2) Blend wheel (except E10 model)                            | (6) Plug connector                                 |
| (3) Inspection window   | (7) Light regulator on/off                         |
| (4) Correction lens wheel with 18 lenses ( $\pm 20$ diopters) | (8) Handle with battery                            |
|   | (9) End cover                                      |

- Place the ophthalmoscope head on the handle with battery (8) and secure it with the screw. Consider the location of the holes on head for securing the screw.
- Insert batteries or accumulator into the handle. Observe the polarity and the lamp voltage (2.5 or 3.5 V)! For detailed information see Section 8: Maintenance and Service.
- Switching on is carried out by pressing the white switch with fixing the on position and moving the light regulator to the left (7). To switch off, move the light regulator to the initial position 0 (until it clicks!). Brightness can be adjusted smoothly.

## KaWe EUROLIGHT E30/ E36

- (1) Light outlet (dustproof)  
 (2) Blend wheel (except E30 model)  
 (3) Inspection window  
 (4) Correction lens wheel with 18 lenses ( $\pm 20$  diopters)  
 (5) Correction value indication (+) black; (-) red  
 (6) Clic connector  
 (7) Light regulator on/off  
 (8) Handle with battery  
 (9) End cover

- a) Place the ophthalmoscope head on the handle with battery (8) and secure it until it clicks. Observe the position of the cutout.  
 b) Insert batteries or accumulator into the handle. Observe the polarity and the lamp voltage (2.5 or 3.5 V)! For detailed information see Section 8: Maintenance and Service  
 c) Switching on is carried out by pressing the white switch with fixing the on position and moving the light regulator to the left (7). To switch off, move the light regulator to the initial position 0 (until it clicks!). Brightness can be adjusted smoothly.

## KaWe EUROLIGHT E25/ E35

- (1) Closing blend  
 (2) Blend wheel  
 (3) Filter 4000 K (F)  
 (4) Filter arm  
 (5) Polarizing filter for lighting (P)  
 (6) Attachment lens wheel  
 (7) Correction value indication (+) green; (-) red  
 (8) Correction lens wheel 71 Lenses (-36 to +35 diopters)  
 (9) Inspection window  
 (10) Polarizing filter for inspection window

**Attention:** When the ophthalmoscope is not operated, close the inspection window (9) with a blend (1) to prevent dust from entering the head!

Blend	Explanations
I	Gap
○	Small circle
○	Big circle
⊗	Fixing point
RF	Green filter (without red color)

Attachment lens wheel position	Diopter range
•	-12 to +11 diopters
-	-13 to -36 diopters
+	+12 to +35 diopters

### E35 head installation

see KaWe EUROLIGHT E30/E36

### E25 head: KaWe MedCenter 5000 installation

The KaWe EUROLIGHT E25 head can only be used with the KaWe MedCenter 5000 wall station.

- a) Insert the E25 head into the handle of the KaWe MedCenter 5000 device.  
 b) Note that the centering hole must be lined up with the support on the retaining screw.  
 c) Tighten the retaining screw lightly.  
 d) To remove the head, unscrew the retaining screw and carefully pull out the head.

Observe the KaWe MedCenter 5000 manual instructions.

en

## KaWe PICCOLIGHT E50/ E56

- (1) Light outlet (dustproof) (5) Correction value indication (+) black; (-) red  
(2) Blend wheel (except E50 model) (6) Thread  
(3) Inspection window (7) On/off slide switch

- (4) Correction lens wheel with 18 lenses ( $\pm 20$  diopters)

- a) Install batteries into the handle (2 pcs. AA 1.5V minion). Observe the polarity. For detailed information see Section 8: Maintenance and Service  
b) Screw the head onto the handle (6).  
c) Move the slide switch (7) to turn the lamp on and off.  
d) Battery and lamp replacement is possible after unscrewing the head. For detailed information see Section 8: Maintenance and Service

### 5. Preparing for operation

- Periodically check that all parts are in good condition - replace defective parts.
- Check the smooth running of moving parts (e.g. correction lens /blend wheel, plug connector).
- Keep all electrical contacts clean.
- Check the batteries charge.

After a successful inspection, the ophthalmoscope is ready for use.

### 6. Hygienic preparation

Keep the ophthalmoscope in its packaging to protect it from dirt and dust.

### Manual cleaning/disinfection

#### Protect the device from moisture!

Do not place the ophthalmoscope in liquids, make sure that no liquid gets into the casing.

Clean the outside of the ophthalmoscope with a damp, soft, lint-free cloth. If necessary, wipe the light outlet glass with a cotton swab slightly moistened with alcohol.

- Do not press on the glass.
- Do not use abrasive cleaning agents!

For disinfection, the ophthalmoscope can be wiped with an alcohol-based surface disinfectant. Use a slightly damp cloth.

### Automatic cleaning

Not applied.

### Sterilization

Not applied.

### 7. Transportation and storage

Ophthalmoscopes are to be stored and transported protected from dust, moisture and contamination.

### Environment parameters

	Temperature	Relative humidity	Air pressure
Operation	+10°C to +35°C	30% to 90%	800 hPa to 1060 hPa
Storage	-10°C to +55°C	10% to 95%	700 hPa to 1060 hPa
Transportation			500 hPa to 1060 hPa

### 8. Maintenance and Service

The end of the useful life is usually determined by wear and tear. Damaged parts are to be sorted out and replaced immediately. If the light intensity decreases, check for the following possible causes:

- Battery charge; replace or recharge if necessary.
- Clean or replace the lamp.

If the above reasons are excluded, repair or replacement may be required. In this case, please, contact a specialized dealer.


## Battery replacement

Check the batteries charge regularly. Low battery voltage (depending on consumption) leads to light intensity decrease and possible appearance of LED lamps flashing. In both cases, it's necessary to replace the batteries. When replacing, use only new, high quality alkaline batteries (or fully charged batteries).


Besides, observe manuals' requirements for use of the batteries.


Installation of batteries for KaWe EURO-LIGHT, 2 pcs. medium size 1.5V (type C)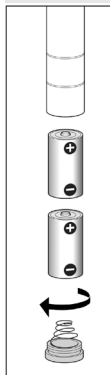

Installation of batteries for KaWe EURO-LIGHT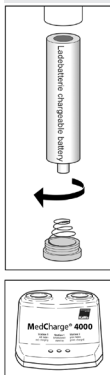

Installation of batteries for KaWe PICCOLIGHT, 2 pcs. minion type 1.5V (type AA)

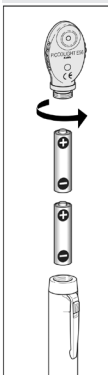## Lamp replacement

The correct performance of the ophthalmoscope with original light quality is only ensured when using original KaWe replacement lamps.

## KaWe EUROLIGHT E10

1. Unscrew the retaining screw on the handle and remove the ophthalmoscope head from the handle.
2. Unscrew the cap ring (1).
3. Pull out the lamp holder (2) on the bottom of the head and replace the lamp.
4. Clean the glass bulb (3) of the new lamp with alcohol. The glass bulb must be clean and free from fingerprints (grease free).
5. Install the lamp holder and lamp back into the head.
6. Screw the cap ring.

## KaWe EUROLIGHT E15/ E16/ E25/ E30/ E35/ E36

1. Remove the ophthalmoscope head from the handle.
  - 1.1. Models E15/E16:  
Unscrew the retaining screw on the handle.
  - 1.2. Models E25/E30/E35/E36:  
Push down on the wickered sleeve.
2. Remove the lamp at the bottom of the head.
3. Clean the glass bulb of the new lamp with alcohol. The glass bulb must be clean and free from fingerprints (grease free).
4. Insert the lamp into the head until it stops.

## KaWe PICCOLIGHT E50/ E56

1. Unscrew the ophthalmoscope head from the handle.
2. Remove the lamp at the bottom of the head.
3. Clean the glass bulb of the new lamp with alcohol. The glass bulb must be clean and free from fingerprints (grease free).
4. Insert the lamp into the head until it stops.

## 9. Accessories and Spare parts

Accessories and spare parts can be found in the general catalog of our company at <https://www.kawemed.com/download/broschueren/> or can also be viewed at your specialist dealer.

## 10. Disposal

The product is to be disposed in accordance with local environmental regulations. Disposal instructions are also shown on the pictograms on the device or packaging.

Defective and/or recyclable electrical / electronic devices and rechargeable batteries must be transferred to a designated waste collection point for recycling.

### **Instructions for service personnel and patients**

In case of problems during the device use, it is necessary to immediately contact the manufacturer and the competent authorities of the relevant member state of the Union, in which the service personnel or the patient are located.

## **11. Annex**

---

### **Warranty & Guarantee**

In case of proper handling and adherence to the rules laid down in this application manual, as well as the use of original KaWe spare parts, the statutory warranty period of two years from the date of sale is valid (excluding lighting equipment, batteries and accumulators). LED lamps are warranted for five years from the date of sale.

For other questions and if repair is required, contact your specialized dealer.

### **Contact information**

Please contact your specialist dealer or directly the manufacturer (see back for address).

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**KIRCHNER & WILHELM GmbH + Co. KG**