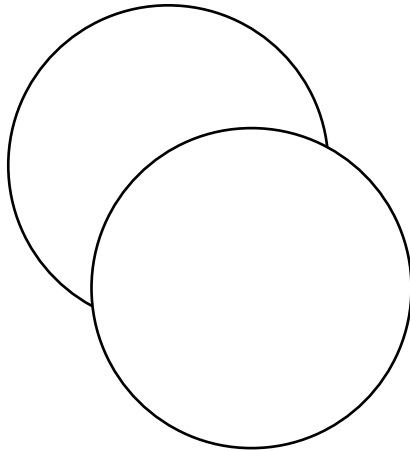
Cøpper

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passend für:

DocCheck "Tülip"

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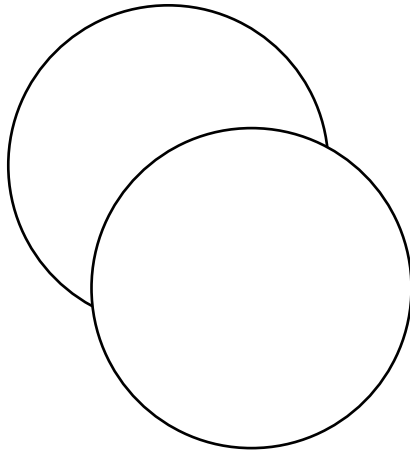
DocCheck "Tülip"

My intended use

I, Cøpper, am an effective antimicrobial copper membrane that you can insert into your stethoscope instead of conventional membranes. As soon as you have attached me to the chest piece of the stethoscope with the corresponding fastening ring, you can use it as usual.

Only use me with the stethoscope models listed on the packaging.

Effectiveness

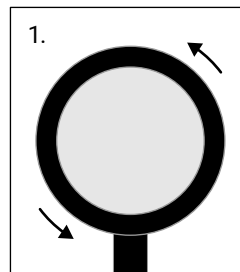
Studies show that I can reduce bacteria and microbial activity. This works through the reaction between my copper surface and the microbes as soon as they touch me.

I reduce the following bacterial strains by 99.9 % within 24 hours:


- MRSA
- E. coli
- Staphylococcus aureus

However, I am of course not a substitute for responsible, hygienic treatment of your patients!


Assembly

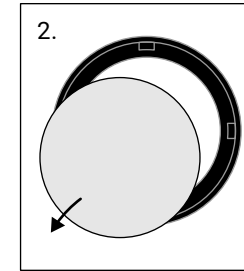1. Release your stethoscope's locking ring by turning it counterclockwise.

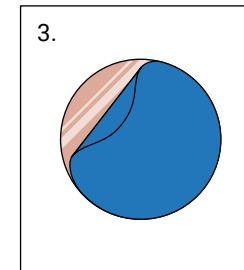
2. Remove the old membrane from the ring.


3. Take me out of the packaging and carefully remove the blue protective film.

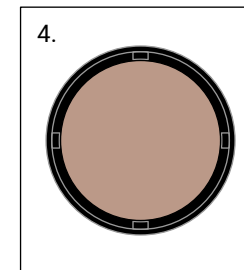I am very thin and my edges can be sharp. Therefore don't hold me by the outside edge.

4. Place me into the ring so that I lay flat.

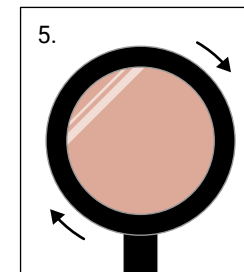Make sure that my shiny side is facing down – this side will come into contact with the patient later.

5. Lock me with the ring onto the chest piece, so that I fit tightly. And you're done!

Please test my functionality again after each replacement.

Due to my materials, my acoustic profile may differ from the original membrane supplied with your stethoscope. Please ensure you familiarise yourself with this profile before using me for diagnostic purposes.

Safety information

I'm 99.9 % copper. A small proportion of the population may be allergic to this material. Do not use me if you suspect that you or your patient has such an allergy.

Be careful not to damage my surface with sharp objects. That can shorten my lifespan.

I am electrically conductive.

Any use other than that described in these instructions for use contradicts the intended purpose. My manufacturer assumes no liability for damage of any kind caused by improper use.

Cleaning & maintenance

I can tolerate ISO alcohol just like conventional stethoscopes. Clean my surface with a smooth cloth, dampened with ISO alcohol, to remove dust and dirt.

After a while, my surface colour could change, which is caused by a reaction with oxygen. My beautiful copper sheen then changes to a darker brown. This is a clear signal to replace me.

My manufacturer recommends replacement after 30 days of normal use.

Disposal

Please only dispose of me in accordance with local / regional / international guidelines.

Manufacturer

DocCheck Shop GmbH
Vogelsanger Str. 66, 50823 Cologne, Germany
[doccheckshop.com](https://www.doccheckshop.com)

Explanation of symbols

 Medical device

 CE mark

 Manufacturer data (address)

 Production date

 Article number

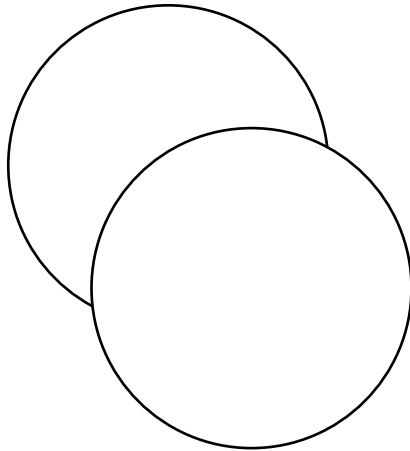
 Batch

 Contains copper

 My hero
[doccheck.com](https://www.doccheck.com)

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