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Zubehör für

MELAseal[®] 200

MELAseal[®] 100+

DE

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EN

Dear customer,

We thank you for your confidence demonstrated by the purchase of this MELAG product.

As an owner-run and operated family concern founded in 1951, we have a long history of successful specialization in hygiene products for practice-based use. Our focus on innovation, quality and the highest standards of operational reliability has established MELAG as the world's leading manufacturer in the instrument treatment and hygiene field.

You, our customer are justified in your demand for the best products, quality and reliability. Providing "**competence in hygiene**" and "**Quality – made in Germany**", we guarantee that these demands will be met. Our certified quality management system is subject to close monitoring: one instrument to this end is our annual multi-day audit conducted in accordance with ISO 13485. This guarantees that all MELAG products are manufactured and tested in accordance with strict quality criteria.

The MELAG management and team

Roll dispenser "Deluxe"

(Art. no.: 10108)

Scope of delivery

Please check the scope of delivery before setting up and connecting the roll dispenser "Deluxe".

Standard scope of delivery

- Roll dispenser "Deluxe"
- Operating manual
- 4x Spacer washers
- Shaft
- Guide plate for MELAfol

to be used in conjunction with

- MELAseal 200
- MELAseal 100+

Intended use

The roll dispenser "Deluxe" enables space-saving storage of the film rolls above the sealing device. The film can be fed to the front of the device over the sealing area using the integrated roll feeder.

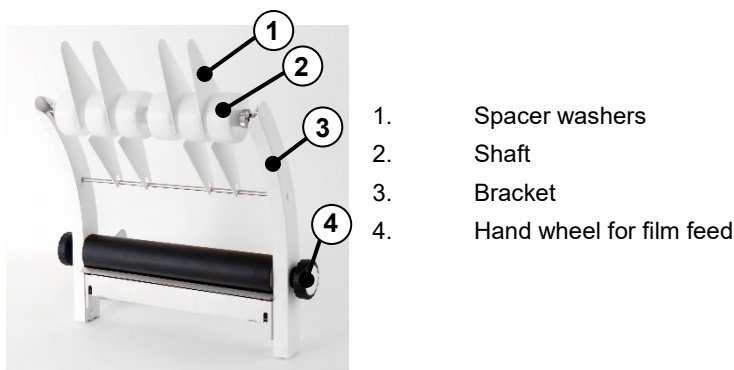
Fig. 1: Roll dispenser "Deluxe"

Assembly

1. Insert the roll dispenser "Deluxe" in the bracket (a) located at the rear of the sealing device
2. Insert the guide plate (b) in the roll dispenser bracket (c) from above.

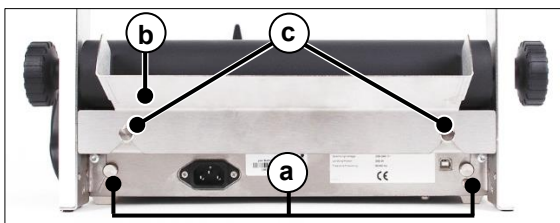
Fig. 2: Introducing the roll dispenser on the sealing device

Application

Fig. 3: Wheel for film feed

Hand wheel for film feed

The film roll can be transported forwards through the sealing device by turning the hand wheel (Fig. 3).


PLEASE NOTE

Only one film roll may be transported with the hand wheel at a time.

Spacer washers

With multiple film rolls, use the spacer washers (see fig. 1, pos. 1) can be clipped in between the shaft segments to separate and at the same time, fix the film rolls.

Usable working width on the roll dispenser "Deluxe"

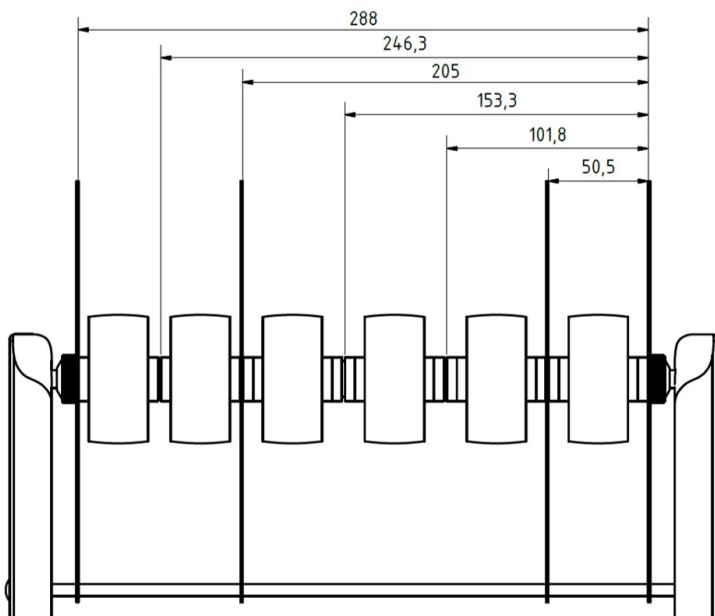Drawing 1: Dimensions and combination possibilities of the film rolls

Technical Data

Model name	Roll dispenser "Deluxe"
Device dimensions (W x D x H)	41.1 x 21.8 x 43 cm
Weight	3 kg
Working width	28.8 cm

MELAG Medizintechnik GmbH & Co. KG

Geneststraße 6-10
10829 Berlin
Germany

E-Mail: info@melag.de

Web: www.melag.com

Original instructions

Responsible for content: MELAG Medizintechnik GmbH & Co. KG

We reserve the right to technical alterations

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Responsabile dei contenuti: MELAG Medizintechnik GmbH & Co. KG