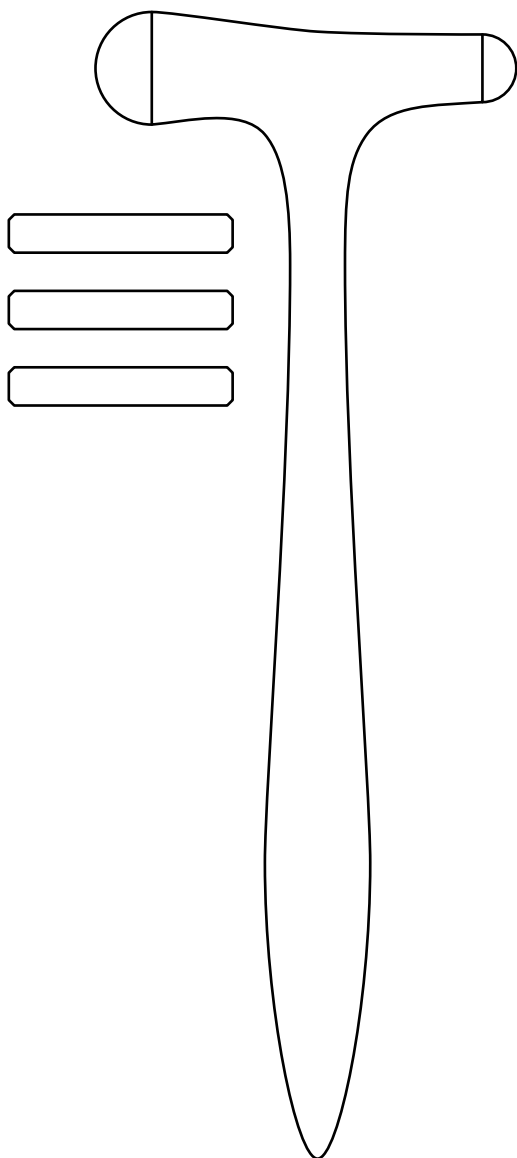
Clôppe Pro

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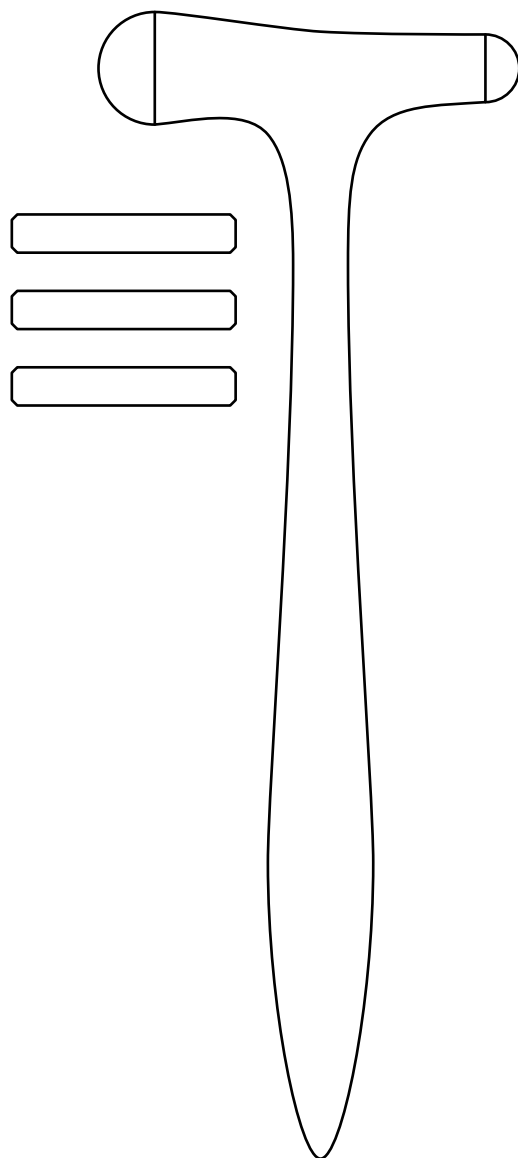
User Manual

My purpose

I, Clôppe Pro, am only intended to be used by healthcare professionals for medical diagnosis. I am a diagnostic instrument for percussion (medical tapping) and for triggering reflexes. And yes, I am reusable.

Unsuitable usage

If you use me differently than described in these instructions, in a way that does not correspond to my purpose, my manufacturer is not liable for damage(s) resulting from this improper use. In this case, you, the user, bear the risk. So be careful!

Usage

You strike me from the wrist, swinging loosely, with one of the two hammer heads on the corresponding tendon or the periosteum into which the tendon radiates.

Please strike carefully and not too hard. This causes a reflex contraction, i.e. a contraction of the relevant muscle.

Depending on how heavy you would like me to be, you can adjust my weight by swapping the metal inserts in my head. This is your personal preference and not dependent on the reflexes you want to test.

As you can see, I taper at the bottom of the handle. With this tip you can check the Babinski reflex on the sole of the foot

Important safety advice

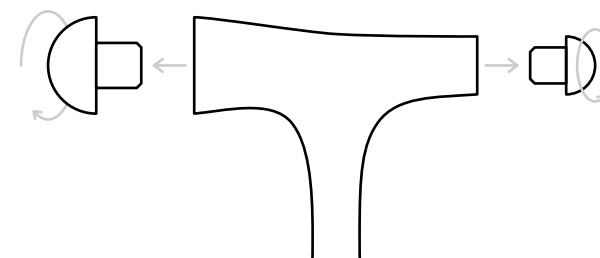
Because of my small parts that can be swallowed (rubber heads, weights), you should not put me in the hands of children under 3 years of age! Believe me, toddlers love to nibble off my attractive rubber parts.

Materials

I have a beautiful shape because I am made entirely of aluminum. My rubber heads are made of silicone. Furthermore, you can use three different metals to alter my weight. The tungsten weight is already pre-assembled for you. If you prefer me a little lighter, just swap it for one of the other weights. Depending on which weight you choose, I weigh 146 g with aluminum, 154 g with stainless steel, and with tungsten I come to a proud 164 g. Please note that this weight may vary slightly due to manufacturing tolerances.


Removing or replacing the hammer heads

My rubber heads are held in place by a screw connection. If you want to remove them, turn them counter-clockwise and pull them out gently. If you want to unscrew the heads or tighten them, turn them clockwise with pressure. Please do this with a lot of sensitivity so that the heads do not break off.

Removing or replacing the weights

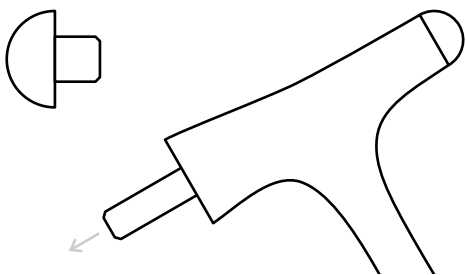To swap the weights, remove the large rubber head as described in the previous section. Now you can remove the weight by tilting me. The weight slides out of my head and you can insert another one. Finally, you just have to put the rubber head back on and I'm ready to go.

Cleaning, disinfection and storage

Manual cleaning

Please ensure to clean me regularly. It really is quite easy: Remove my rubber heads as described above and thoroughly and wipe all my individual parts with a damp cloth. My manufacturer recommends cleaning wipes with 70% isopropyl alcohol or soapy water. Please do not put me back together until I am fully dry.

If you use other cleaning agents or disinfectants, please test if they are compatible first otherwise you could damage my pleasant appearance.

Frequent cleaning has little effect on me, and I love to be clean!

Sterilisation

You CANNOT sterilise me. I will break, which would be sad for us both.

Storage

I am an unpretentious guy. Just store me dry and at room temperature. But I also have nothing against cool rooms. I don't like excessive UV exposure and heat, which can lead to my material becoming bleached, brittle or hardened.

Maintenance and warranty

If parts of mine are damaged, you can sort them out or replace them (rubber heads).

If you use me as intended and follow these instructions for use, the warranty on manufacturing and material defects is two years, starting with the date of sale. Obvious misuse and accidental damage are excluded. So please don't use me to drive nails into the wall.

Control and functional tests

Please check before use that my removable parts (rubber heads, weights) are firmly seated.

A visual inspection for damage and wear is also useful.

Disposal

Please only dispose of me in accordance with local/regional/international guidelines.

Note to the user and/or patient:

All serious incidents related to me must be reported to DocCheck Shop GmbH and the competent authority of the Member State.

Manufacturer

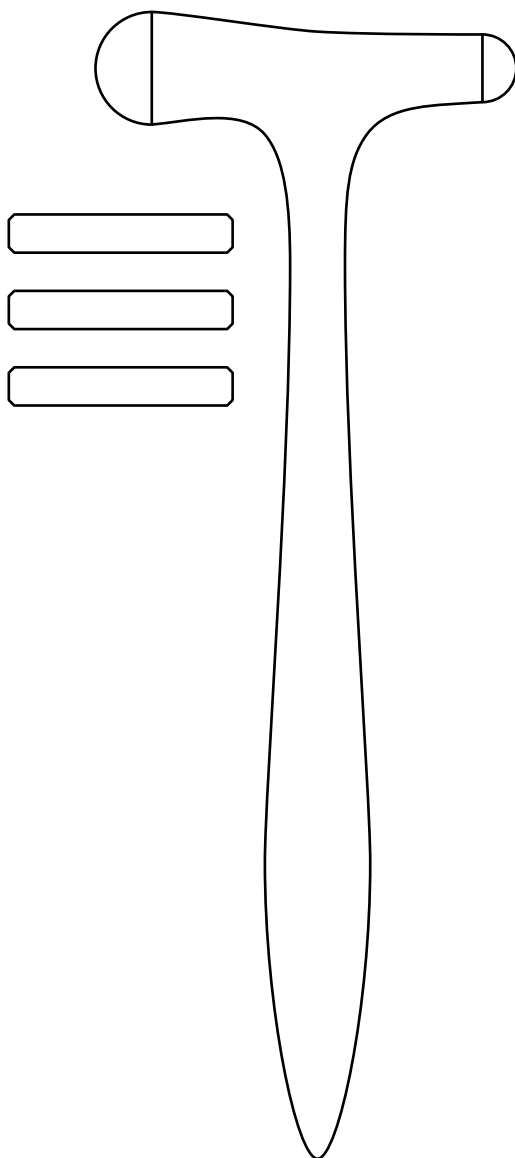
DocCheck Shop GmbH
Vogelsanger Str. 66, 50823 Cologne, Germany
doccheckshop.com

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