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
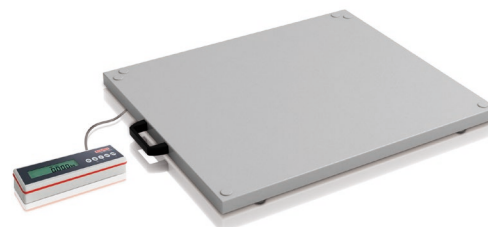
Thank you for choosing this Soehnle Professional product. It is equipped with all features of the very latest technology and is optimised to make operation as simple as possible. If you have any questions, or if there is any problem with the equipment that cannot be resolved with this user manual, please contact your dealer or service partner, or visit our website at www.soehnle-professional.com.

1. MODELS OF SCALES

7702.37.002	Invalid chair scales with display unit Basic E
7702.65.002	Invalid chair scales with display unit Standard
7708.37.xxx	Multifunction scales /Invalid chair scales with display unit Basic E
7708.65.xxx	Multifunction scales /Invalid chair scales with display unit Standard
3705.15.001	Display unit Basic E, verifiable, bench version
3705.55.001	Display unit Basic E, verifiable, wall version
3710.15.001	Display unit Standard, verifiable, bench version
3710.55.001	Display unit Standard, verifiable, wall version

1.1 PRODUCT DESCRIPTION

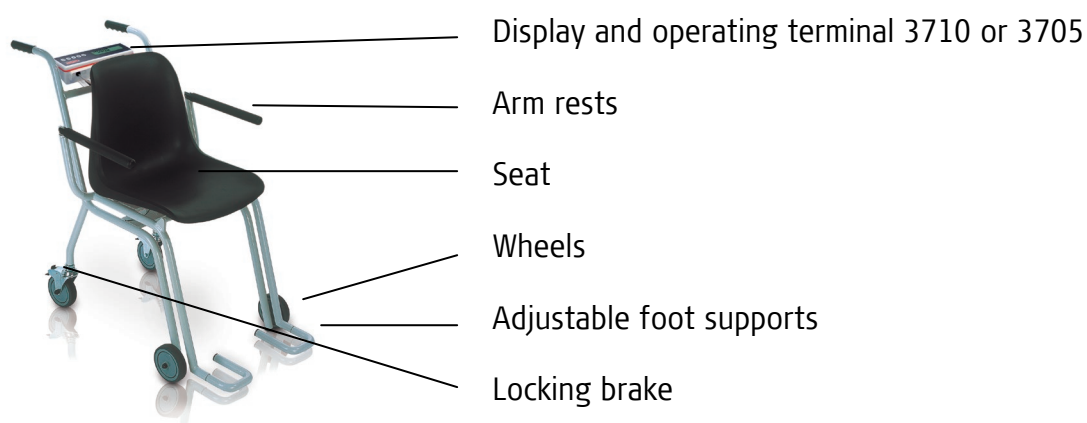
1.1.1 Chair scales, verifiable 7702

The chair scales is designed to weigh people sat down. The maximum weight of the scales is 200 kg.

The intended use of the scales is for the patient to sit on the scales. The personnel, or operator stands behind the scales so that the weight can be read easily.

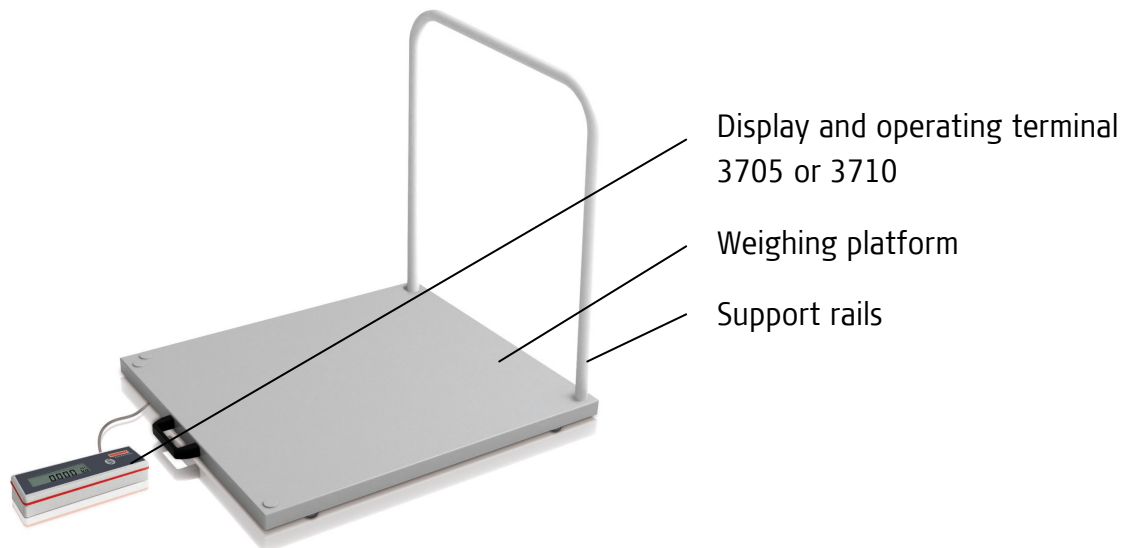1.1.2 Multifunction scale, verifiable 7708

The multifunction scales / invalid chair scales are used for weighting people sat down or standing.

2. FUNCTIONS

2.1 Setting up the scales and start-up

For design reasons, these scales should be used only on level floors. If the set-up location is in Germany, the flatness requirement is met if the floor is within the tolerances for screeded floors defined in DIN 18202, Table 3, Row 4. For other countries, please refer to the relevant national standards. If desired the scales can be adjusted with the help of the supplied underlay plates.

Connect the power adapter to the scales.
Ensure that there is clear access to the power socket.

Before the initial start-up of the scales, the battery must first be charged for 5 hours!

During charging a blue control light flashes in the housing (this control light is found on the side of 3710 and on the back of 3705). As soon as the battery has finished charging the LED stops.

Before using the scales the connections to the arm and foot rests or rails must be checked to ensure they are secure.

The scales should only be used by trained personnel. It is not designed to be used by the patient.

2.2 Operating the display unit Basic E 3705

Turning on

Remove any weights from the scales and press the ON/TARA/OFF button.

After the test routine is completed, the calibration counter appears briefly and then the display switches to zero.

Calibration counter: no fault messages.

The scale is now ready for use.

Weighing

Either stand or sit on the scales.
The display shows the weight.

On chair scales the arms and legs of the patient must lie on the arm and foot rests.

Taring

Place some paper towels or seat covers on the scales and press the ON/TARA/OFF button.

The display shows zero and the letter N for net weight.
Taring is possible over the entire weight range.

Weighing toddlers

An adult steps or stands onto the scales.

› The display could show »67.6 kg« for example

Press the ON/TARA/OFF button.

› The display changes to »0.0 kg«

The adult steps off the scales ...

› Display »-67.6 kg«

...then steps or stands on the scales again holding the child.

› The display now indicates the child's weight, for example »5.1 kg«

Zero setting

Press the ON/TARA/OFF button to correct any minor deviations from the zero point, which may be caused by dirt on the scales.

Zero button: -1 to +3% of the weighing range.

Turning off / Safely end use

Press the ON/TARA/OFF button to switch off the scales.

If the scales are empty and the display indicates 0 weight, the scales can be switched off immediately.

If the scales are loaded, press and hold the ON/TARA/OFF button for 5 seconds.

The equipment switches off when the button is pressed. When the mains plug is removed from the power socket, the equipment is disconnected from all poles of the mains power supply. This ensures that the equipment is powered down.

Note:

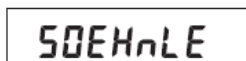
If the display remains at »0.0 kg«, the scales will switch off automatically after a pre-set time. In this case, the display can be activated again by pressing the ON/TARA/OFF button.

2.3 Operating the display unit Standard 3710

Turning on

Remove any weights from the scales and press the ON/OFF button.

After the test routine is completed, the calibration counter appears briefly and then the display switches to zero.

Calibration counter: no fault messages.

The scale is now ready for use.

Weighing

Either stand or sit on the scales.
The weight is displayed on the scales.

On chair scales the arms and legs of the patient must lie on the arm and foot rests.

Taring

Manual taring

Place some paper towels or seat covers on the scales and press the TARA button.

The display shows zero and the letter N for net weight.
Taring is possible over the entire weight range.

Manual tare input

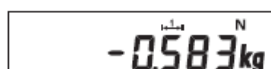
To program the Function key, "Manual Tare Input" must be selected in Setting mode. For Function key settings, see User mode 3710 (see separate description 470.702.099, when needed for using scales or request Soehnle Industrial Solutions).

Set the value using the Tare button or Function key. Press briefly to increase or decrease the value by one. Press and hold to scroll rapidly through the values.

The selected value is stored by pressing the Print button.

Cancel taring

Press the Zero button to cancel taring.

Tare info

Press the Zero button to display the tare information.

If you press the button briefly, the tare weight is deleted, if you press and hold until the weight indication flashes, the tare weight is displayed and not deleted.

Dialysis function (temporary tare function)

The dialysis tare function can be assigned to the Function key.

For Function key settings, see User mode 3710 (see separate description 470.702.099, when needed for using scales or request Soehnle Industrial Solutions).

Unknown tare values are added to the existing tare value without changing the net weight display.

When the Function key is pressed, the word "hold" appears on the display.

Place an additional tare weight or remove the tare weight from the platform.

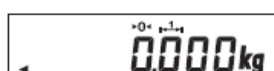
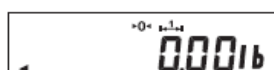
Press the Print button to store the new tare weight.

Switching to another unit of weight

Switching between different units of weight must be enabled in Calibration Mode.

Switching between kg and lb is available in the **non-verifiable** version only!

To program the Function key, "Second unit of weight" must be selected in Setting mode! For Function key settings, see User mode 3710 (see separate description 470.702.099, when needed for using scales or request Soehnle Industrial Solutions).

Switching between units:

Press the Function key to switch to the other unit.

Hold function

The Hold function is activated with the Function key in Setting mode. For Function key settings, see User mode 3710 (see separate description 470.702.099, when needed for using scales or request Soehnle Industrial Solutions).

In weighing mode, the Hold function is activated by pressing the Function key.

In order to freeze the weight data, the following hold functions are available as standard. Default is "0".

For Hold Function settings are carried out in the Setting Mode (see separate description 470.702.099, User mode 3710, when needed for using scales or request Soehnle Industrial Solutions).

Hold mode	Function	To cancel
0	Not active	
1	Hold when stable	On/Off button
2	Hold when stable	Unload the scales
3	Max. value	On/Off button
4	Max. value	Unload the scales
5	Drag pointer	On/Off button
6	Drag pointer	Unload the scales

BMI Function

Load the scales.

Press the F function key. (Default setting of the function key)
 Display shows body size, standard 170 cm (H: 170.)

Changes possible with T or F key.

Confirm entry by pressing Enter.

Display will change between weight and BMI value (bl:26.2 - 82.8kg).

F key returns to the entry of body size.

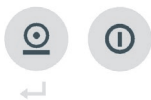
To end the BMI function unload the scales or press the ON/OFF key.

Neutral measurement

The "Neutral measurement" function can be adjusted, e.g for medicine dosage based upon body weight.


To program the Function key, "Neutral measurement" must be selected in Setting mode! For Function key settings, see chapter 1.13 or User mode 3710 (see separate description 470.702.099, when needed for using scales or request Soehnle Industrial Solutions).

The factor for entering the specific weight is the quantity of the measured material relative to the calibrated unit weight (1 kg/1 g/1 lb). Example: Lysis calculation

Entering the conversion factor in Setting mode

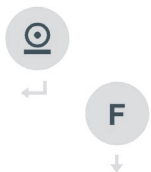Press and hold the Print button then press the On/Off button for 5 seconds until UCAL 1 appears.

Press the Print button to go into UCAL 1.

Using the Function key, scroll back to Position 26.

Use the Print button in the input field to go into the decimal places.

Set the number of decimal places with the Tare button and Function key.

Store with the Print button, the display goes to Position 27.

Use the Print button to go into the input field for the neutral measurement factor.

Using the Tare button and Function key, set a number value in the flashing space.

You can use the Zero button to move by one decade from right to left.

After setting the "Neutral Measurement" factor, store with the Print button, Position 01 is displayed.

Press the Zero button, *UCAL* appears on the display.

To save the setting and exit, press and hold the Print button then briefly press the Zero button. The display returns to weighing mode.

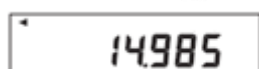
Neutral measurement application

Activate the "Neutral Measurement" function with the Function key.

The display shows *nEUTRAL* for "Neutral Measurement", confirm with the Print button.

The display value appears along with the selected factor without a unit weight. Stabilisation of the scales is represented on the display by a triangle at the top left.

Press the ON/Off button to return to weighing mode.

Weighing toddlers

An adult steps or stands onto the scales.

> The display could show »67.6 kg« for example

Press the TARA key.

> The display changes to »0.0 kg«

The adult steps off the scales ...

> Display »-67.6 kg«

...then steps or stands on the scales again holding the child.

> The display now indicates the child's weight, for example »5.1 kg«

Zero setting

Press the Zero button to correct any minor deviations from the zero point, which may be caused by dirt on the scales.

Verifiable and non-verifiable zero setting range: -1 to +3% of the weighing range.

Turning off / Safely end use

Press the ON/OFF button to switch off the scales.

If the scales are empty and the display indicates 0 weight, the scales can be switched off immediately.

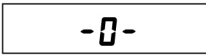
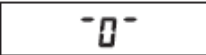
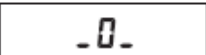
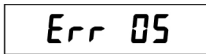
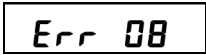
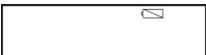
If the scales are loaded, press and hold the ON/OFF button for 5 seconds.

The equipment switches off when the button is pressed. When the mains plug is removed from the power socket, the equipment is disconnected from all poles of the mains power supply. This ensures that the equipment is powered down.

Note:

If the display remains at »0.0 kg«, the scales will switch off automatically after a pre-set time. In this case, the display can be activated again by pressing the ON/OFF button.

3. FAULTS – CAUSES AND REMEDIES

	DISPLAY	REMEDY
	When switched on, the scales are set automatically to zero. If the scales are outside of the designed tolerances the display will show -0-.	Unload the platform. Remove any dirt if present. If the scales do not display a zero after a few seconds, please contact your service partner.
	Display not reset to 0 when switched on.	
		
	Underload: only dashes appear at the bottom of the display field.	Switch off the display. When switched on, zero point is reset automatically.
	Overload: only dashes appear at the top of the display field. Maximum weighing range is exceeded.	Unload the scales, or switch them off and on again. The zero point is reset automatically.
	Zero setting not possible. Weight has gone above or below zero setting limit.	Check the surface under the scales, and check that the weighing platform is level. Test the scales with a force shunt. If the error persists after readjustment, please contact your service partner.
	Taring not possible with the scales unloaded, not yet stabilised, or overloaded.	Sit still, or stabilise the weight on the scales or turn off or on with unloaded scales.
Only on display 3710		
	Printing not possible in the case of underload or overload.	
	kg/lb switching blocked.	
Display 3705 and 3710		
	Battery discharged. About 30 minutes operating time remaining.	Connect the equipment to the power adapter to recharge.
	Battery flat.	Connect the equipment to the power adapter to recharge.

If the fault cannot be cleared, or other error messages appear, please contact your service partner.

4. GENERAL INFORMATION

4.1 Technical data

The weighing range and scale interval are indicated on the data plate on the scales.

Operating temperature:	+ 10 °C to + 40 °C
Storage temperature:	- 20 °C to + 65 °C
Humidity:	20% to 85% relative humidity without condensation
Air pressure:	950 ... 1050 hPa
Battery operation:	7.2 Volt
Mains supply:	100...240 V / 50...60 Hz, 250 mA
Power adapter to be used:	Plug-in power adapter 618.020.059

The technical specifications of the power supply are printed on the power adapter.

On the chair scales the arm and foot rests may be loaded during use with a weight of up to 50 kg in each direction. On the platform scales the railings can be attached as an accessory. These railings must not be loaded laterally.

The technical documents for maintaining the device, which are required for personnel known as service staff by Soehnle Professional, can be obtained from the manufacturer.

4.2 Intended use

This device is intended for measuring the weight of persons, and should be used only for this purpose. Any other type of use is forbidden. These scales are intended for fiscal metering applications. They meet the relevant requirements of EC Directives 2014/30/EC, 2014/35/EC, 2014/31/EC and 93/42/EEC.

Any faults that could put patients at risk, or faults that cause measurement errors, should be reported to the manufacturer's medical product adviser in writing. This user manual is considered an integral part of the equipment. The instructions in the manual must be followed exactly in order to ensure that the equipment is used properly and operated correctly.

Soehnle Professional will accept liability for the safety of the equipment only if the user has paid attention to the information in the user manual and operated the equipment according to the instructions. This product is a medical device and should be used only by persons who have received the appropriate training, and who therefore possess the knowledge required to ensure that the device is handled correctly. Before using this device, the user should check that it is in an acceptable condition and functionally safe. Users and operators must familiarise themselves with the controls. This device is not intended for use in potentially explosive atmospheres in rooms used for medical purposes. Potentially explosive atmospheres can result in areas where there are inflammable anaesthetics, skin cleansers, and skin disinfectants.

Only to be used by trained personnel. It is not designed to be used by the patient.

4.3 Classification

Class I medical device with measurement function.

☐ Electrical protection class II (insulation protected, without earth conductor).

The medical product is a Type B applied part and designed to come into direct contact with the patient. The discharge currents correspond with the classification of Type B applied parts.

EN60529 protection from water ingress: the whole device has IPX2 ingress protection.

4.4 Safety information

This device is intended for professional use only. Please read carefully through the user manual before starting up the equipment. It contains important information concerning the installation, correct use and maintenance of the equipment. The manufacturer will not be liable if the user does not observe the following instructions: When using electrical components with higher safety requirements, you must follow the corresponding regulations. Never carry out work on the device while it is carrying live voltage.

There are no special personnel requirements - the display unit can be used anywhere. This product is not designed for use in critical environments (CT, defibrillators). It is not for use in environments where there is a risk of explosion. The mains-operated display unit must be set up so that it can be easily disconnected from the mains power (ensure that the socket is accessible). The display unit should be protected from moisture.

This device has interference suppression in accordance with the related EC guideline 2014/30/EC. However, the display reading can be affected by extreme electrostatic or electromagnetic interference, for example if a radio or mobile phone is used immediately beside it. If affected in this way, the device should return to normal function once the source of interference is removed. However, it may be necessary to switch off and on again. If there is permanent electrostatic interference, please contact your service partner. This device is a measuring instrument. Drafts, vibrations, rapid temperature changes and direct exposure to sunlight may affect the weight readings. The scales conform to protection class IPX2. Avoid high levels of air humidity, steam, aggressive fluids and heavy soiling.

Improper installation will invalidate the warranty. The electrical connection conditions must be consistent with the rating values printed on the power adapter. The equipment is designed to be used indoors. Note the permitted ambient temperatures for operation (see Technical Data). This equipment meets the requirements for electromagnetic compatibility. Do not exceed the maximum values specified in the standards. If used correctly, there is no danger of other equipment being affected by this product. The service life of this product is

limited to 5 years. If you have any problems, please contact your Soehnle Professional service partner.

Warning: This equipment must not be modified without the manufacturer's permission. If medical equipment is modified, suitable tests and inspections must be performed to ensure that it can still be used safely.

Arm rests and foot rests are not aids for standing.

They may only be used as resting surfaces for arms and feet.

The maximum load is 50 kg.

The product is not to be used as a transport aid for persons or items.

Risk of injury.

4.5 Cleaning

Before cleaning, disconnect the equipment from the power supply by pulling out the mains plug. The equipment should be cleaned only with a moistened cloth. In no circumstances should water be allowed to penetrate the equipment. Disinfectants should be used only on the foil keypad of the display panel.

The following disinfectants are allowed:

Methylated spirits; isopropanol; 2% Kohrsolin; 1% aqueous Sokrena solution; 5% Sagrotan; 5% Gigasept. Do not spray the equipment or the connecting plug.

4.6 Maintenance and service

Metrological inspection was carried out at the initial verification when the product was manufactured. Other regular metrological controls (recalibrations) are to be carried out by the competent weights and measures authorities according to the relevant national regulations. Repairs and replacement of the power cable must be carried out only at a centre authorised by Soehnle Professional, and using original parts.

The device must be undergo preventative inspections regularly (6 monthly intervals). The scales are to be loaded with a known weight and the value displayed compared with the test weight. In the event of any deviations maintenance or calibration must be carried out.

If the equipment is not functioning normally, it is probably because it has been damaged. In this case, it must be returned to a service centre authorised by Soehnle Professional. When repaired by an authorised service centre, only original replacement parts must be used. Original parts are identified by the order numbers indicated in the service documentation.

The function of the installed batteries should be checked periodically. If necessary the battery must be replaced by trained service personnel. A suitable tool is necessary for this.

4.7 Warranty / liability

If the supplied goods contain a defect attributable to Soehnle Professional, Soehnle Professional will be entitled to choose either to rectify the defect or provide a replacement. Replaced parts will remain the property of Soehnle Professional.

If it is not possible to rectify the defect or supply a replacement, then the legal regulations will apply.

The warranty covers a period of **2 years** starting from the date of purchase. Wear parts (e.g. batteries) are covered by a **6 month** warranty. **Please keep your invoice as proof of purchase.** If servicing is required, please contact your dealer or Soehnle Professional Customer Service.

No liability can be accepted especially for damage resulting from any of the following:

Unsuitable, improper storage or utilisation, faulty assembly or start-up by the purchaser or by a third party, natural wear and tear, modifications or tampering, faulty or negligent handling, particularly excessive loads, chemical, electrochemical, electrical influences or dampness, insofar as these cannot be attributed to Soehnle Professional. The warranty guaranteeing overall correct function of the equipment will become void if operating, climatic, or other influences cause a substantial change in the characteristics or the state of the materials. If Soehnle Professional provides an individual warranty, this means that the equipment supplied will be free of defects for the duration of the warranty period.

Please keep the original packaging in case a return shipment is necessary.

4.8 Disposal of batteries

Batteries that contain hazardous substances are identified by a **symbol showing a crossed-out waste bin and should not be disposed of as normal household waste.**

As a consumer, you are required by law to recycle used batteries. Your old batteries can be handed in as special waste at the public recycling points in your community, or anywhere that batteries of a similar kind are sold. These symbols are found on batteries containing hazardous substances: Pb = battery contains lead, Cd = battery contains cadmium, Hg = battery contains mercury.

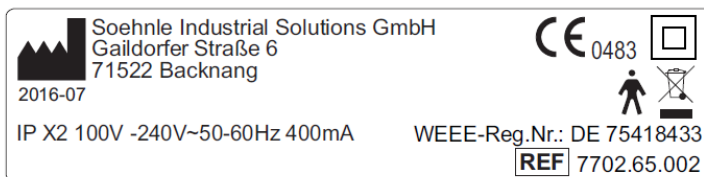
4.9 Disposing of the scales

According to current state of knowledge, this equipment does not contain any substances that are particularly harmful to the environment. It should not be treated as normal waste, but must be taken to a collection point where electric and electronic devices are recycled. You can obtain more information from your municipal authorities, local waste disposal services, or the company from which this product was purchased.

5. OFFICIAL VERIFICATION

5.1 CE mark


This product carries the CE mark
in accordance with the following directives:
EMC directive: 2014/30/EC
Low voltage directive: 2014/35/EC
Weighing instruments directive: 2014/31/EC
Medical directive: 93/42/EEC

Explanation of symbols:

- CE conformity mark
- Protection class of product
- Product item number
- Manufacturer of the product

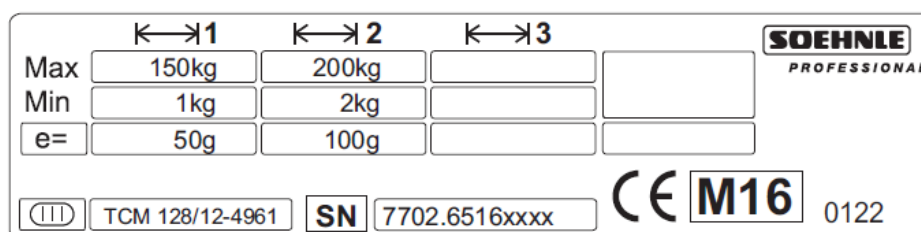5.2 Explanation of verification label and symbols

Variants:

Invalid chair scales 7702: Max: 150 kg / 200 kg, Min 1 kg / 2 kg, e = 50 g / 100 g
Multifunction scales 7708: Max: 300 kg, Min 2 kg, e = 100 g

Explanation of symbols:

	Active weighing range	7702	Manufacturer's product number
Max	Maximum load for weighing range	SN	Serial number of scales (scales type, final digits of year of manufacture, consecutive number)
Min	Minimum load for weighing range	0122	Official number of the notified body (which carried out the initial verification)
e =	Calibration value (scale interval)	M16	CE verification mark
	CE conformity mark with year of application, verification valid from the same time.		
	Accuracy class		

5.3 Explanation of symbols on packaging

	Caution! Fragile
	This side up during transport
	Protect from moisture and damp
	Ensure correct storage temperature

6. WARNINGS

- › In the event of a malfunction, please contact your dealer or the manufacturer.
Unauthorised modifications or repairs can damage your scales and will cause the manufacturer's warranty to become void.
- › Display units should be protected from moisture. Liquids (e.g. water) can damage the display unit. Use a dry cloth, e.g. a towel, to wipe the display.
- › Disconnect the power supply before installation, cleaning or maintenance of the equipment . Otherwise, the equipment could be damaged.
- › Do not expose the display to direct sunlight. This could cause colour changes.
- › If you are not using the display for long periods, disconnect the power supply, or switch off the display if it is operating on battery power. Recharge the batteries after 4 weeks at the latest, to compensate for self-discharge.
- › Avoid stacking any materials on top of the display, or placing any weights on it. This can damage the display.
- › Place the scales on a solid, stable and level surface: this will ensure that measurements are accurate. Accuracy is affected if the scales are tilted or placed on a carpeted floor.
- › Do not connect the scales to an unstable power source.
- › Use only the original equipment. Use of other brands can cause damage to the display.
- › Arm rests and foot rests are not aids for standing. Only to be used when wheels are secure.
- › Contact between the interface and the patient at the same time is not allowed.

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Soehnle Industrial Solutions GmbH