**ekuore**  Pro

User manual

<b>Español</b>	03
<b>English</b>	17
<b>Français</b>	31
<b>Deutsch</b>	45
<b>Italiano</b>	59
<b>Português</b>	73
<b>Polski</b>	87
<b>Română</b>	101
<b>Suomi</b>	115
<b>Svenska</b>	129

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## 1. INTRODUCTION

eKuore Pro® Series is an electronic stethoscope. This user manual covers the models eKuore Pro® and eKuore Pro® Amplified. It's been designed for medical use. It can be used for detection and amplification of sound heart lungs, arteries, veins and other internal with the use of selective frequency ranges organs. It can be used on any person undergoing a physical examination.

This manual provides comprehensive information on how to handle eKuore Pro® devices, so additional training on the operation is not required. eKuore Pro® Series is a family of Medical Equipment devices internally powered by a rechargeable battery that is intended to be reusable and multi patient.

## 2. SYMBOLOGY AND SAFETY INSTRUCTIONS

### CAUTION

Non-compliance may result in moderately serious injury or damage.

**LLXXXXXX** Label serial number (back side of the device)

LL= Lot number

XXXXXX=physical address of the communication module (unequivocal)

Manufacturer's Name and Address

Reference

Conforms to the guidelines 93/42/CEE established by organization n° 1639

Acceptable Temperature Range for storage and transportation

Humidity limit (operational)

Federal (USA) law restricts this device to sale to or on the order of a clinician.

### ATTENTION

Give tips and useful information.

This product contains an intentional RF radiator certified by the FCC

The device cannot be disposed of in the rubbish. It must be disposed of the appropriate way

Serial number

Applied part B Type

Attention: Read the user manual

MR un-safe!

**CAUTION**

- Do not use eKuore Pro® Series while charging the battery
- Do not use the product if present any damage
- Do not use the device over open wound or on infected skin
- Ensure that the chestpiece is tightly curled as it can interfere with the operation of eKuore Pro® Series
- Ensure that the chest piece is screwed on carefully and tightly so as not to damage the membrane or interfere with the device operation.
- Clean the stethoscope from patient to patient.
- In case of electrostatic discharges, the mobile phone/tablet may behave abnormally. In this case, restart the phone/tablet.
- Follow the operating instructions, since misuse can cause leakage currents.
- Do not use the eKuore Pro Series in environments with high interferences or electromagnetism.
- Check the product box before use, verify that it contains all necessary components.
- eKuore Pro Series requires for EMC precautions to be incorporated.
- eKuore Pro Series can be affected by radiofrequency communications.
- The use of accessories other than those indicated by eKuore may result in increased emissions or decreased immunity of the eKuore Pro Series.
- The eKuore Pro Series is powered by an internal battery for continuous operation.
- Do not use the product if it does not work properly.
- The eKuore Pro Series has no defibrillation protection in place.
- The equipment may be subject to interferences due to electrosurgery or high-frequency electrosurgery
- MR-unsafe! Do not expose the device to a magnetic resonance (MR) environment. The device may present a risk of projectile injury due to the presence of ferromagnetic materials that can be attracted by the MR magnet core. Thermal injury and burns may occur due to the metal components of the device that can heat during MR scanning. The device may generate artifacts in the MR image. The device may not function properly due to the strong magnetic and radiofrequency fields generated by the MR scanner.

**i** ATTENTION

- There are not materials or substances that may pose an unacceptable risk to the user or patient.
- In case of fault: Contact technical service.
- We recommend checking regularly, every six months, the proper functioning of the battery.
- It is recommended that you fully charge eKuore Pro® Series battery for the first time
- Use only USB chargers in compliance with IEC 60950 standard. (5 V DC and 0.5 A)
- During charging of the battery the temperature may increase.
- If the device will not be used in an undetermined time, ensure the battery is charged up 30 to 50%
- Membrane and ring of eKuore Pro® Series are an applied B type part.
- Do not immerse eKuore Pro® Series in any liquid or subject it to any sterilization process
- Sound only could be played through external headphones, speakers, or wireless audio transmitter, not through the smartphone's speaker.
- It is highly recommended that users of the mobile App use device and networking security features to protect patient data created and stored using this software, in addition to security features embedded in the system.

**2.1 Patient Privacy**

Patient privacy may be protected by state, federal or international laws governing how the information may be used, stored, transmitted and disclosed. Third-parties access may be prohibited without the patient's prior authorisation.

The user is responsible for understanding and complying with all laws governing the storage, transmission, and disclosure of any electronic patient data through the use of software. If the user is unable to comply with applicable laws or restriction on the use and disclosure of this data, the user should not collect or store such information.

eKuore Pro Series' application allows the user to modify the name of the stored recording, so they can include identifiable health information. In this case, the user assumes any risk and responsibility incurred by the transmission of such information.

### 3. PACKAGE CONTENTS

eKuore Pro Series includes:

- An eKuore Pro® / eKuore Pro® Amplified system (ref EP0002/EP0099)
- Earbuds (EP1202)
- USB cable (ref EP1500)
- Quick guide (ref EP0015)
- Chestpiece eKuore Pro® Size L (ref EP1002)

### 4. INDICATION FOR USE

The eKuore Pro Series is intended to be used as a part of a physical assessment of a patient by healthcare professionals for diagnostic decision support in clinical settings. eKuore Pro Series is intended for use on pediatric and adult patients. It can electronically filter and transfer sounds to the accompanying mobile software application.

It can be used to record heart sounds and cardiac murmurs, bruits, respiratory sounds and abdominal sounds during physical examination in normal patients or those with suspected diseases of the cardiac, vascular, pulmonary or abdominal organ systems.

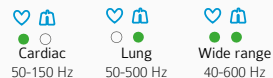
## 5. CONFIGURATION

### MODE / FILTER / RECORDING BUTTON

#### Short press (1 click):

Alternate between the different filters



#### Intermediate press: Start recording

(eKuore Pro have to be connected to the APP)

**Long press:** Change mode (Wifi / No-wifi)
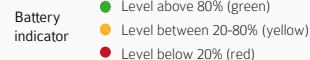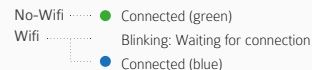
#### Devices status:


### 5.1 First Steps

Before starting using eKuore Pro remember to charge it for at least 2 hours.


1. Tighten the chestpiece into the top threaded hole.
2. Connect the earbuds (ref. EP1202) on the audio output.
3. Switch on the eKuore Pro clicking ON/OFF button.

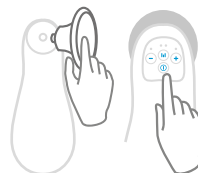**i** Sound can only be played through external headphones, speakers, or wireless audio transmitter, not through the smartphone's speaker.

## 5.2 Use without Smartphone

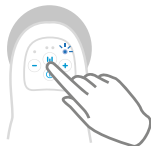1. Follow the first steps explained before.
2. Make a long press on the 'mode button' until the 'device status' LED turned into green (No-Wifi mode).


## 5.3 Adjust the sound

### HEADPHONES

1. Connect the earbuds to the device audio output
2. Adjust the volume pressing – and + buttons

### SPEAKER

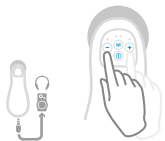1. Reduce the volume to the minimum. If you can control bass sounds, place it between the middle and maximum.
2. Place the speaker at a distance of at least 1 metre to avoid background noise.
3. Tap the chestpiece membrane to test the sound
4. In case of not sound enough, turn slightly the volume up
5. Repeat steps 4 and 5 until the sound was optimal.

**Note:** The device has 5 volume levels and always starts at minimum. Sound only could be listened through the headphones/earbuds or external speaker, smartphone/tablet speaker is not permitted.

## 5.4 Use eKuore Pro Series with the APP

### 5.4.1 Download the APP

#### ANDROID

1. Open "Google Play Store"
2. Search for "eKuore Pro APP"
3. Select install and follow the instructions until the installation has finished
4. Open the APP

#### iOS

1. Open "App Store"
2. Search for "eKuore Pro"
3. Select install and follow the instructions until the installation has finished
4. Open the APP

**Note:** Install the APP and its updates from oficial sources.

### 5.4.2 Use of the APP

#### ANDROID

1. Open the application if you have not open it before and follow the steps explained at the APP
2. Press the button "Select"
3. On the wifi networks menu, tap the eKuore Pro network which correspond the SN number of your eKuore Pro device, it is placed at the rear.
4. Once you have selected the wifi net, go back to eKuore Pro APP.
5. Click on "Ready"

#### iOS

1. Go to wifi settings on your smartphone.
2. Select the wireless network which match the last 5 characters of your eKuore Pro serial number.
3. Open eKuore Pro APP
4. Click on 'Go to Monitor'
5. When the connection is established, the blue LED will stay fixed (Wifi mode).

### 5.5 Make a recording

1. Open the APP eKuore Pro
2. Connect the device
3. Tap the red button on the screen or press 'mode button' intermediately
4. To stop the recording tap again on the recording APP button or press stethoscope 'mode button' intermediately

## 6. MAINTENANCE AND WARRANTY

### 6.1 Environmental conditions

To extend eKuore Pro® Series' product lifetime, avoid storage and transportation temperatures outside the range of -20 to 45 °C, and humidity levels between 15 and 93%; and operation temperatures outside the range of 0 to +40 °C, and humidity levels between 15 and 93%. As well as solvents and oils. Failure to follow the recommendations for use and maintenance could result in damage to the internal components of the stethoscope.

### 6.2 Cleaning

Turn off the device and be sure is not charging, the entire plastic surface can be cleaned sliding an 70% isopropyl alcohol wipes. Excess fluid during cleaning can cause leakage of moisture on internal components.

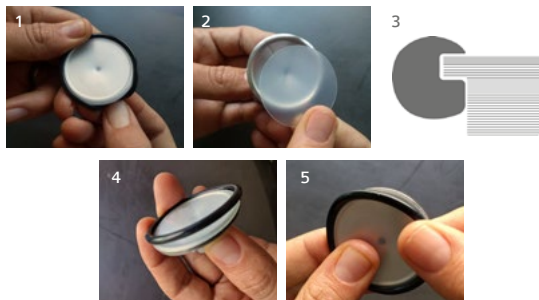
Usually, it is not necessary to remove the membrane for cleaning. If necessary, thoroughly follow the instructions in section 6.3

### 6.3 Replacing the membrane and ring

The replacement of the membrane should be done when the user has appreciated that it has lost quality, either by use or by damage thereof. The steps for this replacement are as follows:

1. Using your thumb pressure to remove the ring and the corresponding membrane. With no membrane, then clean the chestpiece of the stethoscope
2. Align the new membrane with the chestpiece.
3. Check that the position of the ring corresponds to the graph on the right, i.e., the thickest part should look inward and the finest one in contact with the patient
4. Insert the ring laterally and guide with the fingers to make it enter completely.
5. Press with your thumb to finish locating it carefully not to bend and damage the membrane.
6. Check that the ring fixes completely the membrane and there are no exposed parts.

### 6.4 Technical service

Modifications to the product are not permitted. It must be repaired by authorised personnel. For repairs, send equipment to:

**eKuore Technical Service**  
 C/ Valencia 45 B. 46100  
 Burjasot, Valencia. España

Outside Spain, for information on the maintenance and repair of eKuore Pro® Series, contact an authorized distributor or send an email to:

[support@ekuore.com](mailto:support@ekuore.com)

### 6.5 Replaceable System Parts

eKuore Pro Series' replaceable parts are:

- S/M/L Chestpiece (EP1000/EP1001/EP1002)
- S/M/L Chestpiece membrane (EK1501/EK1503/EK1505)
- S/M/L Chestpiece ring (EK1502/EK1504/EK1506)
- USB cable (EP1500)
- Earbuds (EP1202)

For more information, visit [www.ekuore.com/shop/](http://www.ekuore.com/shop/)

## 6.6 Support and warranty

eKuore offers its users the following support channels:

Website: <https://www.ekuore.com/support/>

Email: [support@ekuore.com](mailto:support@ekuore.com)

Telephone: +34 961 15 67 38

For any support communication, please indicate the serial number of your device. Any adverse events or damage to the device should also be notified.

For further details on the warranty, visit [www.ekuore.com/warranty/](http://www.ekuore.com/warranty/)

## 6.7 Troubleshooting guide

PROBLEM	POSSIBLE CAUSE	SOLUTION
The device does not turn on	Prolonged use of the battery.	Connect the equipment to the network After 10 hours if the problem persists, call the technical service.
The result is abnormal	The signs are vague. This may be due to improper use in positioning the device.	Read the instructions that tell you how to perform measurements correctly.
The blue LED keeps flashing	The device is searching for a connection which is available.	1. Make sure the WiFi connection is activated on your mobile or tablet. 2. Follow the tutorial of the Mobile App.
You cannot hear lungs / heart clearly	You have not selected the appropriate filter.	Read the instructions that tell you how to choose the most suitable for the type of filter you want to auscultate.
The speakers sound distorts.	1. The speaker volume is too high. 2. It is too close to the speakers.	1. Follow the instructions given in the instruction manual to handle eKuore Pro® Series with speakers. 2. Keep a distance of at least 1 meter away between eKuore Pro® Series and the speakers
Wireless transmission lost/ coexistence issues	1. App or smartphone not working 2. High separation distance	1. Follow the tutorial of the Mobile App 2. Recommended separation distance with stethoscope and smartphone is between 20cm to 3m.

## 7. EMC COMPLIANCE AND ELECTRICAL SAFETY

### 7.1 EMC Compliance

#### EMC Compliance Europe

The eKuore Pro® Series may be affected by the use of certain devices such as electrosurgery, diathermy and electromagnetic equipment; defibrillators; and mobile phones whose interference exceeds the limits set forth in standard EN60601-1-2.

### 7.2 Electromagnetic Emissions and Immunity

Emissions test	Compliance	Electromagnetic Environment - Guide
RF emissions CISPR 11	Group 1	The eKuore Pro® Series only utilises RF energy for its internal function. Therefore, their Rf emissions are very low and are not likely to cause any interference to nearby electronic equipment
RF emissions CISPR 11	Class B	The eKuore Pro® Series is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network that powers buildings used for domestic purposes.
Harmonic emissions IEC 6100-3-2	Non applicable	
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	Non applicable	

### 7.3 Using the device

The use of accessories that have not been supplied by us may result in non-compliance with the safety levels of the eKuore Pro® Series, reducing the safety level, increasing the level of emissions, and reducing immunity. The manufacturer has foreseen most of the possible faults of the device, which have been included in the internal monitoring system. However, if the patient depends on this system, it is recommended to have an alternative system. The user must read and understand this manual before using any device in the eKuore Pro® Series. This manual has been written to provide the necessary information for the use of any device in the eKuore Pro® Series, but should not be considered a substitute for medical instructions essential for the adjustment of the eKuore Pro® Series to adapt it to the needs of the patient.

### 7.4 Power supply

Since the device has an internal battery, it is recommended to charge it regularly. If it is not used frequently, charge the battery to 50% before storing to keep it in good condition. No more than six months should elapse between charges.

The energy supply of the stethoscope eKuore Pro® Series is provided a lithium polymer rechargeable battery, which is charged using a USB to USB cable connected to a battery charger with the electrical output characteristics of 5 V DC and 0.5 A in compliance with IEC 60950 standard by a microUSB connector. **5V** 

## 8. TECHNICAL SPECIFICATIONS

Weight without chestpiece	85 gr
Dimensions without chestpiece	13cm x 5 cm x 3 cm (WxDxH)
Weight with chestpiece	150 gr
Dimensions with chestpiece	13cm x 5 cm x 3 cm (WxDxH)
Input voltage during the load	5V DC, 500 mA
Power	2,5 W
Battery	Rechargeable, mAh lithium polymer. Do not replace.
USB Port	Do not connect while the device is been use on the patient
Temperature	0 to +40° C
Humidity	+15 to +93%
Transportation and storage temperature	-20 to +45 °C
Transportation and storage humidity	+15 to +93%
Transmission frequency range	Between 2.412 GHz y 2.484 GHz
Modulation type	DSSS
Wireless standard protocol	IEEE 802.11b/g
Effective radiated power	15dBm (32mW)
Operating system requirements	Android 6.0 or later, and with iOS 10.0 or later devices.
Wireless quality of service	Throughput provided by the WiFi connection must be enough to such an audio bitrate. 705,6 Kbps
Wireless security	The device creates an encrypted private wlan
Wireless functions	Audio transmission

## Guidance and Manufacturer's Declaration - Electromagnetic Immunity

Immunity Test IEC	IEC 60601 Test Level	Compliance level	Electromagnetic environment – Indications
Discharging electrostatic electricity IEC 61000-4-2	Contact: $\pm 15$ kV Air: $\pm 8$ kV	Complies	The floor must be Wood, cementor ceramic coating. If the floor is covered by a synthetic material, the relative humidity must be less than 30%.
Fast electrical transients IEC 61000-4-4	$\pm 2$ kV on lines of mains supply $\pm 1$ kV on input /output lines	Not applicable Not applicable	The quality of the power should be comparable to that a commercial establishment or hospital.
Shock waves IEC 61000-4-5	$\pm 1$ kV on line $\pm 2$ kV on line and mass	Not applicable Not applicable	The quality of the power should be comparable to that a commercial establishment or hospital
Short interruptions and. Voltage dips IEC 61000-4-11	0% Unom during 250/300 cycles 0% Unom during 0.5 cycles 0% Unom during 01 cycle 70% Unom during 25/30 cycles	Complies	The quality of the power should be comparable to that a commercial establishment or hospital.
Magnetic field to network frequency (50/60 Hz) IEC 61000-4-8	30 A/m	Complies	The magnetic fields to network frequencies should be of the same level of characteristics to that of a commercial establishment or hospital
Proximity fields from wireless transmitters IEC 61000-4-3	80MHz to 1GHz and 1.4GHz to 2.7GHz. at 10V/m (710, 745, 780, 5240, 5500, 5785) MHz. at 9V/m; (385,450, 810, 870,930, 1720, 1845, 1970, 2450) MHz. at 28V/m	Complies	The quality of the power should be comparable to that a commercial establishment or hospital
RF conducted IEC 61000-4-6	3 Vrms 150 kHz a 80 MHz	Not applicable	Portable radio communication must not be used in the vicinity of eKuore Pro® Series, including cables, unless it is of the recommended calculated distance applicable to the transmitter's frequency. Separation distance recommended: $d = [3,5\sqrt{P}] \sqrt{P}$ $d = [3,5/E1] \sqrt{P}$ 80 MHz a 800 MHz $d = [7/E1] \sqrt{P}$ 800 MHz a 2,5 GHz Where P is the transmitter's maximum power in Watts (W), according to the transmitter's manufacturer, and is recommended separation distance in meters(m). It is appropriate that the fields of intensity are fixed RF transmitters, determined by an electromagnetic investigation in sitea, are lower than the levels of conformity, in every frequency rangeb. Interferences may take place in proximity to devices equipped with the following symbol: 
RF irradiate IEC 61000-4-3	10 v/m 80 MHz a 2,7 GHz	Complies	

**NOTE 1:** At 80 MHz and at 800 MHz, the highest frequency range is applicable.

**NOTE 2:** These guidelines are not necessary applicable in all situations. The electromagnetic propagation is affected by the absorption and reflections of structures, objects and people.

The intensity of the fixed transmitters, i.e. the base stations for radiophones (cellular/Wireless) and mobile radios, amateur radio, AM and FM radio, TV, cannot be theoretically foreseen with precision. To evaluate the electromagnetic environment due to fixed transmitters, it is possible to consider an electromagnetic investigation on site. If the field's intensity, measured at the location where eKuore Pro® / eKuore Pro® amplified is used, exceeds the level of conformity for RF below, it is advised to verify that eKuore Pro® / eKuore Pro® amplified is working properly. If abnormal performance is observed, supplementary measures could be necessary, in order to re-orientate or reposition eKuore Pro® / eKuore Pro® amplified. b On the frequency range 150 kHz to 80 MHz, it is advised that the field's intensity is less than 3 V/m

### Recommended Separation Distances between mobile radio equipment and E KUORE PRO®

eKuore Pro® Series must be used in an electromagnetic environment where the radio interferences are controlled. The user can limit the electromagnetic interferences by maintaining a minimum distance between the mobile radio equipment (transmitter) and eKuore Pro® / eKuore Pro® amplified as recommended below, according to the maximum power of the radio equipment.

Transmitter's maximum power output (W)	Separation distances according to the transmitter's frequency in meters (m)		
	150 kHz a 80 MHz $d = [(3,5)/V1] \sqrt{P}$	80 MHz a 800 MHz $d = [(3,5)/E1] \sqrt{P}$	800 MHz a 1 GHz $d = [7/E1] \sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

The device has no essential performance.

The device may be affected your Wi-Fi connection to radiated electromagnetic fields of frequencies around 2400 MHz and filter change sound when you apply electrostatic discharge, without affecting all the basic safety equipment.

**NOTE 1:** At 80 MHz and at 800 MHz, the highest frequency range is applicable.

**NOTE 2:** These guidelines are not necessary applicable in all situations. The electromagnetic propagation is affected by the absorption and reflections of structures, objects and people.

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