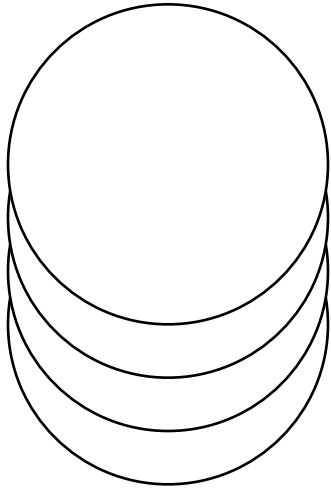
# Cøpper

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passend für:

Littmann® Classic III™  
Littmann® Cardiology IV™

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 Artikelnummer

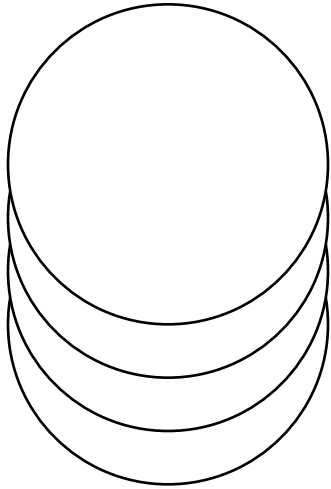
 Charge

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 Mein Held  
[doccheck.com](https://www.doccheck.com)

# Copper

## User Manual

suitable for:

**Littmann® Classic III™**

**Littmann® Cardiology IV™**

### My intended use

I, Cøpper, am an effective antimicrobial copper membrane that you can insert into your stethoscope instead of conventional membranes. As soon as you have attached me to the chest piece of the stethoscope with the corresponding fastening ring, you can use it as usual.

Only use me with the stethoscope models listed on the packaging.

### Effectiveness

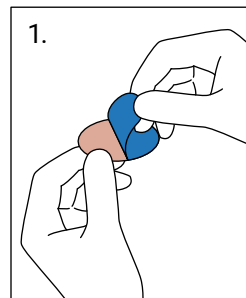
Studies show that I can reduce bacteria and microbial activity. This works through the reaction between my copper surface and the microbes as soon as they touch me.

I reduce the following bacterial strains by 99.9 % within 24 hours:


- MRSA
- E. coli
- Staphylococcus aureus

However, I am of course not a substitute for responsible, hygienic treatment of your patients!


### Assembly

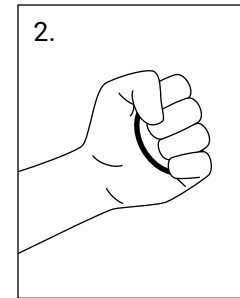
1. Take me out of the packaging and carefully remove the protective film. Please don't bend me. I am very thin and my edges can be sharp. Therefore don't hold me by the outside edge.

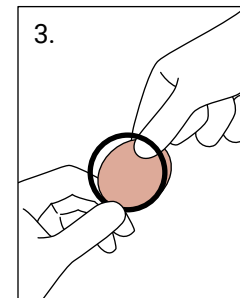
2. Take a matching ring and press it a few times in the palm of your hand. The movement and heat make the ring more flexible and assembly is easier.

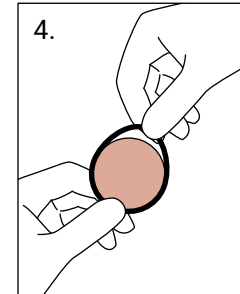3. Now put me about a quarter of the way into the ring. Don't try to insert me completely into the ring straight away. That could damage my sensitive body.

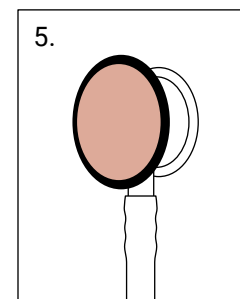4. Now hold me between your thumb and forefinger and gradually pull me into the ring, always pulling up a small piece of the ring and letting me snap into place. Repeat this all around until I am completely in the ring.

5. Now you can mount me on the chest piece of the stethoscope as usual. Make sure my best side (the shiny side) is facing up and that this side is making contact with the patient. If you've attached me correctly, my surface should look smooth and stretched, similar to a drum.

Please test my functionality again after each replacement.

Due to my materials, my acoustic profile may differ from the original membrane supplied with your stethoscope. Please ensure you familiarise yourself with this profile before using me for diagnostic purposes.

### Safety information

I'm 99.9 % copper. A small proportion of the population may be allergic to this material. Do not use me if you suspect that you or your patient has such an allergy.

Be careful not to damage my surface with sharp objects. That can shorten my lifespan.

I am electrically conductive.

Any use other than that described in these instructions for use contradicts the intended purpose. My manufacturer assumes no liability for damage of any kind caused by improper use.

### Cleaning & maintenance

I can tolerate ISO alcohol just like conventional stethoscopes. Clean my surface with a smooth cloth, dampened with ISO alcohol, to remove dust and dirt.

After a while, my surface colour could change, which is caused by a reaction with oxygen. My beautiful copper sheen then changes to a darker brown. This is a clear signal to replace me.

My manufacturer recommends replacement after 30 days of normal use.

### Disposal

Please only dispose of me in accordance with local / regional / international guidelines.

### Manufacturer

DocCheck Shop GmbH  
Vogelsanger Str. 66, 50823 Cologne, Germany  
[doccheckshop.com](http://doccheckshop.com)

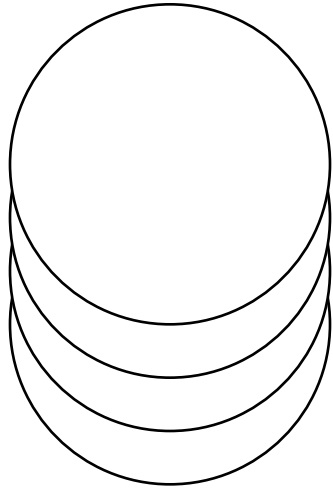
Littmann®, Classic III™ and Cardiology IV™ are registered trademarks of 3M™ Company, St. Paul, Minnesota, USA and are in no way affiliated with DocCheck Shop GmbH.

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