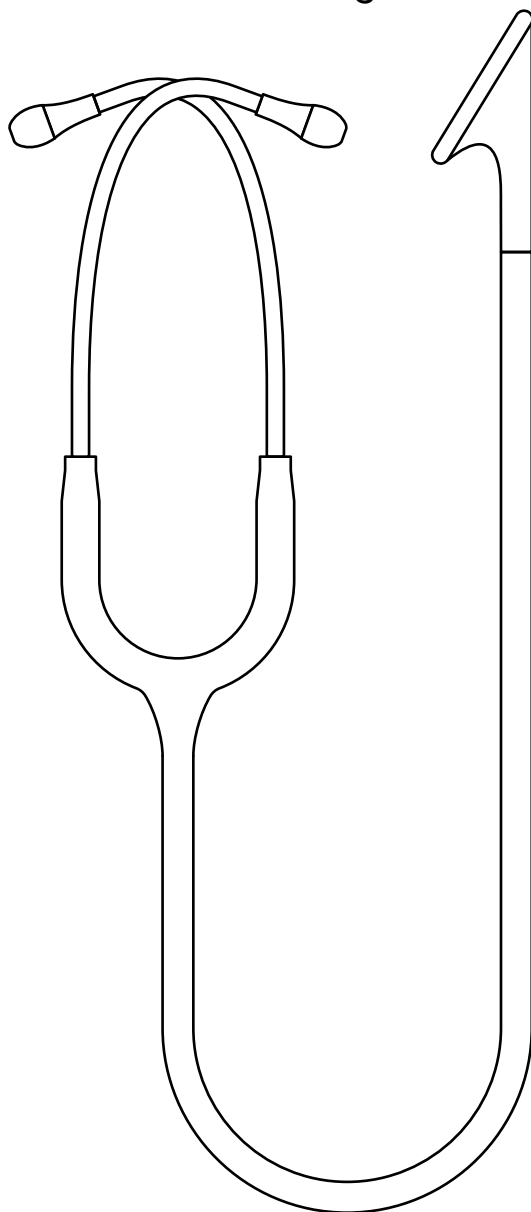
Tülip

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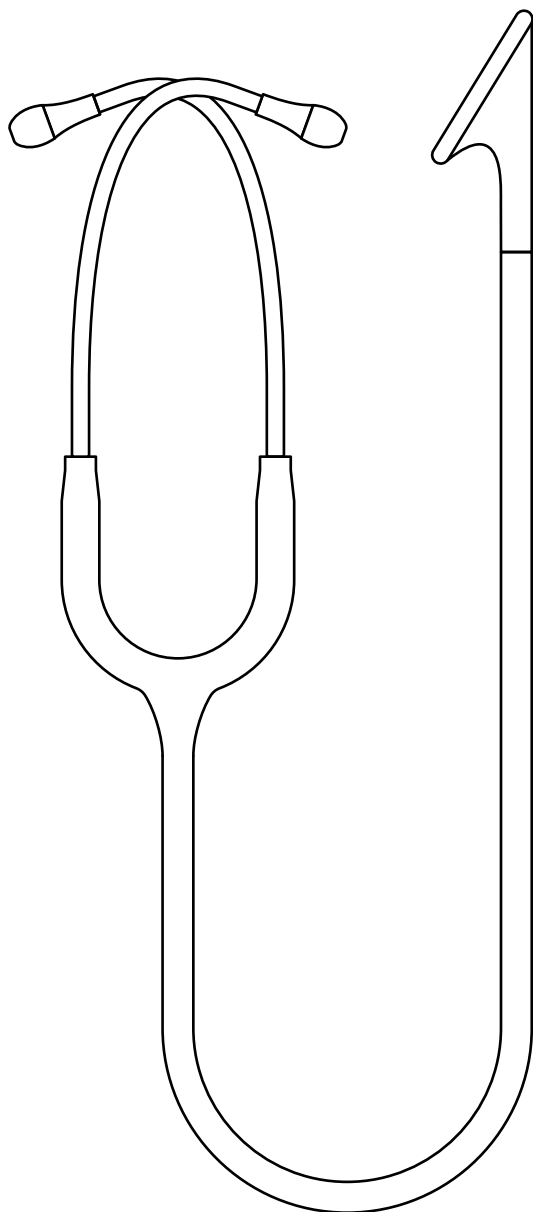
Mein Held
[doccheck.com](https://www.doccheck.com)

Tūlip

User Manual

My intended use

I, Tūlip, may only be used by authorised and qualified professionals.

I am an acoustic diagnostic tool and help you to hear body sounds better, especially those of the heart and lungs.

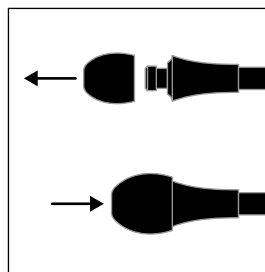
Read the following instructions closely and we will be friends for a long time.

Inappropriate use

Inappropriate use means you use me differently to what this user manual states. My manufacturer is not responsible for damage caused by inappropriate use. In this case only you, the user, will be responsible. So please be careful!

Operation

In order to use me correctly, put my ear tips into your ears first. You will know that the ear tips are properly placed when they point towards the outer ear canal. If this isn't the case, test which size suits you best to ensure optimal noise cancellation.

The ear tips come in three different sizes (S, M & L). To switch these, simply remove the previous ear tips and slide the new ones onto the adapter's end.

You can adjust my ear tubes' tension by pulling them apart or squeezing them together. Pulling apart the ear tubes is particularly helpful if you feel that the ear tips are sitting too tight in your ear canal.

Check my membrane by gently rubbing your finger over it before applying it to the patient. Please wear gloves to avoid transferring bacteria to the membrane.

If you can't hear any noise, make sure my components are assembled correctly. I'm ready to go.

For auscultation, place my chest piece on the body area you want to examine. Apply gentle pressure to the chest piece to hear low frequencies. Increase the pressure to listen to all frequencies at a higher volume. My chest piece should be used directly on the skin. This is the only way to guarantee that rustling does not distort the quality of auscultation.

Information about Tūlip Clean Edition

In the Clean Edition, I come with copper membranes. Studies show that this enables me to significantly reduce bacteria and microbial activity.

However, I do not replace responsible hygienic practices when dealing with your patients.

The acoustic profile of my special membrane differs from that of conventional membranes. Please make sure you familiarise yourself with the profile before using me for diagnostic purposes.

A small portion of the population may be allergic to copper. Do not use me if you suspect that you or your patient has such an allergy.

My membrane is electrically conductive.

Information about Tūlip HiFi Edition

If you are the happy owner of the HiFi edition, please note the following:

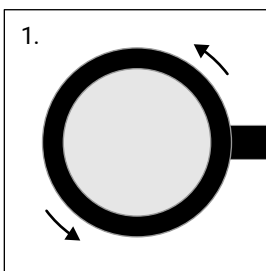
I have a highly sensitive aluminum membrane and can especially hear body sounds ranging from 100 to 400 Hz, making me perfect for auscultating heart sounds.

The acoustic profile of my special membrane differs from that of conventional membranes. Please make sure you familiarise yourself with the profile before using me for diagnostic purposes.


A small portion of the population may be allergic to aluminum. Do not use me if you suspect that you or your patient has such an allergy.

Due to its special processing, my membrane is very delicate and should be handled with great care. Additionally, it is electrically conductive.


Changing the membrane

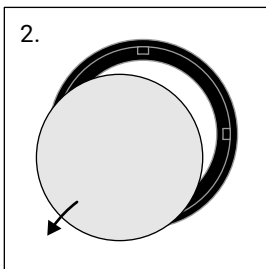
1. Release the ring by turning it counterclockwise.

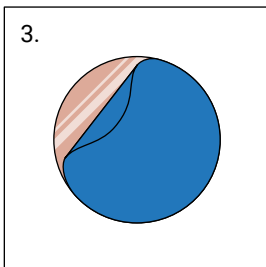
2. Remove the old membrane from the ring.

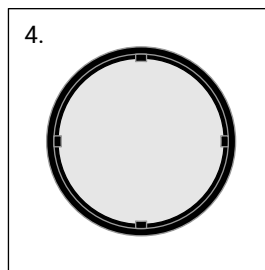3. **For Tülip Clean Edition:** Carefully remove the blue protective film from the copper membrane.

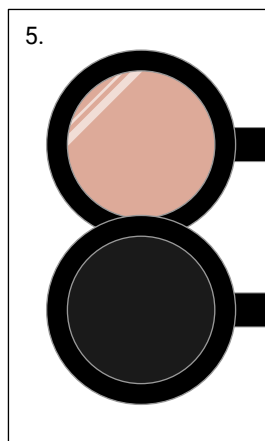4. Place a new membrane into the ring so that it lays flat.


5. Make sure the correct side of the membrane faces outward:

For the Clean Edition:

The brighter, shiny side.

For the HiFi Edition:

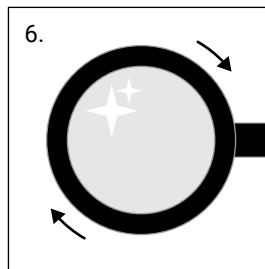The completely black side without visible laser engraving.

6. Tighten the ring and membrane onto the chest piece so that they fit tightly.

Do not be afraid that the ring may break when tightening. If it's not tight enough, there's a possibility that it will come off the chest piece.

Please test my functionality again after each membrane change.

Safety notice

Please do not use me without membrane or ear tips. Never put my ear tips too deep into your ear canal or use me near loud sound sources. This can lead to hearing damage!

I am not a children's toy. I contain small, swallowable parts (ear tips), so please make sure to keep me out of children's reach.

Note to the user and/or the patient:

All serious incidents related to me must be reported to DocCheck Shop GmbH and the competent authority of the Member State.

Materials

I look so elegant because my chest piece is made of aluminium. My fastening ring looks the part, too, as it is made of nylon. The ear tips are made of silicone.

My tube is made of PVC. This material becomes hard when it comes into contact with skin for a long time. Therefore, please avoid contact with the skin as much as possible.

Cleaning, disinfection and storage

Clean and disinfect me regularly. This is of great importance – it serves both to protect the patient, the user and third parties, but also to maintain my quality.

Manual cleaning

Please cleanse me, especially my chest piece and membrane, after each patient. It's really easy: Use a cleaning cloth with 70% isopropyl alcohol or soapy water. Keep me away from other liquids or solvents.

I also don't like being dipped in liquids, so please refrain from this.

When cleaning, don't forget to pay particular attention to my tube. Care for it with special plastic care products so that it does not harden over time.

For a thorough cleaning, please remove my individual parts. Before you reassemble me, all parts and surfaces should be dry.

In addition, you should disinfect me regularly. If I am particularly dirty, you can use a 2% bleach solution. Please note that excessive use can lead to discoloration of the tube.

Sterilisation

If you want to enjoy me for a long time, please don't sterilise me.

Storage

After use, put me back in the packaging or in a safe place so that I don't get any defects on my chest piece!

Also, do not expose me to strong UV radiation or heat, since they can lead to my material fading or hardening.

Maintenance and warranty

You can replace my ear tips or membrane at any time.

The warranty for the entire product expires when using spare parts that were not purchased on the DocCheck shop.

If you use me as intended and follow these instructions for use, the warranty for manufacturing and material defects is two years, starting from the date of sale. Obvious misuse and accidental damage are excluded. Cosmetic damage, such as scratches on the surface, are also excluded from the warranty.

Check and functional test

Before use, please check that all my parts such as membranes or ear tips are assembled and securely in place.

Visual inspection for damage and wear is also useful.

Disposal

Please only dispose of me in accordance with local / regional / international guidelines.

Manufacturer

DocCheck Shop GmbH
Vogelsanger Str. 66, 50823 Cologne, Germany
[doccheckshop.com](https://www.doccheckshop.com)

Explanation of symbols

Medical device

CE mark

Manufacturer data (address)

Production date

Article number

Batch

Contains no latex/natural rubber

Contains aluminum

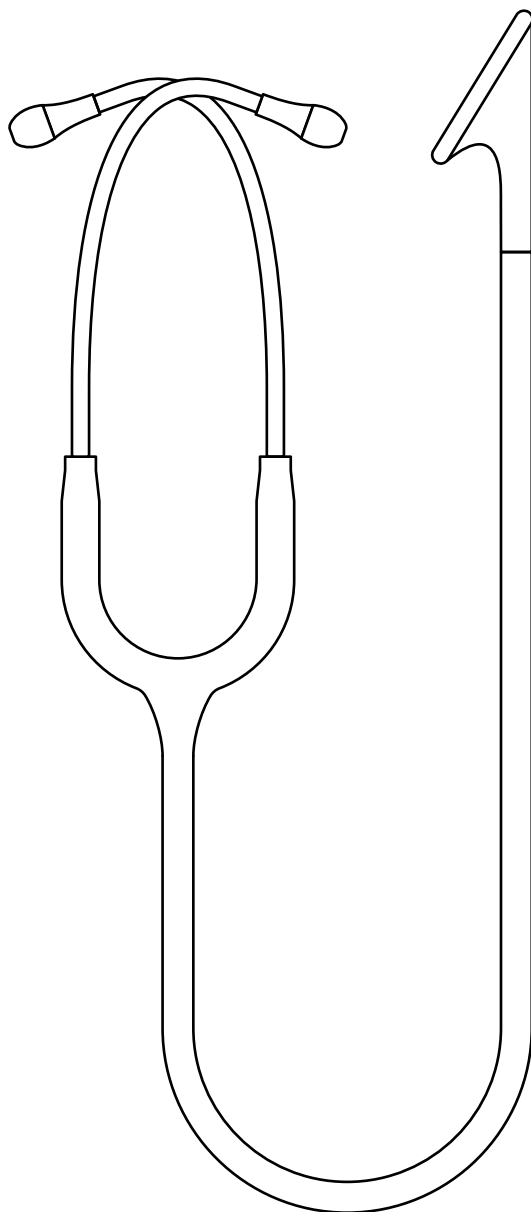
Contains copper

My hero
[doccheck.com](https://www.doccheck.com)

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