



DART™

Intranasal Mucosal Atomization Device


Quick Reference Guide

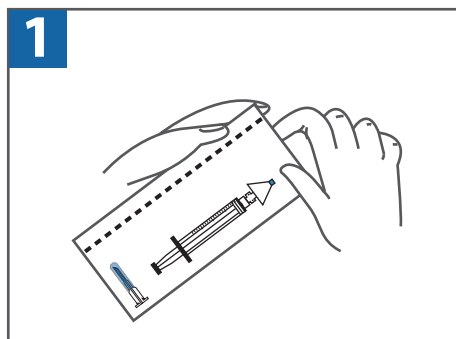


1 Remove the DART from the packaging.

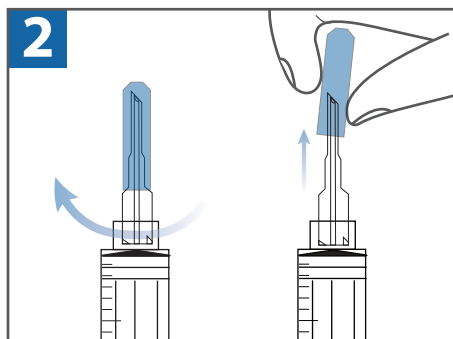


2 Twist the Vial Adapter on clockwise and remove the cap.

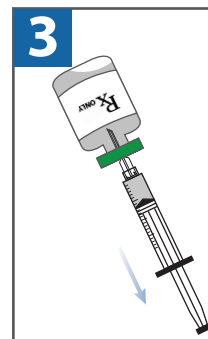


3 Add the desired medication to the syringe.

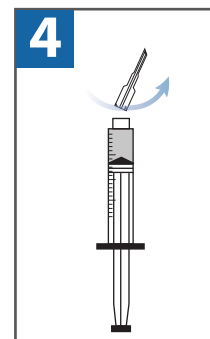


4 Twist Counter clockwise to remove the Vial Adapter

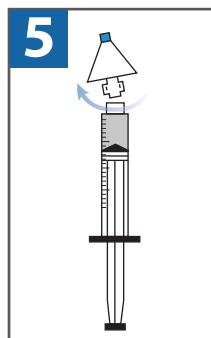

5 Reattach the Nasal device to the syringe (turn clockwise.)

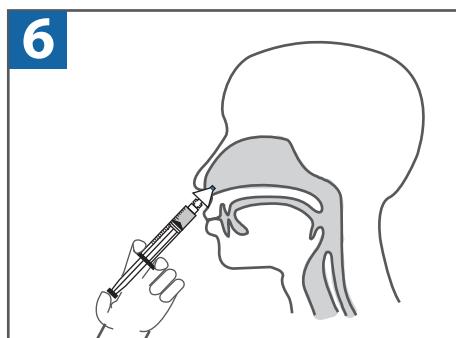

6 Place the tip of the device into the patients nostril. The device should be pointing slightly up and laterally (towards the tip of the patients ear on the same side of the head).

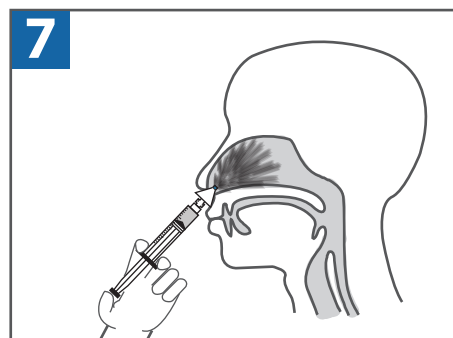

7 Compress the syringe plunger briskly until half the dose enters one nostril, repeat on the opposite side to administer the remaining medication.

8 Titrate to clinical effect in 5-15 minutes with another appropriate dose of medication if indicated.



DART100 - DART Nasal Device with 3mL Syringe



DART300 - DART Nasal Device without Syringe



DART140 - DART Nasal Device with 3mL Syringe and Vial Adapter

Additional Tips:

- Compress the Syringe with a brisk pressure to maximize the atomization.
- Utilize both nostrils to increase the treated surface area.
- Mucous, blood, and vasoconstrictors may reduce absorption of medication.