ultraMEDIC[®]

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Contents

Page

1. Product description	2
1.1 Technical data	2
1.2 Intended use of the product.	10
1.3 User groups.	10
1.4 Customer service.	10
1.5 Declaration of conformity	10
1.6 Functions	11
2. Information on the safety and protection of the product	11
3. Use of the ultraHEAD-LOCK I.	11
3.1 Fastening / Using with the spine board	12
3.2 Fastening / Using with the scoop stretcher.	13
4. Preparation for reuse	13
5. Cleaning and care	14
6. Maintenance	14
7. Storage	15
8. Service life	15
9. Identification and warranty certificate	16

Please read the operating manual and safety instructions carefully before you start using the ultraHEAD-LOCK I. Keep the operating manual in a safe place to avoid operating errors. Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealer or ultraMEDIC GmbH directly. Always quote the LOT- or serial number to identify the ultraHEAD-LOCK I.

1.2 Intended use of the product

The ultraHEAD-LOCK I is a head restraint system for use with spinal boards or scoop stretchers.

1.3 User groups

This product may only be used by qualified, expert personnel. A prerequisite is theoretical and practical experience in handling the head restraint system.

1.4 Customer service

The manufacturer grants a warranty period of 24 months on this product. The warranty only encompasses the costs of the spare parts.

Any shipping or labor costs incurred are not guaranteed. The following points are not covered by the warranty:

- transportation,
- operator error,
- incorrect, improper or no maintenance,
- damage caused by incorrect operation of the product,
- repairs carried out by unauthorized persons,
- damage caused by use in unsuitable conditions,
- normal wear and tear.

1.5 Declaration of conformity

The ultraHEAD-LOCK I was manufactured in accordance with the current EU regulation. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

1.6 Functions

The ultraHEAD-LOCK I consists of soft foam and has Velcro straps for head fixation. The material is PVC-free, X-rayable, easy to care and completely washable. The side openings in the center of the supporting cushions leave a patient's ears uncovered, thereby facilitating visual inspection of head injuries and communication between paramedics and patients.

2. Information on the safety and protection of the product

1. Before and after every use, check the components and patient restraining belts for damage and completeness.
2. Ensure that the patient restraining belts are correctly installed.
3. Before initial use, conduct a test run with the ultraHEAD-LOCK I to become aware of the handling of the product.
4. In the event of damage or suspected damage to the straps or the ultraHEAD-LOCK I, remove the product from operation immediately until any occurring irregularities have been remedied.
5. All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.


3. Use of the ultraHEAD-LOCK I

Manual immobilisation may not be interrupted when applying the ultraHEAD-LOCK I on the patient.


When taking care of a patient with suspected cervical spine injuries, dispensing with additional immobilisation aids can result in further injuries.

3.1 Fastening / Using with the spine board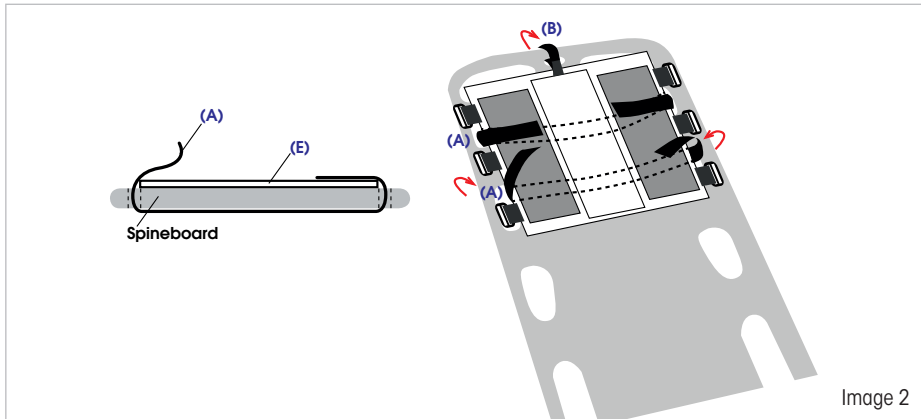

- Dismantle the head immobilising system into its individual components.
- Place the base plate (E) at the head end of the spine board. For fastening the base plate, the retaining straps (A) are threaded through and pulled out on both sides of the head part and fastened on the velcro strap of the base plate.
- The upper retaining strap (B) is threaded from the top through the hole at the end of the head piece and secured (image 2).
- In case of a head injury and/or a cervical spine injury, a cervical spine support collar is generally wrapped before the patient is moved. Moving is done by the usual methods employed by the emergency services, and will not be dealt with here.
- Attach the support pads to the large Velcro panel with the straight side next to the patient's head.
- Put the headband (C) over the forehead of the patient (velcro side upwards). In doing so, the padded middle part should lie uniformly on the forehead. For fastening, the two ends of the head band are threaded through the D-rings located on the base plate, carefully pulled tight and fastened.
- Correct the strap length if necessary.
- The position of the headband depends on the type of injury. Decide, depending on the conditions, whether applying the headband is necessary. Also with regard to the chin strap (D), check whether it is necessary to place it over the chin and the cervical spine supporting collar.


3.1 Fastening / Using with the scoop stretcher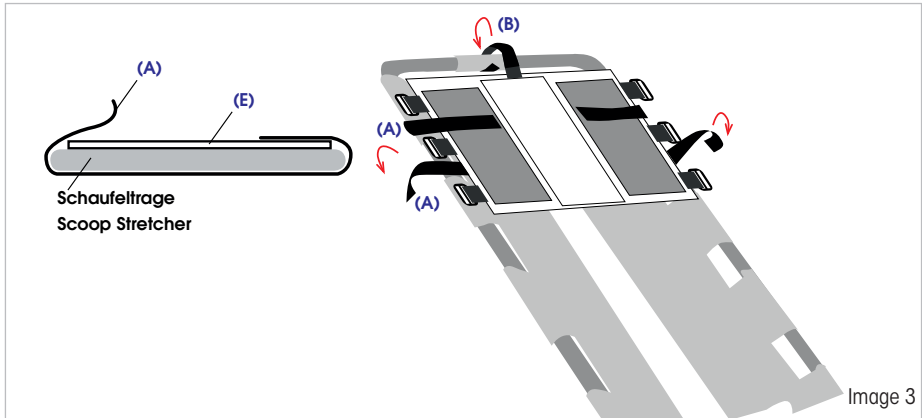

- In case of a suspicion of a cervical spine injury, generally, a cervical spine supporting collar is inserted, and then the scoop stretcher continues to be used.
- Place the patient on the scoop stretcher. **Manual immobilization must not be interrupted!**
- Carefully slide the base plate on the head section of the stretcher under the patient's head and secure it to the scoop stretcher with the two attachment straps (A).
- Place the upper retaining strap (B) around the tubular frame of the scoop stretcher in the area of the fastener from the outside inwards and secure it (image 3).
- Attach the support pads to the large Velcro panel with the straight side next to the patient's head.
- Place the headband (C) over the forehead of the patient (velcro side upwards). The padded middle portion should then be uniformly on the forehead. For fastening, the two ends of the head band are led through the D-rings located on the left and right of the base plate, pulled tight carefully and fastened.
- Correct the strap length if necessary.

4. Preparation for reuse

After proper cleaning, all parts of the ultraHEAD-LOCK I can be reused. ultraHEAD-LOCK can be folded up into a compact package and stowed in a space-saving manner. , follow the steps below:

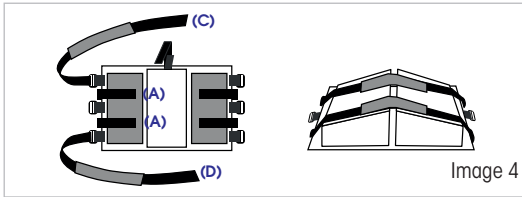- Thread the retaining straps (A) below the base plate and fasten them with the velcro straps (Image 4).
- Place the support cushion on the base plate and fasten it with the headband (C) and chin strap (D) through the D-rings.

This method of storage also facilitates fast fixing of the ultraHEAD-LOCK I when used again.

5. Cleaning and care

Never use products with aggressive agents (bleach, ammonia, etc.) for cleaning, avoid the use of scratching materials to remove stains such as brushes, steel wool and knife blades.

For **disinfection**, you can use commercially available alcohol-based agents. Other solutions or cleaning chemicals may damage the material.

Straps: The straps that have gotten wet should only dry naturally. Avoid direct heat radiation from heat sources. These factors could damage the material.

6. Maintenance

In order to ensure the safety of the user and the injured party during a rescue mission, we recommend that you have the ultraHEAD-LOCK I and straps visually inspected by an expert on an **annual** basis.

For German users, we recommend consideration to the following regulations and guidelines of MPBetreibV.

For users outside of Germany, a regular check according to country-specific requirements is required.

The following parts must be checked and the following defects must be excluded:

Base plate	deformation, tears, breaks, broken parts
Straps	Cutting and abrasion points, damaged seams, tissue-damaging stains
other damages	

In the event of damage or suspected damages, remove the product from operation immediately until any occurring irregularities have been remedied.

7. Storage

Store the ultraHEAD-LOCK I in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors can damage the material.

Protect the blocking devices and moving parts from dust and dirt deposits to maintain their maximum effectiveness during use.

8. Service life

The service life of the ultraHEAD-LOCK I depends on the individual conditions of use.

Depending on how frequently and intensely the ultraHEAD-LOCK I is used, wear and tear on the product is normal. In this regard, correct use is assumed. The material is subject to an ageing process even if handled carefully. The product should be discarded depending on its condition.

9. Identifizierungs- und Gewährleistungszertifikat
Identification and Warranty Certificate
Certificat d'identification et de garantie
Certificado de identificación y de garantia

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