

D-72336 Balingen

Tel: +49-[0]7433- 9933-0 Fax: +49-[0]7433-9933-149 

## **Operating instructions** Personal floor scale with BMI function

## KERN MPE

MPE 250K100HNM MPE 250K100PNM MPE 200K-1HEM **MPE 200K-1PEM** 

Version 5.5 2022-02 GB





D	Weitere Sprachversionen finden Sie online unter www.kern-sohn.com/manuals
BG	Други езикови версии ще намерите в сайта <u>www.kern-sohn.com/manuals</u>
DK	Flere sprogudgaver findes på websiden www.kern-sohn.com/manuals
EST	Muud keeleversioonid leiate Te leheküljel www.kern-sohn.com/manuals
E	Más versiones de idiomas se encuentran online bajo www.kern-sohn.com/manuals
GR	Άλλες γλωσσικές αποδόσεις θα βρείτε στην ιστοσελίδα <u>www.kern-sohn.com/manuals</u> s
F	Vous trouverez d'autres versions de langue online sous www.kern-sohn.com/manuals
LV	Citas valodu versijas atradīsiet vietnē www.kern-sohn.com/manuals
FIN	Muut kieliversiot löytyvät osoitteesta www.kern-sohn.com/manuals
LT	Kitas kalbines versijas rasite svetainėje <u>www.kern-sohn.com/manuals</u>
GB	Further language versions you will find online under www.kern-sohn.com/manuals
RO	Alte versiuni lingvistice veţi găţi pe site-ul www.kern-sohn.com/manuals
I	Trovate altre versioni di lingue online in www.kern-sohn.com/manuals
SK	Iné jazykové verzie nájdete na stránke www.kern-sohn.com/manuals
NL	Bijkomende taalversies vindt u online op www.kern-sohn.com/manuals
SLO	Druge jezikovne različice na voljo na spletni strani <u>www.kern-sohn.com/manuals</u>
Р	Encontram-se online mais versões de línguas em www.kern-sohn.com/manuals
CZ	Jiné jazykové verze najdete na stránkách www.kern-sohn.com/manuals
PL	Inne wersje językowe znajdą Państwo na stronie <u>www.kern-sohn.com/manuals</u>
SE	Övriga språkversioner finns här: www.kern-sohn.com/manuals
Н	A további nyelvi változatok a következő oldalon találhatók: www.kern-sohn.com/manuals
HR	Druge jezične verzije su dostupne na stranici: www.kern-sohn.com/manuals
NO	Andre språkversjoner finnes det på www.kern-sohn.com/manuals



## **KERN MPE**

Version 5.5 2022-02

# Operating instructions Personal floor scale with BMI function

## Contents

<b>1</b> 1.1	Tolerances height meter	_
<b>2</b> 2.1	Declaration of conformity  Explanation of the graphic symbols for medical devices	
3	Appliance overview	11
4	Keyboard overview	13
5	Overview of display	14
<b>6</b> 6.1	Basic Information (General)	14
6.2 6.3 6.4	Proper use	16
6.5 6.6 6.7	Monitoring of Test Resources  Plausibility checkReport of serious incidents	18
<b>7</b> 7.1 7.2 7.3 7.4	Basic Safety Precautions  Pay attention to the instructions in the Operation Manual  Personnel training  Preventing contamination  Preparation for use	19 19 19
8 8.1 8.2 8.3 8.3.1 8.4	Electromagnetic compatibility (EMC)  General hints  Electromagnetic interferences  Electromagnetic noise immunity  Crucial features of performance  Minimum distances	20 21 22
<b>9</b> 9.1 9.2	Transport and storage  Testing upon acceptance  Packaging / return transport	25
10 10.1 10.2 10.3 10.3.1 10.3.2 10.4 10.4.1 10.5 10.6	Unpacking, Setup and Commissioning.  Installation Site, Location of Use. Unpacking Scope of delivery. MPE-HM and MPE-PM models MPE-HEM and MPE-PEM models Balance assembly and installation Installing the body height meter Mains connection (only MPE-NM models). Battery operation is possible by obtaining an optional battery power pack	25 26 26 26 27 28
10.6	Battery operation is possible by obtaining an optional battery power pack	30

10.7	Battery operation	31
10.8	Initial Commissioning	
11	Operation	32
11.1	Weighing	
11.2	Taring	33
11.2.1	Subsequent tare weight	
11.3	HOLD function	
11.4	Show second decimal place (non-verified value)	
11.5 11.5.1	Calculation of the Body Mass Index	
11.5.1	Measuring body height (MPE-HM and MPE-HEM only)	
11.5.2	Classification of BMI values	
11.6	Automatic switch-off function "AUTO OFF"	
11.7	Display background illumination	39
12	The menu	40
12.1	Navigation in the menu	40
12.2	Menu overview	41
13	Data output RS 232	44
13.1	Pin allocation of balance output bushing:	44
13.2	Technical data	44
13.3	Printer operation	45
14	Error messages	45
15	Service, maintenance, disposal	46
15.1	Cleaning	
15.2	Cleaning / disinfecting	46
15.3	Sterilisation	
15.4	Service, maintenance	
15.5	Disposal	47
16	Instant help	47
17	Verification	48
17.1	Verification validity period (current status in G)	
18	Adjustment	51
19	Equipment (option)	
	Equipment (option)	

## 1 Technical data

KERN (Type)	MPE 250K100HNM	MPE 250K100PNM	
Model	MPE 250K100HM	MPE 250K100PM	
Display	6-d	ligit	
Weighing capacity (max)	250	) kg	
Minimum load (Min)	2	kg	
Verification value (e)	10	0 g	
Repeatability	0.1	kg	
Linearity ±	0.1	kg	
Display	LCD with 25n	nm high digits	
Recommended adjustment weight, (Class)	≥ 20 (M	0 kg 11)	
Stabilization time (typical)	3 s	ec.	
Warm-up time	10	min	
Operating temperature	0° C	. + 40° C	
Humidity of air	max. 80 % (no	ot condensing)	
Electric Supply	Input voltage 100 V - 240 V, 50 / 60 Hz		
Auto Off	After 3 min without load change (adjustable)		
Balance (W x D x H) mm	365 x 570 x 2134 365 x 570 x 1030		
Weighing plate mm	365 x 360 x 80		
Weight kg (net)	11.5	10.8	
Verification according to 2014/31/EU	class III		
Medical device according to 93/42/EEC	class I with measuring function (Im)		
Height measuring stick in tripod integrated, extendable (from 88 cm to 205 cm)	✓	-	
Rechargeable battery operation	optional; 6 x 1.2 VAA = 7.2 V/2000 mA		
Batteries	6 x 1.5 V AA		
Data interface provided as standard	RS 232 C		

KERN (Type)	TMPE 250K-1HEM-A	TMPE 250K-1PEM-A	
Model	MPE 250K-100HEM	MPE 250K-1PEM	
Display	6-d	ligit	
Weighing capacity (max)	250	) kg	
Minimum load (Min)	2	kg	
Verification value (e)	10	0 g	
Repeatability	0.1	kg	
Linearity ±	0.1	kg	
Display	LCD with 25n	nm high digits	
Recommended adjustment weight, (Class)	≥ 20 (M	•	
Stabilization time (typical)	3 s	ec.	
Warm-up time	10	min	
Operating temperature	0° C	. + 40° C	
Humidity of air	max. 80 % (not condensing)		
Electric Supply	Input voltage 100 V - 240 V, 50 / 60 Hz		
Auto Off	After 3 min without load change (adjustable)		
Balance (W x D x H) mm	365 x 570 x 2134	365 x 570 x 1030	
Weighing plate mm	365 x 3	60 x 80	
Weight kg (net)	11.5	10.8	
Verification according to 2014/31/EU	clas	s III	
Medical device according to 93/42/EEC	class I with measu	uring function (Im)	
Height measuring stick in tripod integrated, extendable (from 88 cm to 205 cm)	-		
Rechargeable battery operation	optional; 6 x 1.2 VAA = 7.2 V/2000 mA		
Batteries	6 x 1.5 V AA		
Data interface provided as standard	RS 232 C		

Note on "Medical product in accordance with 93/42/EEC"

## 1.1 Tolerances height meter

Measured value (cm)	Tolerance (cm)
90	± 0.5
100	± 1.0
150	± 1.0
200	± 1.0

## 2 Declaration of conformity

To view the current EC/EU Declaration of Conformity go to:


www.kern-sohn.com/ce

The scope of delivery for verified weighing balances (= conformity-rated weighing balances) includes a Declaration of Conformity.

Solely these weighing balances are classified as medical devices.

## 2.1 Explanation of the graphic symbols for medical devices

All medical scales marked in this way meet the requirements of the following directives:

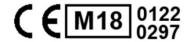

- 1. 2014/31/EU: Directive on non-automatic weighing instruments
- 2. 93/42/EC: Directive concerning medical devices for "Medical product in accordance with 93/42/EEC"



Scales marked in this way underwent the conformity assessment procedure according to Directive 2014/31/EU for scales belonging to 3rd accuracy class.

WF 170012

Designation of the serial number of every device, applied at the device and on the packaging

Number here as example



Identification of the manufacturing date of the medical product.

Year and month here as example



"Please note the accompanying documents" or "Please note the operating instructions"



"Please note the operating instructions"



"Please note the operating instructions"



Identification of manufacturer of medical product including address



"Electro-medical appliance" with attachment for type B



Device protection category II



Dispose of old appliances separately from your household waste!

Instead, take them to communal collection points.



Display of supply voltage for scales with polarity display.



Mains connection




Sealing mark KERN SEAL



Supply voltage direct current



Information

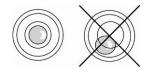


Level balance before use



Electrostatically endangered structural components


## 3 Appliance overview

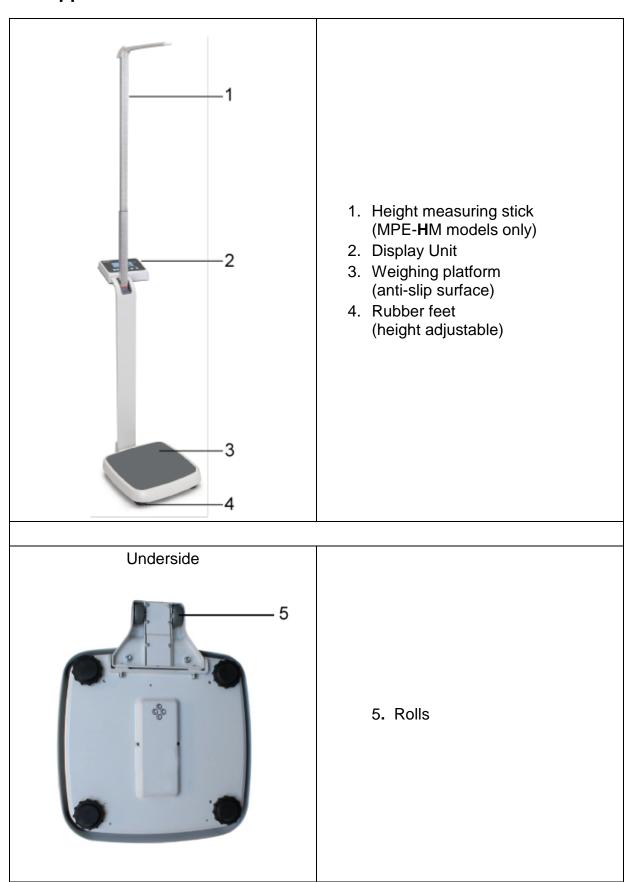

## Secondary display at rear





## Display unit at rear

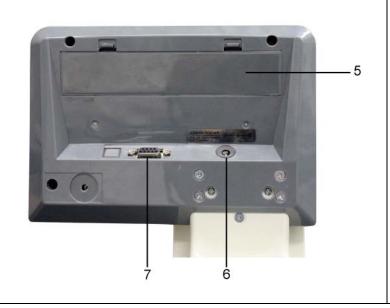

- 5 Rechargeable battery compartment
- 6 Mains connection (MPE-NM)
- 7 RS 232 C

## 4 Keyboard overview

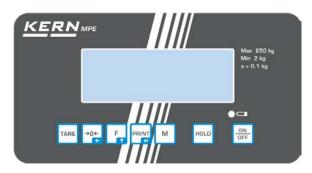

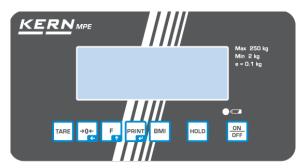

Type MPE 250K100HNM Type MPE 250K100PNM Type TMPE 250K-1HEM-A Type TMPE 250K-1PEM-A

Key	Description	Function
ON OFF	ON/OFF-switch	Turn on/off
HOLD	HOLD button	Hold function/Calculation of a stable weight value
M or BMI	BMI key	Calculation of the Body Mass Index
PRINT	PRINT button	Data transfer via interface In menu:
F	Function key	In menu:
→0 <i>←</i>	Zeroing key	Weighing scale will be reset to "0.0"  For numeric entry:  • Change decimal place
TARE	TARE key	Tare balance

## 5 Overview of display

Display	Description	Description
	Stability display	Scales are in a steady state
<b>→0</b> ←	Zeroing display	Should the balance not display exactly zero despite empty weighing plate, press the button. Your balance will be set to zero after a short standby time.
NET	Net weight display	Illuminated when net weight is displayed Illuminated after weighing scale was tared
GROSS	Gross weight display	Illuminated when gross weight is displayed
HOLD	HOLD function	HOLD function active
ВМІ	BMI function	Illuminated while BMI function is enabled

## 6 Basic Information (General)



Weighing instruments have to be verified for the purposes stated below in accordance with Directive 2014/31/EC. Article 1, paragraph 4. "Determination of mass in the practice of medicine that is, weighing patients for reasons of medical supervision during medical surveillance, examination and treatment."

## 6.1 Specific function

#### Indication •

- Determining the body weight in the medical practice area.
- Use as "non-standalone weighing scale", that is, a person steps carefully onto the weighing platform's centre. Once a steady display value is shown, you can read the weight value.

## Contra- • indication

No contraindication known

## 6.2 Proper use

This weighing scale is designed for determining the weight of a person whilst standing, such as in doctor's surgeries. The balance is suitable for recognising, preventing and controlling illnesses.



Scales fitted with a serial interface may only be connected to appliances in compliance with Directive EN60601-1.

On personal weighing scales, the person should step onto the centre of the weighing platform and remain standing without moving.

As soon as a stable weighing value is reached the weighing value can be read. The weighing scale is designed for continuous duty.



The weighing platform may only be stepped on by persons capable of standing on both feet on the weighing platform.

The weighing platforms are fitted with an anti-slip surface that must not be covered during weighing a person.

The balance should be checked for correct condition prior to each utilisation by a person familiar with proper operation of the balance.

When using balance with body height measuring rod, ensure that the top flap is turned downwards immediately after use in order to avoid risk of injury.



If the balance doesn't have any contact with the transfer cable, do not touch the transfer port in order to avoid an ESD-failure.



### Non-intended product use / contraindications

Do not use these scales for dynamic weighing processes. Do not leave permanent load on the weighing pan. This may damage the measuring system.


Impacts and overload exceeding the stated maximum load (max) of the weighing plate, minus a possibly existing tare load, must be strictly avoided. This could cause damage to the balance. Never operate balance in explosive environment. The serial version is not explosion protected. It should be noted that a flammable mixture of anaesthetics and oxygen or laughing gas may occur.



The structure of the balance may not be modified. This may lead to incorrect weighing results, safety-related faults and destruction of the balance.

The balance may only be used according to the described conditions. Other areas of use must be released by KERN in

If the balance is not used for a longer time, take out the batteries and store them separately. Leaking battery liquid could damage the balance.

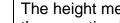

## Improper use of the optional height meter



The height meter may only be assembled such as described in the operating instructions. The structure of the body height measuring rod may not be modified. This may cause incorrect measurement results, safety-related faults as well as destruction. The height meter may only be used according to the defined specifications. Other areas of use must be released by KERN in writing.

### 6.4 Warranty

Warranty claims shall be voided in case

- Our conditions in the operation manual are ignored
- The appliance is used outside the described uses
- The appliance is modified or opened
- Mechanical damage and damage caused by media, liquids,
- Natural wear and tear
- The appliance is improperly set up or incorrectly electrically connected
- The measuring system is overloaded
- Dropping the balance

### 6.5 Monitoring of Test Resources

In the framework of quality assurance the measuring-related weighing properties of the balance and, if applicable, the testing weight, must be checked regularly. The responsible user must define a suitable interval as well as type and scope of this test. Information is available on KERN's home page (<a href="www.kern-sohn.com">www.kern-sohn.com</a> with regard to the monitoring of balance test substances and the test weights required for this. In KERN's accredited DKD calibration laboratory test weights and balances may be calibrated (return to the national standard) fast and at moderate cost.

Using measuring technology to check the accuracy of the measuring device is recommended for personal floor scales with body height measurement but is not absolutely essential as the calculation of the human body height is always subject to a great deal of inaccuracy.

### 6.6 Plausibility check

Please make sure that the measurement values computed by the appliance are plausible and are allocated to the respective patient, before storing and using the values for further purposes. This applies especially also for values transferred via interface.

## 6.7 Report of serious incidents

All serious incidents appeared related to this product must be reported to the manufacturer and the responsible authority of the member state where the user and/or the patient are residents.

"Serious incident" that means an incident which directly or indirectly had, could have or could have had one of the following consequences:

- the death of a patient, a user or another person,
- > the temporary or permanent fatal deterioration of the health status of a patient, a user or other persons,
- > a serious danger for public health,

## 7 Basic Safety Precautions

## 7.1 Pay attention to the instructions in the Operation Manual



 ⇔ Carefully read this operation manual before setup and commissioning, even if you are already familiar with KERN balances.



#### 7.2 Personnel training

The medical staff must apply and follow the operating instructions for proper use and care of the product.

## 7.3 Preventing contamination

The prevention of cross-contamination (fungal skin infections ...) requires regular cleaning of the weighing platform. Recommendation: after a weighing procedure that could potentially result in contamination (e. g. after weighing that involves direct skin contact).

#### 7.4 Preparation for use

- Check the personal balance for damage before any use
- Maintenance and re-verification (in Germany MTK)
   The personal balance must undergo maintenance and re-verification at regular intervals.
- Do not use the appliance on slippery surfaces or in rooms with risk of vibration
- During installation the personal balance must be levelled.
- If possible, the product must remain in its original packaging during the transport. Should this not be possible, make sure that the product is protected against damage.
- Step onto and leave the personal balance only when a qualified person is present

## 8 Electromagnetic compatibility (EMC)

#### 8.1 General hints



The installation and use of this electrical medical device requires special precautionary measures as outlined in the EMC information below.

This device complies with the limits set for medical electrical devices of group 1, class B (as per EN 60601-1-2).

Electromagnetic compatibility (EMC) describes a device's ability to perform reliably within an electromagnetic environment without causing inadmissible electromagnetic interference at the same time. Amongst other things, such disturbances may be emitted by connecting cables or the air.

Inadmissible disturbances from the environment may result in incorrect displays, inaccurate measured values or incorrect behaviour of the medical device. By the same token the medical device may in some cases cause such disturbances in other devices. To eliminate problems of that kind, we recommend you to take one or several of the measures listed below:

- Change the alignment or distance of the device to the source of EMI.
- Install or use the floor scales MPE at a different location.
- Connect the floor scales MPE to a different power source.
- For further questions please contact our customer services.

Disturbances may be caused by improper modification or add-ons to the device or not recommended accessories (such as power units or connecting cables). The manufacturer will not be responsible for these. Modifications may also result in a loss of authorisation relating to the use of the device.



Devices emitting high frequency signals (mobile telephones, radio transmitters, radio receivers) may cause interference in the floor scale MPE. For that reason do not use them near the floor scale MPE. Chapter 8.4 contains details about recommended minimum distances.

## 8.2 Electromagnetic interferences

## Guidelines and manufacturer's declaration – electromagnetic interferences

The floor scales MPE is designed for use in an electromagnetic environment that meets the requirements stated below. The customer or user of the medical electrical device must ensure that operation takes place in such an environment.

Emitted interference measurements	Conformity	Electromagnetic environment - guideline	
HF emissions as per CISPR 11 / EN 55011	Group 1	The floor scales MPE uses HF energy merely for its internal working. Its HF emission therefore is very low and it is highly unlike to interfere with adjacent electronic devices.	
HF emissions as per CISPR 11 / EN 55011	Class B	The floor scales MPE is designed for use in all equipment including those in living areas and those	
Emission of harmonics as per IEC 61000-3-2	Class A	connected directly to the public power grid that also supplies buildings used for living purposes.	
Emission of voltage fluctuations / flicker	Conforms with		
as per IEC 61000-3-3			

Do not put the floor scale MPE directly next to other devices and do not stack it with other devices. If this type of operation is necessary, observe the floor scale MPE to ensure normal operation in such an arrangement.

## 8.3 Electromagnetic noise immunity

## Guidelines and manufacturer's declaration - electromagnetic noise immunity

The floor scales MPE is designed for use in an electromagnetic environment that meets the requirements stated below. The customer or user of the medical electrical device must ensure that operation takes place in such an environment.

Noise immunity tests	IEC 60601 test level	Conformity	Electromagnetic environment - guideline
Discharge static electricity (DSE) as per IEC 61000-4-2	± 6 kV contact discharge ± 8 kV air discharge	± 6 kV ± 8 kV	Floors should be made of wood or concrete or tiled with ceramic tiles. If floors are covered with synthetic material, relative air humidity must be at least 30%.
Fast transient electrical disturbances / bursts as per IEC 61000-4-4	± 2 kV for power lines  ± 1 kV for input and output lines	± 2 kV ± 1 kV	The quality of the supply voltage should match that of the typical business or hospital environment.
Impulse voltages / surges as per IEC 61000-4-5	± 1 kV voltage Live wire - live wire  ± 2 kV voltage Live wire - earth	± 1 kV Inapplicable	The quality of the supply voltage should match that of the typical business or hospital environment.
Voltage dips, short-term disruptions and fluctuations in supply voltage as per IEC 61000-4-11	< 5 % U <sub>T</sub> (> 95 % dip of U <sub>T</sub> ) for ½ period  40 % U <sub>T</sub> (> 60 % dip of U <sub>T</sub> ) for 5 periods  70 % U <sub>T</sub> (> 30 % dip of U <sub>T</sub> ) for 25 periods  < 5 % U <sub>T</sub> (> 95 % dip of U <sub>T</sub> ) for 5 s	Compliance with requirements under all postulated conditions  Controlled switch off Return to undisturbed situation after user intervention.	The quality of the supply voltage should match that of the typical business or hospital environment. Where the user of the medical device demands continuous operation even during disruptions to the power supply, we recommend powering the floor scale MPE by no-break power supply or battery.
Magnetic field for supply frequency (50/60 Hz) as per IEC 61000-4-8	3 A/m	3 A/m 50/60 Hz	Magnetic fields for the supply frequency should match the typical values found in the particular business or hospital environment.

NOTE Urequals AC line voltage prior to application of test level.

## Guidelines and manufacturer's declaration - electromagnetic noise immunity

The floor scales MPE is designed for use in an electromagnetic environment that meets the requirements stated below. The customer or user of the medical electrical device must ensure that operation takes place in such an environment.

Noise immunity tests	IEC 60601 test level	Conformity	Electromagnetic environment - guideline
Conducted HF disturbance variables as per IEC 61000-4-6	3 V <sub>ms</sub> 150 kHz to 80 MHz	3 V	Do not use portable or mobile radio sets nearer to the floor scales MPE or its wires than the distance recommended as safety distance which is calculated according to the equation relevant for its
Emitted HF disturbance variables as per IEC 61000-4-3	3 V <sub>ms</sub> 80 MHz to 2.5 GHz	3 V/m	according to the equation relevant for its transmission frequency.  Recommended safety distance:  d = 1.2√P  d = 1.2√P  for 80 MHz to 800 MHz  d = 2.3√P  for 800 MHz to 2.5 GHz  Use P as rated capacity of radio transmitter in Watt (W) as per details given by the radio transmitter manufacturer and d as recommended safety distance in metres (m).  The field intensity of stationary radio transmitters should for all frequencies be lower according to an in situ a examination than the conformity level.b  Interference may occur near devices bearing the symbol below.

NOTE 1 Higher frequency range applies to 80 MHz and 800 MHz.

NOTE 2 These guidelines may not be applicable in all cases.

The spread of electromagnetic variables is influenced by absorption and reflections in buildings, objects and humans.

- The field intensity of stationary radio transmitters such as base stations of wireless telephones and mobile radio sets, amateur radio stations, AM and FM radio and television stations cannot be reliably predicted in advance. To determine the electromagnetic environment in respect of stationary transmitters, you should consider a study of electromagnetic phenomena at the location. If the measured field intensity at the location where the floor scales MPE is to be used exceeds the conformity level above, you should observe the floor scales MPE in order to ensure normal operation. If you observe unusual features of performance you may have to take additional measures such as a change in alignment or a different location for the floor scale MPE.
- b For a frequency range of 150 kHz to 80 MHz field intensity should be less than 3 V/m.

### 8.3.1 Crucial features of performance



The floor scales MPE does not have any crucial features of performance as per IEC 60601-1. The system may be subject to interference by other devices even if these devices conform to current emission requirements as per CISPR.

#### 8.4 Minimum distances

## Recommended safety distances between portable and mobile HF telecommunication devices and the medical device

The floor scales MPE is designed for use in an electromagnetic environment in which HF disturbance variables are controlled. The customer or user of the medical electrical device can help avoiding electromagnetic disturbances by keeping the minimum distance between portable and mobile HF telecommunication devices (transmitters) and the floor scales MPE – depending on the output performance of the communication device, as stated below.

Rated capacity of transmitter	The safety distance depends on the transmission frequency m			
	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3\sqrt{P}$	
0.01	0.12	0.12	0.23	
0.1	0.38	0.38	0.73	
1	1.20	1.20	2.30	
10	3.80	3.80	7.30	
100	12.00	12.00	23.00	

For transmitters with a maximum rated capacity not stated in the table above you can calculate the recommended safety distance in metres (m) yourself by using the equation belonging to each column, whereby P equals the maximum rated capacity of the transmitter in Watt (W) as per details provided by the transmitter manufacturer.

NOTE 1 Higher frequency range applies to 80 MHz and 800 MHz.

buildings, objects and humans.

NOTE 2 These guidelines may not be applicable in all cases.

The spread of electromagnetic variables is influenced by absorption and reflections in

MPE HM / MPE PM-BA-e-2255

## 9 Transport and storage

#### 9.1 Testing upon acceptance

When receiving the appliance, please check packaging immediately, and the appliance itself when unpacking for possible visible damage.

## 9.2 Packaging / return transport



- ⇒ Only use original packaging for returning.
- ⇒ Prior to dispatch disconnect all cables and remove loose/mobile parts.
- ⇒ Reattach possibly supplied transport securing devices.
- ⇒ Secure all parts such as the weighing platform, power unit etc. against shifting and damage.

## 10 Unpacking, Setup and Commissioning

#### 10.1 Installation Site, Location of Use

The balances are designed in a way that reliable weighing results are achieved in common conditions of use.

You will work accurately and fast, if you select the right location for your balance.

#### On the installation site observe the following:

- Place scales on a stable, even surface;
- Avoid extreme heat as well as temperature fluctuation caused by installing next to a radiator or in the direct sunlight;
- Protect the balance against direct draughts due to open windows and doors;
- Avoid jarring during weighing:
- Protect the balance against high humidity, vapours and dust;
- Do not expose the device to extreme dampness for longer periods of time.
   Non-permitted condensation (condensation of air humidity on the appliance) may occur if a cold appliance is taken to a considerably warmer environment.
   In this case, acclimatize the disconnected appliance for ca. 2 hours at room temperature.
- Avoid static charge of the balance and of the person to be weighed.
- Avoid contact with water.

Major display deviations (incorrect weighing results) may be experienced should electromagnetic fields (e.g. due to mobile phones or radio equipment), static electricity accumulations or instable power supply occur. Change location or remove source of interference.

## 10.2 Unpacking

Remove the individual components of the balance or the complete balance from the packaging with care and install at the intended location. When using the power pack, ensure that the power cable does not produce a risk of stumbling.

## 10.3 Scope of delivery

#### 10.3.1 MPE-HM and MPE-PM models

- Ralance
- Power pack unit (EN 60601-1 attestation of conformity)
- Operating instructions

#### 10.3.2 MPE-HEM and MPE-PEM models


- Weighing scale with tripod
- Batteries 6 x AA 1,5 V
- Operating instructions

## 10.4 Balance assembly and installation


## Refitting Procedure:


 ⇒ Mount tripod (1) on tripod holder (2) at weighing platform

Ensure that the cable (3) does not get clamped!

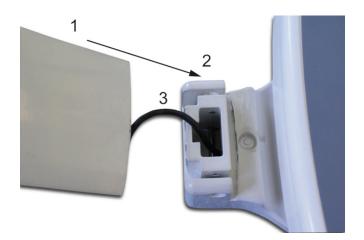

⇒ Fasten tripod, using 4 screws

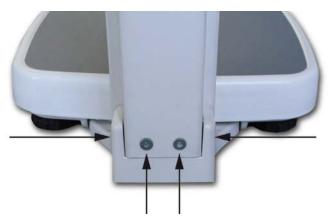

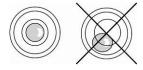

- ⇒ Level balance with foot screws until the air bubble of the water balance is in the prescribed circle.
- ⇒ Check levelling regularly.

## 10.4.1 Installing the body height meter

The force required to extend the telescopic body height meter may be adjusted using to adjustment bolts on the stand (see the figure).

To do it, proceed as follows:



⇒ Remove both plastic plugs in the lower part of the stand.



⇒ Using an appropriate flat-head screwdriver, set the desired force with two adjustment bolts.

(When you use it frequently, remember that the process must be repeated after some time).





Once installation is complete, check all screws for tight fit. Otherwise the person to be weighed may suffer an injury.

## 10.5 Mains connection (only MPE-NM models)

Power is supplied by the external power unit which also serves to isolate the mains supply from the scale. The stated voltage value must be the same as the local voltage.

Always use genuine approved KERN power pack units as per EN 60601-1 directive.

The small sticker attached to the side of the display unit indicates the power port:




The LED remains illuminated as long as the weighing scale remains connected to the mains.

The LED display informs you during loading about the loading status of the rechargeable battery.

green: Rechargeable battery is completely discharged

**blue:** Charging rechargeable battery

### 10.6 Battery operation is possible by obtaining an optional battery power pack.

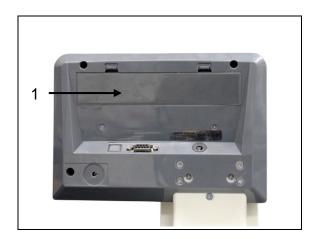

Open the battery compartment cover (1) at the base of the display unit and insert the rechargeable battery. Charge the battery for at least 12 hours before initial use.

The appearance of the symbol in the weight display indicates that the battery is almost exhausted. The weighing scale will remain ready for operation for a few more minutes before switching off in order to save battery (s. chap. 11.6 Auto off). Load rechargeable battery.

- Voltage has dropped below prescribed minimum.
- Rechargeable battery very low.
- Rechargeable battery completely reloaded

If the balance is not used for a longer time, take out the rechargeable battery and store it separately. Leaking liquid could damage the balance.

### 10.7 Battery operation

As an alternative to rechargeable battery operation, the balance may also be operated with 6x AA batteries.

Open battery compartment cover (1) at the lower side of the display unit and insert batteries according to the example below. Lock the battery cover again. If the

batteries are empty, in the balance display appears the symbol Change batteries. To save battery power, the balance switches off automatically


(see chap.11.6 Auto off).

Capacity of batteries exhausted.

Batteries will soon be flat.

Batteries are completely charged

#### **Insert batteries**

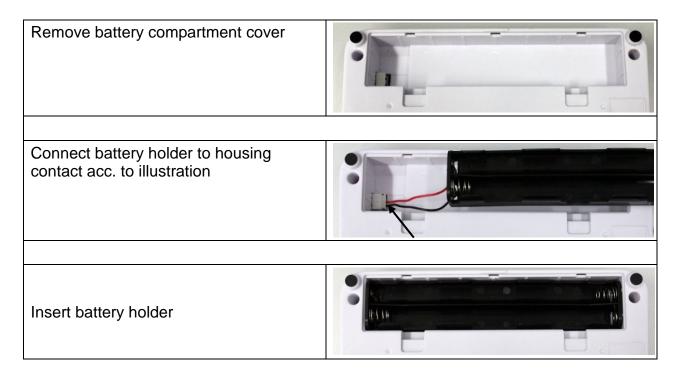

Insert batteries into battery compartment and lock with battery compartment cover.




## 10.8 Initial Commissioning

In order to obtain exact results with the electronic balances, your balance must have reached the operating temperature (see warming up time chap. 1). During this warming up time the balance must be connected to the power supply (mains, accumulator or battery) and be switched on.

The accuracy of the balance depends on the local acceleration of gravity. The value of gravity acceleration is shown on the type plate.

## 11 Operation

## 11.1 Weighing

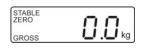

Start balance by pressing The balance will carry out a self-test
The scales are ready for operation as soon as the weight display for "0.0 kg" has appeared.




- However, you can reset the weighing scale to zero by pressing the key.



 If the person is heavier than the weighing range, "OL" (=overload) will appear in the display.

## 11.2 Taring


The tare weight of any preloads can be deducted by pressing a button so that the actual weight of the person is displayed in subsequent weighings.

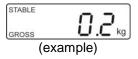

⇒ Place an object onto the weighing plate



⇒ Press the zero display appears.
"NET" is shown at the bottom on the left.

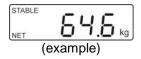

⇒ Allow the person to step onto the centre of the weighing platform. Wait until the standstill display "STABLE" appears, then read the weighing result.



- When the balance is unloaded the saved taring value is displayed with negative sign.
- To delete the stored tare value, release scales and press




### 11.2.1 Subsequent tare weight


The balance can be tared several times successively.

#### 11.3 HOLD function


The balance has an integrated standstill function (mean value calculation). With this function it is possible to weigh people accurately even if they do not stand still on the weighing plate.

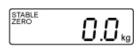

⇒ Start balance by pressing off.
Wait for stability display "STABLE to appear.

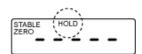

⇒ Press in the display "----- will appear and the "HOLD" symbol appears.

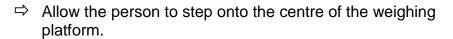



After a short time the stability display "STABLE" appears and the weighing value of the person is displayed and "frozen".



After unloading the balance, the weighing value remains displayed for approx. 10 seconds, than the balance changes automatically into the weighing mode. The symbol "HOLD" disappears.




There is no average value calculation in the event of too much movement.

## 11.4 Show second decimal place (non-verified value)

Press and hold for about 2 s whilst weighed result is being shown. The third decimal place will be shown for approx. 5 s.

#### 11.5 Calculation of the Body Mass Index

You need to know a person's body height before you can calculate the BMI for that person. It should either be known or can be determined directly with the MPE-HM and MPE-HEM model.

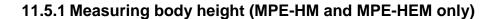



- ⇒ Push measuring stick upwards and set the stopper horizontally.
- ⇒ Push measuring rod carefully down until the headpiece touches the person's head. (It is recommended to take measurements with shoes removed).



A fixed stopper pointing outwards poses a risk of injury.



⇒ Read body height on measuring stick.



A correctly exercised body height measurement will achieve an accuracy of up to 5 mm.

# 11.5.2 Calculating Body Mass Index

You need to know the body height before you can calculate the BMI for the corresponding person. It should either be known or can be measured with the MPE-HM model.




- ⇒ Start balance by pressing
- ⇒ Wait for stability display "STABLE" to appear.



Press or or.

The most recently entered body height will be shown with the enable digit flashing. The "BMI" symbol lights up.

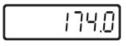

⇒ To enter body height, press the and key.



⇒ Confirm your entered value by pressing BMI "0.0" is displayed



Allow the person to step onto the centre of the weighing platform. "-----, is shortly displayed, followed by the BMI value of the person.

- □ Unload weighing plate
- STABLE ZERO O.O kg
- ⇒ To return to weighing mode, press → or →. The BMI symbol will disappear and the kg display will reappear.



- Reliable calculation of BMI is restricted to a body height of 100 cm to 200 cm and a weight of >10 kg.
- If weighing has to take place under unsteady conditions, you can be stabilise the display by applying the Hold function.

### 11.5.3 Classification of BMI values

Weight classification for adults over 18 years of age using the BMI in accordance with WHO, 2000 EK IV and WHO 2004.

Categorie	BMI ( kg/m²)	Risk of diseases associated with overweight
Underweight	< 18.5	low
Normal weight	18.5 – 24.9	Average
Overweight	<u>≥</u> 25.0	
Pre-adipose	25.0 – 29.9	A bit high
Adipose degree I	30.0 – 34.9	High
Adipose degree II	35.0 – 39.9	up
Adipose degree III	<u>≥</u> 40	Very high

# 11.6 Automatic switch-off function "AUTO OFF"

The balance offers the possibility of an automatic switch-off. It can be adjusted: Selectable between 3, 5, 15, 30 seconds and off.




Menu settings:


**[F1 oFF]** ⇒ **[oFF 0/3/5/15/30]** (see chap. 12)




- ⇒ Start balance by pressing C
  - Wait for stability display "STABLE to appear.

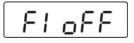

⇒ Press , [F1 oFF] is displayed

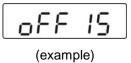


⇒ Press to display the most recently set time, such as [oFF 15]

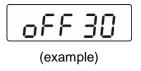


⇒ Press repeatedly until the desired time is displayed, e.g. [oFF 30]

[oFF 0]	AUTO OFF - function disabled
[oFF 3]	Weighing system will be turned off after 3 min.
[oFF 5]	Weighing system will be turned off after 5 min.
[oFF 15]	Weighing system will be turned off after 15 min.
[oFF 30]	Weighing system will be turned off after 30 min.

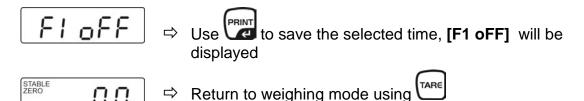

# 11.7 Display background illumination



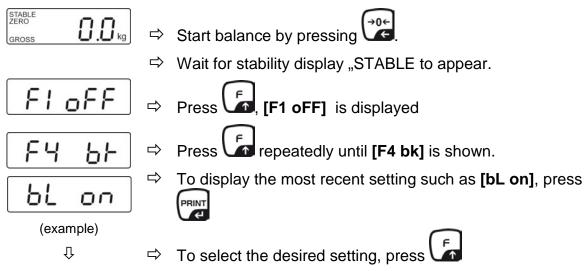

bL on	Continuous background lighting	
bL off	Background illumination off	
bL Auto	Automatic background illumination on when weighing pate is loaded or key pressed.	



## 12 The menu



Access to service menu "tCH"is locked in verified weighing scales. To disable the access lock, destroy the seal and actuate the adjustment switch. For position of adjustment switch, see chap. 16.

### Attention:

After destruction of the seal the weighing system must be re-verified by an authorised agency and a new verification wire/seal mark fitted before it can be reused for applications subject to verification.

# 12.1 Navigation in the menu

Call up menu Select function	<ul> <li>⇒ In weighing mode, press and the first function [F1 oFF] will be displayed.</li> <li>⇒ With help of , the individual functions can be</li> </ul>
Change settings	<ul> <li>⇒ Confirm selected function by pressing setting will be displayed.</li> <li>⇒ To select the desired setting, press and confirm your selection by selection by the weighing scale will return to the menu.</li> </ul>
Exit menu/ Return to weighing mode	⇒ Press and the scales will return to weighing mode.

#### 12.2 Menu overview

Menu item Submenu	Available settings / explanation	
oFF 0*	Automatic shutdown off	
oFF 3	Automatic shutdown after 3 min	
oFF 5	Automatic shutdown after 5 min	
oFF 15	Automatic shutdown after 15 min	
oFF 30	Automatic shutdown after 30 min	
oFF	Not documented	
Prt		
Pr ACC		
	off 0* off 3 off 5 off 15 off 30  off	



## 1. RS-232 mode

Interface parameter

Select desired mode by , then confirm with Weight will be added to summation memory and printed P Prt after pressing PRINT P Cont Continuous data output Not documented Series Remote control instructions: ASK W: Send all weighing results S: Send stable weighing result T: Tare Z: Zero setting P cnt 2 Not documented P Stab Automatic data output of stable weighing values Weighed result will be added automatically to P Auto summation memory and issued

#### 2. Baud rate

The currently set baud rate (b xxx) will be shown after the RS-232 mode was confirmed. Select desired Baudrate by pressing

and confirm by pressing RINT.

Available Baudrate: 600, 1200, 2400, 4800, 9600

	3. Data output format  (P Prt, P Auto, P Cont settings only) the currently set data output format will be shown after the baud rate was confirmed. Select desired format by confirm with PRINT.  PRINT CONFIRM PRINT			ed format by and
		Cont 1	Default	Sd0 – on/off Cont. data outputselectable "send 0", yes / no
	Only when set P Cont	Cont 2	Not documented	
	Only v P Co	Cont 3	Not documented	
	After to	r type will to the desire	utput format has been conce displayed.  ed printer type by and documented this setting	PRINT
E4 55	bl on		Back lighting for display on	
Background	bl oFF		Display background illumination off	
illumination	bl AU*		Backlighting for display will come on automatically as soon as the weighing scale is operated.	
	Str on		Subaguant tara ON	
Subsequent tare value locked in devices with type approval certificate.	Str oFF*		Subsequent tare ON  Subsequent tare OFF	

E [ H Service menu	Pin	Password entry: press , TARE, M or BMI subsequently.
	Operate adjustmen	t switch; for position see chap.17
P : SPd	15*	
Display speed	30	Niet de cum ente d
Display speed	60	Not documented
	7.5	
P2 (AL	Adjustment, see o	chap. 18
P3 Pro	tri*	Not documented
(F) FF0	CoUnt	Not documented
	rESEt	Reset weighing scale to factory setting
	SEtGrA	Not documented

<sup>\*</sup> default setting

# 13 Data output RS 232

You can print weighing data automatically via the RS 232 interface or manually by pressing via the interface according to the setting in the menu.

This data exchange is asynchronous using ASCII - Code.


The following conditions must be met to provide successful communication between the weighing balance and the printer.

- Use a suitable cable to connect the weighing balance to the interface of the printer. Faultless operation requires an adequate KERN interface cable.
- Communication parameters (baud rate, bits and parity) of balance and printer must match. For detailed description of interface parameters see chap. 12.1)



In a medical context only auxiliary equipment in compliance with Directive EN 60601-1 may be connected to the interface.

# 13.1 Pin allocation of balance output bushing:

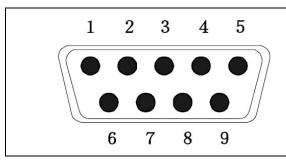

Pin 2: TXD - Output Pin 3: RXD - Input

Pin 5: GND - Signal ground

#### 13.2 Technical data

Connection 9 pin d-subminiature bushing

Pin 2 output Pin 3 input

Pin 5 signal earth

Baud rate Optional 600/1200/2400/4800/9600

Parity no

Databits 8 bits

Stopbits 1 bit

# 13.3 Printer operation

Printout examples:

Prt	
0/2	60.0kg
1/3	60.0kg 170.0cm 20.7BMI

### Remote control instructions:


S٠

29.03.2017 29.03.2017	09:31:21: 09:31:51:	ST ST	20.0kg - 20.0kg	stable weighing value positive stable weighing value negative
W:				
29.03.2017	09:32:25:	US	44.3kg	instable weighing value positive
29.03.2017	09:35:33:	US	- 18.4kg	instable weighing value

# 14 Error messages

# Display

# **Description**

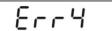


# Zero range exceeded

(on start-up or when pressing the key)

- Load on weighing pan
- Excess load, during zero setting of weighing scale

negative

- Incorrect adjusting process
- Fault on load cell

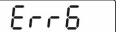

# Value outside the A/D converter range

- Damaged weighing cell
- Damaged electronics



### Unable to initialise zero point

- · Measuring cell defective / overloaded
- Object on weighing pan / contact
- Main board defective

Should other error messages occur, switch balance off and then on again. If the error message remains inform manufacturer.

# 15 Service, maintenance, disposal

### 15.1 Cleaning



Before any maintenance, cleaning and repair work disconnect the appliance from the operating voltage.

# 15.2 Cleaning / disinfecting

Clean weighing platform (such as seat) as well as casing with household detergents or commercially available disinfectants, e.g. 70% isopropanol. We recommend a disinfectant suitable for wiping disinfection. Please follow manufacturer's instructions.

Do not use abrasive or aggressive cleaners such as spirits or alcohol or similar as they might damage the high-quality surface.

To prevent cross-contamination (fungal skin infection) please observe the following time intervals for disinfection:

- Weighing plate before and after any measurement with direct skin contact
- When required:
  - o Display
  - Touch-sensitive keyboard



Do not spray disinfectants onto appliance.

Make sure that disinfectant does not penetrate the interior of the balance.

Remove dirt immediately.

### 15.3 Sterilisation

Sterilisation of the appliance not allowed.

### 15.4 Service, maintenance

The appliance may only be opened by trained service technicians who are authorized by KERN.

We recommend regular verification of conformity with technical safety requirements (STK).

Disconnect the scales before opening.

### 15.5 Disposal

Disposal of packaging and appliance must be carried out by operator according to valid national or regional law of the location where the appliance is used.

# 16 Instant help

In case of an error in the program process, briefly turn off the balance and disconnect from power supply. The weighing process must then be restarted from the beginning.

### Fault Possible cause

The displayed weight does • The balance is not switched on. not glow.

- The mains supply connection has been interrupted (mains cable not plugged in/faulty).
- Power supply interrupted.
- Rechargeable battery/batteries inserted incorrectly or empty
- No rechargeable battery/batteries inserted

The displayed weight permanently changing

- is Draught/air movement
  - Table/floor vibrations
  - The weighing plate is in contact with foreign bodies or is not correctly positioned.
  - Electromagnetic fields / static charging (choose different location/switch off interfering device if possible)

The weighing result obviously incorrect

- is The display of the balance is not at zero
  - Adjustment is no longer correct.
  - Great fluctuations in temperature.
  - Warm-up time was ignored.
  - Electromagnetic fields / static charging (choose different location/switch off interfering device if possible)

Should other error messages occur, switch balance off and then on again. If the error message remains inform manufacturer.

### 17 Verification

#### General introduction:

According to EU directive 2014/31/EC balances must be officially verified if they are used as follows (legally controlled area):

- a) For commercial transactions if the price of goods is determined by weighing.
- b) For the production of medicines in pharmacies as well as for analyses in the medical and pharmaceutical laboratory.
- c) For official purposes
- d) For manufacturing final packages
- e) Determination of mass in the practice of medicine that is, weighing patients for reasons of medical supervision during medical surveillance, examination and treatment.

In cases of doubt, please contact your local trade in standard.

### **Verification notes:**

An EU type approval exists for balances described in their technical data as verifiable. If a balance is used where obligation to verify exists as described above, it must be verified and re-verified at regular intervals.

Re-verification of a balance is carried out according to the respective national regulations. For verification validity period, s. chap. 17.1.

The legal regulation of the country where the balance is used must be observed!




#### Verification of the balance is invalid without the seal.

The seal marks attached on balances with type approval point out that the balance may only be opened and serviced by trained and authorised specialist staff. If the seal mark is destroyed, verification looses its validity. Please observe all national laws and legal regulations. In Germany a reverification will be necessary.

# Balances with obligation to verify must be taken out of operation if:

- The **weighing result** of the balance is outside the **error limit.** Therefore, in regular intervals load balance with known test weight (ca. 1/3 of the max. load) and compare with displayed value.
- The reverification deadline has been exceeded.

# Position adjustment switch and seals:

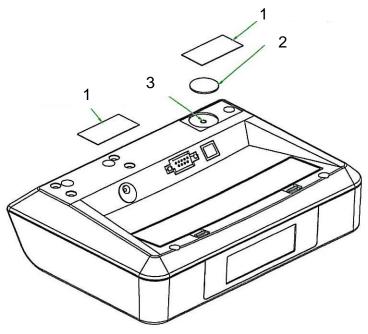

- 1. Self-destroying seal mark
- 2. Cover
- 3. Adjustment switch

# 17.1 Verification validity period (current status in G)

Personal scales (including chair and wheelchair scales) in hospitals	4 year
Personal scales, when not located in hospitals (for example, doctor's offices and nursing homes)	unlimited
Baby weighing scales and mechanical birth weight scales	4 year
Bed scales	2 year
Scales in dialysis stations	unlimited

Rehab clinics and health authorities are treated as hospitals. (4 years of verification validity)

Not treated as hospitals (verification validity not limited) are dialysis stations, nursing homes and doctor's surgeries.

(Details derived from: "Information by the verification authority, weighing scales applied in medical use")

# 18 Adjustment

As the acceleration value due to gravity is not the same at every location on earth, each display unit with connected weighing plate must be coordinated - in compliance with the underlying physical weighing principle - to the existing acceleration due to gravity at its place of location (only if the weighing system has not already been adjusted to the location in the factory). This adjustment process must be carried out for the first commissioning, after each change of location as well as in case of fluctuating environment temperature. To receive accurate measuring values it is also recommended to adjust the display unit periodically in weighing operation.



- Prepare the required adjustment weight. The adjustment weight to be applied depends on the capacity of a weighing scale, see chap. 1.
   Carry out adjustment as closely as possible to admissible maximum load of weighing scale. Information about test weights you will find in the internet under http://www.kern-sohn.com
- Observe stable environmental conditions. For warm-up time required for stabilisation see chap. 1.




Access to service menu "tCH"is locked in verified weighing scales.

To disable the access lock, destroy the seal and actuate the adjustment switch. Position of the adjustment switch see chap. 17.

#### Attention:

After destruction of the seal the weighing system must be re-verified by an authorised agency and a new verification wire/seal mark fitted before it can be reused for applications subject to verification.

#### Procedure:

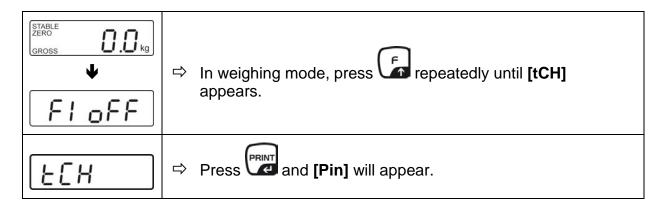

Pin	Press f, tare and or mor one after the other and [P1 SPd] will appear
PI 5Pd ♥ P2 CRL	<ul> <li>⇒ Press f, "P2 CAL" will be displayed</li> <li>⇒ Operate adjustment switch; for position see chap.16</li> </ul>
dE50	⇒ Press and [dESC] will appear
[AL	⇒ Press repeatedly until "CAL" will be displayed.
UnLoRd	<ul> <li>⇒ To confirm, press and [UnloAd] will appear</li> <li>⇒ Ensure that there are no objects on the weighing pan.</li> <li>⇒ Wait until the stability display "STABLE" is displayed and then confirm by pressing</li> </ul>
(example)	<ul> <li>⇒ The size of the currently set adjustment weight will be displayed.</li> <li>To change, select the digit to be altered by , and the numerical value by .</li> </ul>
	⇒ Confirm by pressing and [LoAd] will be displayed.

LoAd • • •	के के के	Place adjustment weight in the centre of the weighing pan Wait until stability display "STABLE" appears  Confirm by pressing [PASS] will be displayed.
	$\Rightarrow$	The balance carries out a selftest, after that <b>[Err19]</b> will be displayed and a signal will sound.
	$\Rightarrow$	Switch off the balance
	$\Rightarrow$	Take away adjustment weight
STABLE ZERO GROSS GROSS GROSS	$\Rightarrow$	Turn on balance again, after the selftest the balance changes into the weighing mode. Adjustment has now been completed successfully.

# 19 Equipment (option)

Item number	Type number	Product
YKA-43	TYKA-43-A	External mains adapter (EU, CH, UK)
YKA-44	TYKA-44-A	External mains adapter (EU)
MBC-A08	MBC-A08	Rechargeable battery
CFS-A01	CFS-A01	Interface cable RS-232