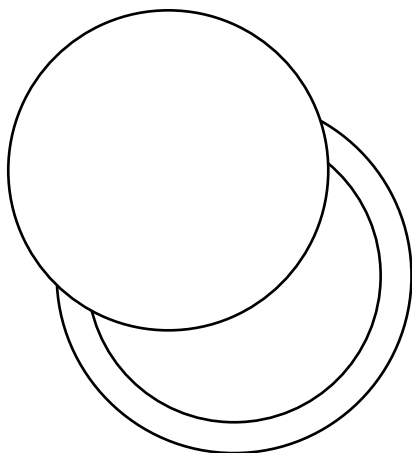
Cøpper

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für

DocCheck "Tülip"

Littmann® Master Classic II™

Littmann® Classic II Kinderstethoskop

Littmann® Classic III™

Littmann® Cardiology IV™

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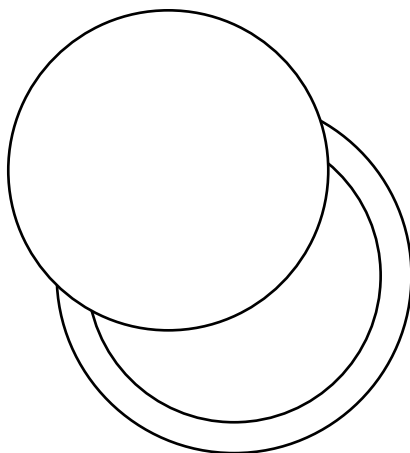
Littmann® Cardiology IV™

My intended use

I, Cøpper, am an effective antimicrobial copper membrane that you can insert into your stethoscope instead of conventional membranes. You will be able to use your stethoscope as soon as you have attached me to its chestpiece with the corresponding fastening ring.

Only use me with the stethoscope models listed on the packaging.

Effectiveness

Studies show that I can reduce bacteria and microbial activity. This is thanks to my copper surface, which reacts to microbes as soon as they touch me.

I reduce the following bacterial strains by 99.9 % within 24 hours:

- MRSA
- E. coli
- Staphylococcus aureus

However, I am of course not a substitute for responsible, hygienic treatment of your patients!

Safety information

I'm 99.9 % copper. A small proportion of the population may be allergic to this material. Do not use me if you suspect that you or your patient has such an allergy.

Be careful not to damage my surface with sharp objects. That can shorten my lifespan.

I am electrically conductive.

Any use other than described contradicts my intended purpose. My manufacturer does not assume any liability for damage of any kind caused by improper use.

Note to the user and/or the patient:

All serious incidents related to me must be reported to DocCheck Shop GmbH and the competent authority of the Member State.

Cleaning & maintenance

I can tolerate ISO alcohol just like conventional stethoscopes. Clean my surface with a smooth cloth, dampened with ISO alcohol, to remove dust and dirt.

My surface colour could change after some time because it reacts to oxygen. My beautiful copper sheen then changes to a darker brown. This is a clear signal to replace me.

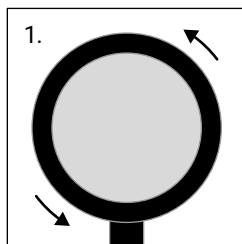
My manufacturer recommends replacing me after 30 days of normal use.

Assembly


Please test my functionality again after each replacement.

Due to my materials, my acoustic profile may differ from the original membrane supplied with your stethoscope. Please make sure you familiarise yourself with this profile before using me for diagnostic purposes.


Assembly for "Tülip"

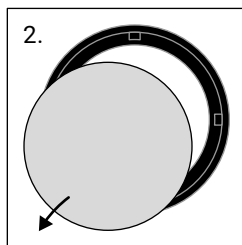
1. Release your stethoscope's locking ring by turning it counterclockwise.

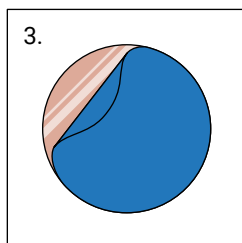
2. Remove the old membrane from the ring.

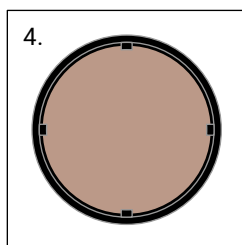3. Take me out of the packaging and carefully remove the blue protective film. I am very thin and my edges can be sharp. Therefore don't hold me by the outside edge.

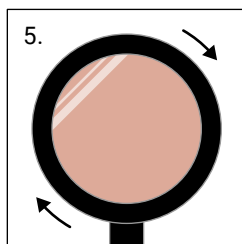
4. Place me into the ring so that I lay flat. Make sure that my shiny side is facing down – this side will come into contact with the patient later.


5. Lock me with the ring onto the chest piece, so that I fit tightly. And you're done!

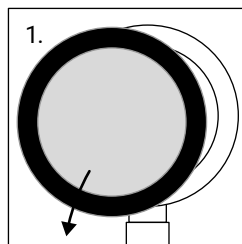
Assembly for Littmann

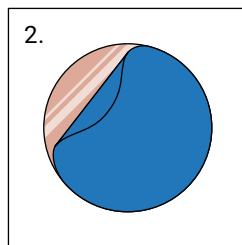
1. Remove the old membrane from your stethoscope.

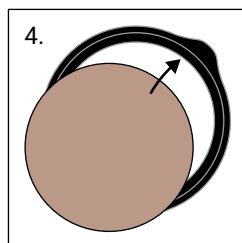
2. Take me out of the packaging and carefully remove the blue protective film. I am very thin and my edges can be sharp. Therefore, don't hold me by my outside edge.

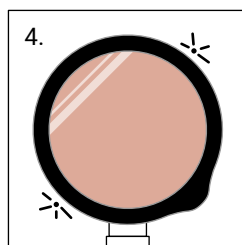3. Take the matching ring and place me flat on it. Make sure my shiny side is facing down – this side will be the one that touches the patient later.

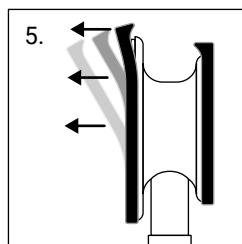4. Click me with the ring on your stethoscope's chestpiece. You're done!

5. If you want to remove me, lift the ring by the tab and pull it from the stethoscope piece by piece.

Disposal

Please only dispose of me in accordance with local / regional / international guidelines.

Manufacturer

DocCheck Shop GmbH
Vogelsanger Str. 66, 50823 Cologne, Germany
[doccheckshop.com](https://www.doccheckshop.com)

Explanation of symbols

MD Medical device

CE CE mark

Manufacturer data (address)

Production date

REF Article number

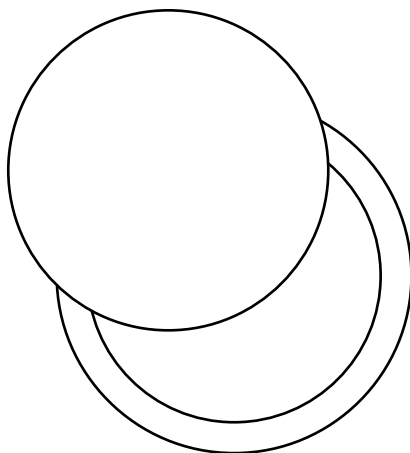
LOT Batch

Cu Contains copper

My hero
[doccheck.com](https://www.doccheck.com)

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