Instruction for use
HEINE DELTA[®]one
HEINE DELTA[®] 30
HEINE DELTA[®] 30 PRO

Instruction for use

HEINE DELTA[®]*one*

HEINE DELTA[®] 30

HEINE DELTA[®] 30 PRO

DEUTSCH	3
ENGLISH	8
FRANÇAIS	13
ESPAÑOL	18
ITALIANO	23
SVENSKA	28
NEDERLANDS	33
DANSK	38
NORSK	43
SUOMI	48
PORTUGUÊS	53

HEINE DELTA[®]one

HEINE DELTA[®] 30

HEINE DELTA[®] 30 PRO

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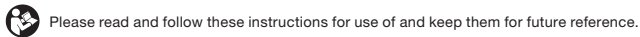
Intended use

DELTAone, DELTA 30 and DELTA 30 PRO Dermatoscope

The dermatoscope is an internally powered medical examination light for non-invasive transient contact and non-contact examination of the skin.

In contact mode, skin showing a pathological change or a change due to a disease can be examined.

The small contact plate is not suitable for use on mucosa. The examination of wounds is only permitted in non-contact mode. The contact plate must be removed for non-contact mode. The working ring is intended for the examination and instrument access of a skin section that has a pathological or disease-related change.

The contact surface of the working ring must not come into contact with this altered skin.

The product may only be used by qualified medical professionals and in professional healthcare facilities.

The application is not limited to any patient demography.

DELTAone, DELTA 30 and DELTA 30 PRO Dermatoscope with "Smartphone Adapter" option

The "Smartphone Adapter" option can be used to mechanically connect a smartphone with a respective app.

The app is designed for documentation by recording, storing and displaying images.

⚠ For U.S. only:
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and safety information

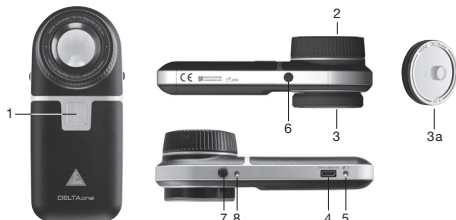
⚠ CAUTION! This symbol indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

📌 NOTE! This symbol indicates valuable advice. This notes are important, but no related to hazardous situations.

Product overview

HEINE DELTAone

- 1 On/Off switch
- 2 Focusing ring
- 3 Contact plate
- 3a Contact plate small
- 4 Micro-USB connector
- 5 Charge indicator
- 6 Brightness control (right)
- 7 Polarisation button (left)
- 8 Polarisation indicator

HEINE DELTA 30

- 9 On/Off switch
- 10 Polarisation button
- 11 Brightness control
- 12 Contact plate
- 12a Contact plate small
- 13 Focusing ring
- 14 Polarisation indicator
- 15 Micro-USB connector
- 16 Charge indicator
- 20 HEINE diSTANCE working ring

HEINE DELTA 30 PRO

- 9 On/Off switch
- 10 Polarisation button
- 11 Brightness control
- 12 Contact plate
- 12a Contact plate small
- 13 Focusing ring
- 14 Polarisation indicator
- 16 Charge indicator
- 17 Colour temperature indicator
- 18 USB-C connector
- 19 Colour change button
- 20 HEINE diSTANCE working ring

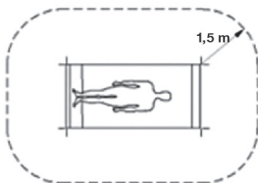
Setting up

Charge the instrument fully before first use.

Charging

Charge the device outside of the patient environment (1.5 m).

Charging via USB (DELTAone and DELTA 30)

Use a USB cable to connect the USB connector (4,15) of the dermatoscope to a USB power supply (e.g. HEINE E4-USB).

Charging via USB-C (DELTA 30 PRO)

Use the USB-C cable delivered with the DELTA 30 PRO cable to connect the USB-C connector (18) of the dermatoscope to a USB power supply.

Using the charging station (DELTA 30 and DELTA 30 PRO)

Use the charging station to charge DELTA 30 and DELTA 30 PRO. The blue LED on the charging station indicates contact with the dermatoscope. The charge indicator is located on DELTA 30 and DELTA 30 PRO.

Pay attention to the fact that no foreign objects are between the device and the charging station.

When using DELTA 30 or DELTA 30 PRO, make sure that the charging station and DELTA 30 or DELTA 30 PRO are not contaminated.

The setting up and operation of the charging station is described in a separate instruction of use.

Charge indicator

A charge indicator (5, 16) is located on the side or underside of the dermatoscope.

Colour temperature indicator

For the DELTA 30 PRO there is an indicator located on the left side of the dermatoscope, showing the selected colour temperature.

Indicator colour	Selected colour temperature
Orange	Neutral warm white
Purple	Neutral cold white
Light blue	Cold white
Deep blue	Blue cold white

DELTAone

Instrument	Charger	Battery	Charge indicator
switched off	connected	charging	green, flashing
		full	green
switched on	not connected	full ($\geq 25\%$)	green
		empty ($< 25\%$)	orange
		residual capacity ($< 10\%$)	orange, flashing

DELTA 30 and DELTA 30 PRO

Instrument	Charger	Battery	Charge indicator segments		
			Green - 1	Green - 2	Orange - 3
switched off	connected	100 %	on	on	on
		full (99 % – 66 %)	flashing	on	on
		sufficiently charged (66 % – 33 %)	off	flashing	on
		empty ($\leq 33\%$)	off	off	flashing
switched on	not connected	full ($\geq 66\%$)	on	on	on
		sufficiently charged (66 % – 33 %)	off	on	on
		empty (33 % – 10 %)	off	off	on
		residual capacity ($< 10\%$)	off	off	flashing

If the device is switched off, the charge indicator remains on for a further 3 seconds and then goes off.

 Do not look directly into the light source when checking the charge level.

Operation

To switch on the illumination, press the button (1,9) on the instrument. The device starts with the last selected brightness level and in polarisation mode.

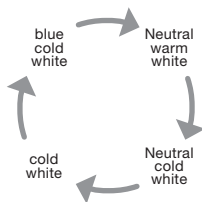
For DELTA 30 PRO: The device starts with the last selected colour temperature, last selected brightness level and in polarisation mode.

Press the button (6, 11) to adjust the brightness in three steps.

The device switches off automatically after 10 minutes if the polarisation buttons, the colour temperature buttons or the brightness control are not used.

DELTA 30 PRO colour temperature change

By pressing the button (19) labelled as "C" on the front of the DELTA 30 PRO, the colour temperature is changed forward to a more bluish colour with a high CCT value. There are 4 colour temperatures to cycle through. If the highest value is reached, the cycle starts again with neutral warm white.

HEINE DELTA 30 PRO is intended for a transient examination of less than 20 minutes with a 10 minutes break until the next application.

Contact mode (with contact plate)

Place the contact plate (3, 3a, 12, 12a) on the instrument, making sure that it snaps into place.

The working ring (20) is used instead of the contact plate (3, 3a, 12, 12a). Place the working ring on the instrument, making sure that it snaps into place. Place the instrument with the contact plate or the working ring onto the area of skin being examined.

With DELTAone, extend the contact plate (3, 3a) over the focusing ring (2).

When carrying out examinations with polarised illumination, you do NOT need any contact liquid such as immersion oil or disinfection spray. In polarised mode, the polarisation indicator (8, 14) lights up blue.

Use the polarisation button (7, 10) to switch to non-polarised mode. In this case, use a contact fluid such as immersion oil. When using the small contact plate (3a, 12a), examination is only possible in non-polarised mode.

Adjusting the focus in contact mode

Your eye needs to be at a distance of approximately 2 cm from the focusing ring (2, 13). Rotate the focusing ring until you see a sharp image. The image has 10-fold magnification.

Non-contact mode (without contact plate)

Take off the contact plate (3, 3a, 12, 12a) or the working ring (20) and hold the device at a distance of approximately 1 cm above the area of skin being examined. Use the polarisation button (7, 10) to switch to non-polarised mode.

Adjusting the focus in non-contact mode

Your eye needs to be at a distance of approximately 2 cm from the focusing ring (2, 13).

With DELTA 30 and DELTA 30 PRO, rotate the focusing ring (13) until you see a sharp image.

With DELTAone, the focus is adjusted via the distance between the skin and the optics. There is no need to rotate the focusing ring (2).

The image has 10-fold magnification.

Reset of the device

If the device cannot be operated even though the battery is sufficiently charged, press buttons 10 and 11 for 8 seconds on the DELTA 30 or buttons 10 and 19 for 8 seconds on the DELTA 30 PRO.

Smartphone Adapter

To connect the dermatoscope to a smartphone, you need a suitable adapter for your smartphone.

Place the adapter onto the focusing ring (2, 13). The adapter is held in place by magnetic force.

Install the corresponding app on your smartphone. The app is available in the App Store*.

To avoid any possible malfunctions, please deactivate all other services (switch on flight mode).

*App Store is a service mark of Apple Inc.

 Protecting patient data is users' sole responsibility (e. g. protection of the smartphone with a pin, data transmission via a secure connection). Ensure that your smartphone is adequately protected against access by third parties.

An electrical conducting connection between smartphone, PC and a mains power source is not permissible during the recording of the picture.

 All images are saved exclusively on the smartphone in the app's sandbox. The user is responsible for the regular creation of a data backup to prevent data loss, which can happen for example through deleting the app or losing the smartphone. A data backup can be made via cable to another computer or via the network to the iCloud.

Hygienic reprocessing

The instruction is available:

- at www.heine.com

- in a paper version which you can request from the address listed

Maintenance

The device is maintenance free.

Service

The device has no components serviceable by the end-user.

General notes

 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

The expected life cycle amounts with designated use and the observation of warning and safety information as well as the maintenance instructions up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Note to the user and/or patient:

All serious incidents that occur in connection with the product must be reported to HEINE Optotechnik GmbH & Co. KG and the member state's competent authority.

Charge your device during longer storage periods to protect the battery from deep discharge.

Store and use the device only in a dry and dust-free environment.

To reduce the contact temperatures in high ambient conditions, the DELTA 30 and DELTA 30 PRO have a brightness reduction feature.

General warnings

 Use only CE marked USB power supplies (5 V) from reputable manufacturers that conform with the safety requirements from the IEC 60601-1 Medical electrical equipment.

Do not use power supplies where damage has been detected.

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments).

This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Do not look directly into the light source to avoid dazzle from the intense light.

The dermatoscope is not suitable for eye examination.

Disposal

 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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Electromagnetic disturbances – Requirements and tests

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	<p>Inside professional healthcare facilities except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.</p> <p>The supply voltage quality should be that of a typical hospital environment.</p> <p>Floors should be wood, concrete or covered with ceramic tiles. If floors are covered with synthetic material, the relative humidity should be at least 30 %.</p>	
Performance features of the ME system that have been determined to be essential to the performance	None	
Necessary instructions for maintaining basic safety and essential performance with regards to electromagnetic disturbances for the expected life cycle	<p>Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.</p> <p>Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.</p> <p>Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the considered device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.</p> <p>DELTA 30/DELTA 30 PRO/DELTAone: Illumination failures may occur due to electrostatic discharge.</p>	
Warning	<p>The EMISSIONS characteristics of the DELTA 30 PRO make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required), the DELTA 30 PRO might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the DELTA 30 PRO.</p>	
Note	<p>The EMISSIONS characteristics of the DELTA 30 PRO make it suitable for use in industrial areas and hospitals (CISPR 11 class A). If it is used in a residential environment (for which CISPR 11 class B is normally required), the DELTA 30 PRO might not offer adequate protection to radio-frequency communication services. The user might need to take mitigation measures, such as relocating or re-orienting the DELTA 30 PRO.</p>	
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	EMC compatibility is only ensured if original HEINE spare parts, accessories and power sources are used as described in the chapter "Options". The EMC compatibility when using power sources from other manufacturers must be evaluated by the user.	
Test	Compliance	
RF emissions CISPR11	DELTA 30 / DELTAone	DELTA 30 PRO
	Group 1 Class B	Group 1 Class A
Conducted emissions (EN 55011/CISPR 11)*	Passed	
Radiated emissions (EN 55011/CISPR 11)		
Harmonic current emissions (IEC 61000-3-2)*		
Voltage changes, voltage fluctuations and flicker (IEC 61000-3-3)*		
Immunity	See attached immunity test levels	

Immunity test levels

Test	Test level	
	IEC 60601-1-2 test levels	Compliance test levels
Electrostatic Discharge (IEC 61000-4-2)	Contact Discharge: ± 8 kV Air Discharge: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV	
Radiated RF EM fields (IEC 61000-4-3)	3 V/m 80 – 2700 MHz 80 % AM at 1 kHz	
Electrical fast transients / bursts (IEC 61000-4-4)*	± 2 kV 100 kHz repetition frequency	
Surges (IEC 61000-4-5)*	± 0.5 kV, ± 1 kV	
Conducted disturbances induced by RF fields (IEC 61000-4-6)*	3 V 0.15 MHz – 80 MHz 6 V in ISM bands between 0.15 MHz and 80 MHz 80 % AM at 1 kHz	
Proximity fields from RF wireless communications equipment (IEC 61000-4-3)	385 MHz; Pulse Modulation: 18 Hz; 27 V/m 450 MHz, FM: ± 5 Hz deviation: 1 kHz sine; 28 V/m 710, 745, 780 MHz; Pulse Modulation: 217 Hz; 9 V/m 810, 870, 930 MHz; Pulse Modulation: 18 Hz; 28 V/m 1720, 1845, 1970 MHz; Pulse Modulation: 217 Hz; 28 V/m 2450 MHz; Pulse Modulation: 217 Hz; 28 V/m; 5240, 5500, 5785 MHz; Pulse Modulation: 217 Hz; 9 V/m	
Power frequency magnetic fields (IEC 61000-4-8)	30 A/m; 50Hz or 60 Hz	DELTA 30/DELTA 30 PRO: Compliance test levels are identically to the IEC 60601-1-2 test levels DELTAone: Test was not necessary, because there are no magnetic sensitive components.
Voltage dips (IEC 61000-4-11)*	0 % U_T ; 0.5 cycle; at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° 0 % U_T ; 1 cycle and 70 % U_T ; 25/30 cycles Single phase: at 0°	DELTA 30/DELTA 30 PRO: Compliance test levels are identically to the IEC 60601-1-2 test levels DELTAone: Test was not performed, because it was not considered necessary. No risks were identified.
Short interruptions (IEC 61000-4-11)*	0 % U_T ; 250/300 cycles	

*n/a: "Not applicable" in the internally powered mode

Technical specification

Environmental specification DELTAone, DELTA 30 and DELTA 30 PRO

Environmental conditions for operation	DELTA 30/DELTAone +10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa	DELTA 30 PRO +10 °C to +35 °C 10 % to 75 % non-condensing 700 hPa to 1060 hPa
Environmental conditions for storage	DELTA 30/DELTAone +5 °C to +45 °C 45 % to 80 % rel. humidity	DELTA 30 PRO Less than 1 month: +5 °C to +50 °C Less than 3 months: +5 °C to +40 °C Less than 1 year: +5 °C to +20 °C rel. humidity < 70 % 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa	

Technical specification DELTAone

Power supply	USB 2.0 Micro B: 5 V DC, 650 mA
Power consumption	max. 3.25 W
Class	Charging: II Operation: internally powered
Charging time	max. 75 min
Operating time	typ. 80 min at highest brightness in polarisation mode
Device classification according to IEC 62471	Risk group 1
Applied part	Contact plate type BF

Technical specification DELTA 30 and DELTA 30 PRO

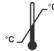
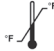
Power supply	DELTA 30 USB 2.0 Micro B: 5 V DC, 1.2 A HEINE Charger 30	DELTA 30 PRO USB-C: 5 V DC, 1.2 A HEINE Charger 30
Power consumption	max. 6 W	max. 6 W
Class	Charging: II Operation: internally powered	Charging: II Operation: internally powered
Charging time	USB: typ. 120 min. HEINE Charger 30 typ. 120 min.	USB-C: typ. 150 min. HEINE Charger 30 typ. 150 min
Operating time	typ. 160 min at highest brightness in polarisation mode	Min. 210 min. @ 5300 K, Polarised, 100 % Brightness
Device classification according to IEC 62471	Risk group 1	Risk group 1
Applied part	Contact plate, working ring type BF	Contact plate, working ring type BF

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